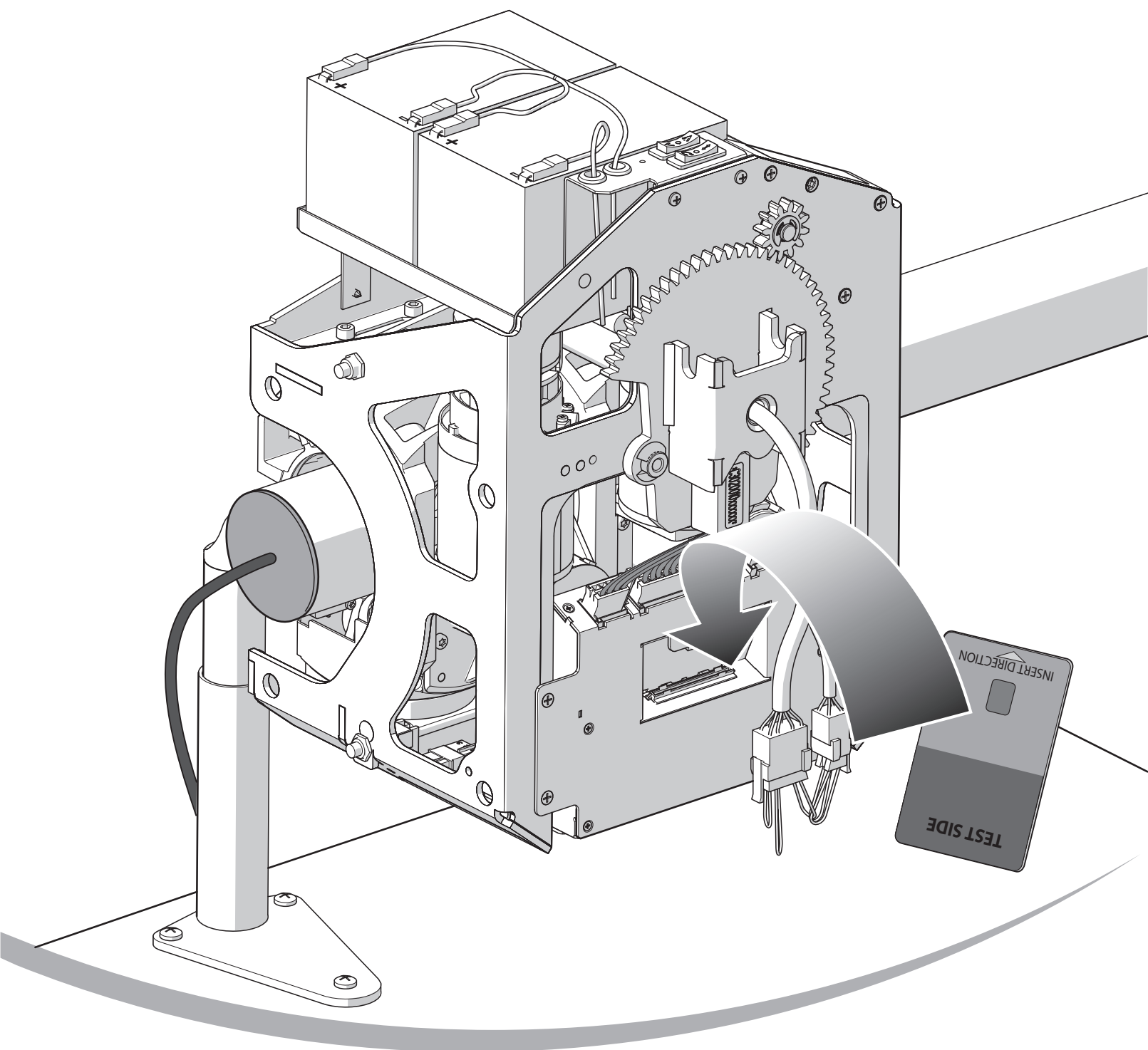
FREECURVE | INSTALLATION MANUAL

MONORAIL | MANUAL D'INSTALLATION

Classic
Elegance
Alliance

EN All rights reserved. No part of this publication may be reproduced and/or published by means of print, photocopy, microfilm or in any other way or form, without prior written permission from the manufacturer. This also applies for the accompanying drawings and diagrams. The manufacturer reserves the right to change parts at any time, without prior or direct notification to the customer. The contents of this manual may also be changed without prior warning. This manual is intended for the standard models of the lift. The manufacturer therefore cannot be held liable for any damage resulting from specifications that differ from those of the standard model of the lift supplied to you.

This manual has been compiled with the best intentions and every possible care but the manufacturer cannot accept responsibility for any errors in this manual or for the consequences thereof.

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UK

The Dutch manual is the original language

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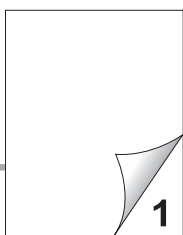
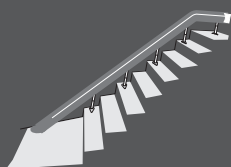
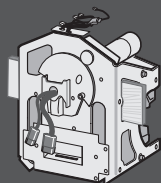
**1****2**

Thank you for choosing a Handicare stairlift.
This manual is for the Classic and Elegance/
Alliance seat types, and includes installation instruc-
tions for the standard features and optional upgrades.

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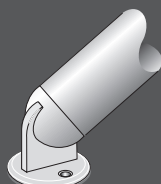
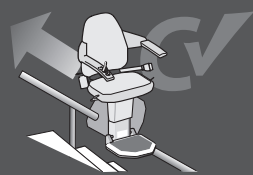
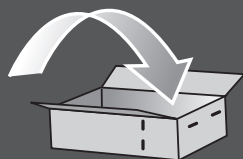
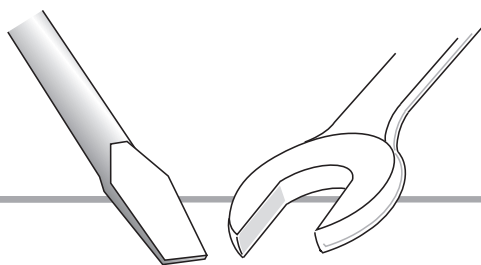
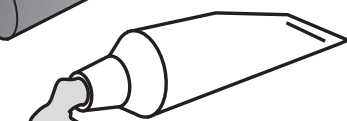
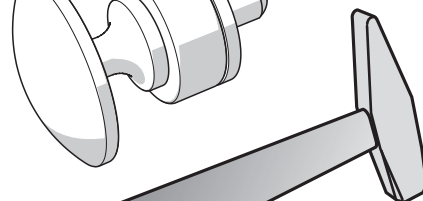
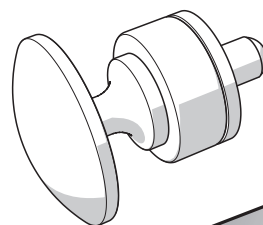
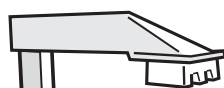
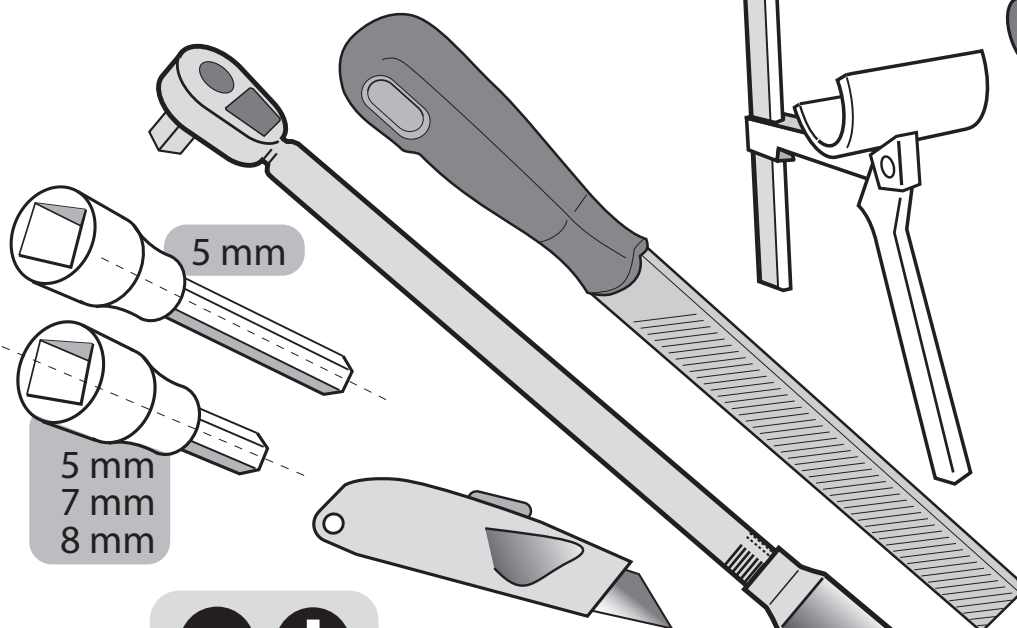
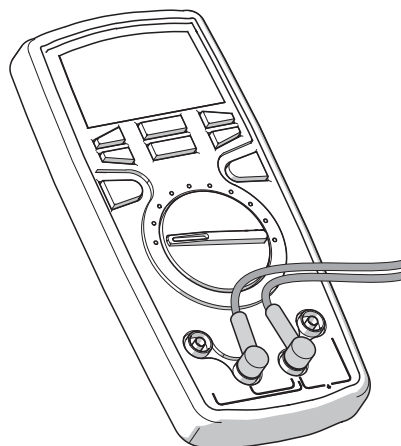
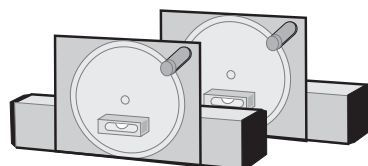
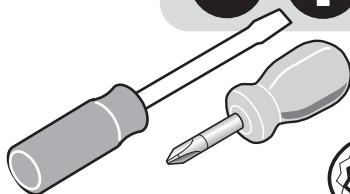
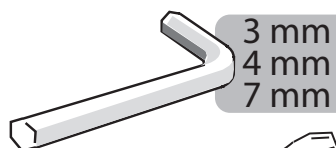
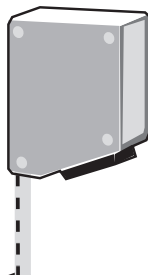
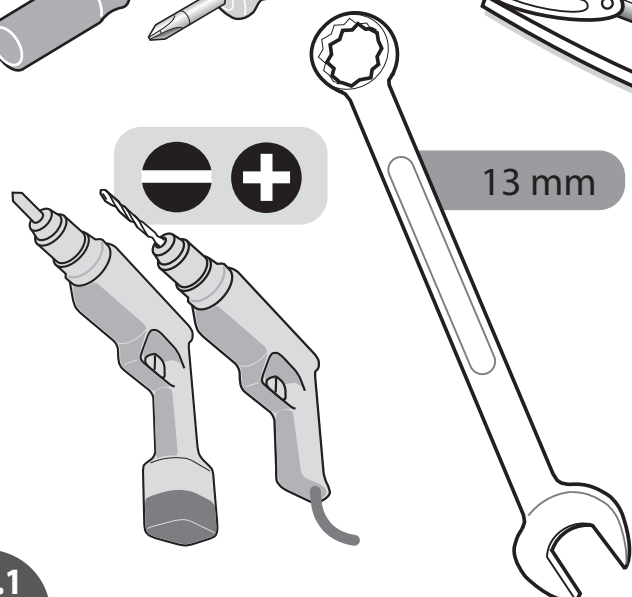
5 mm
8 mm

1.1

Only a certified technician should install this lift. The latest instructions of the lift and instructions for options are available for download at www.handicarepartners.com. The installations of the options are not all described in this manual. These instructions are provided with the corresponding option kit. Ensure good and sufficient tools; the shown tools are minimum required to install the stairlift.

UK In case of installation a new part on a existing lift always replace the part by a part with the same article number.

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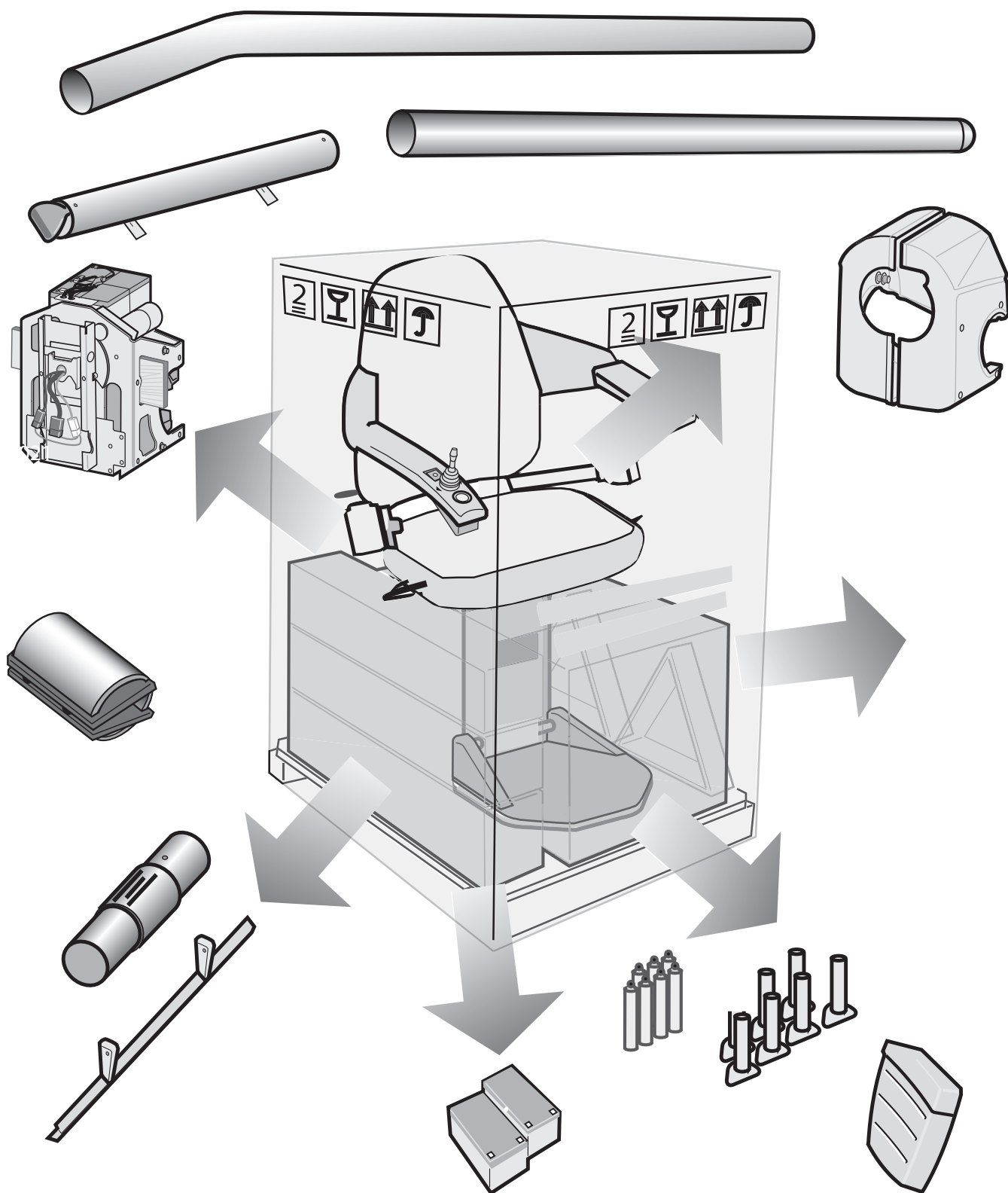
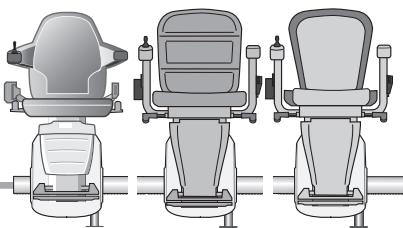
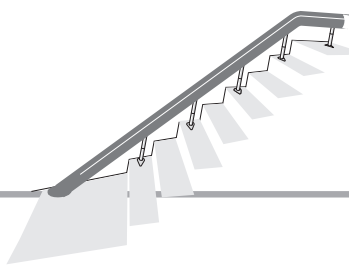
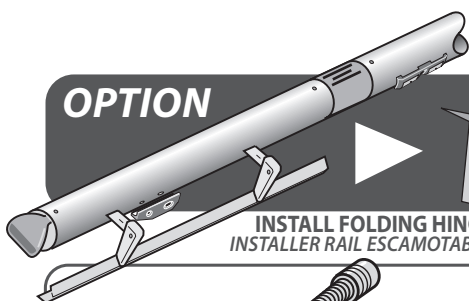
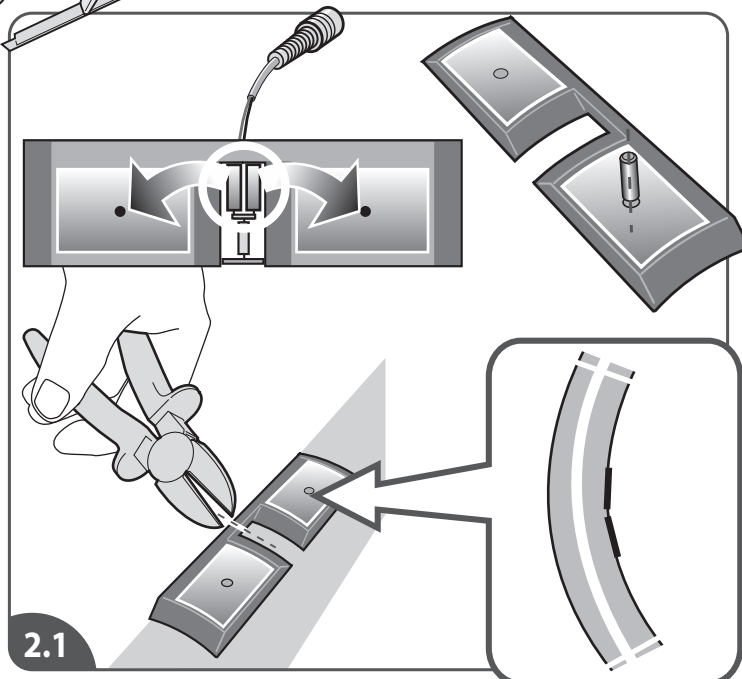
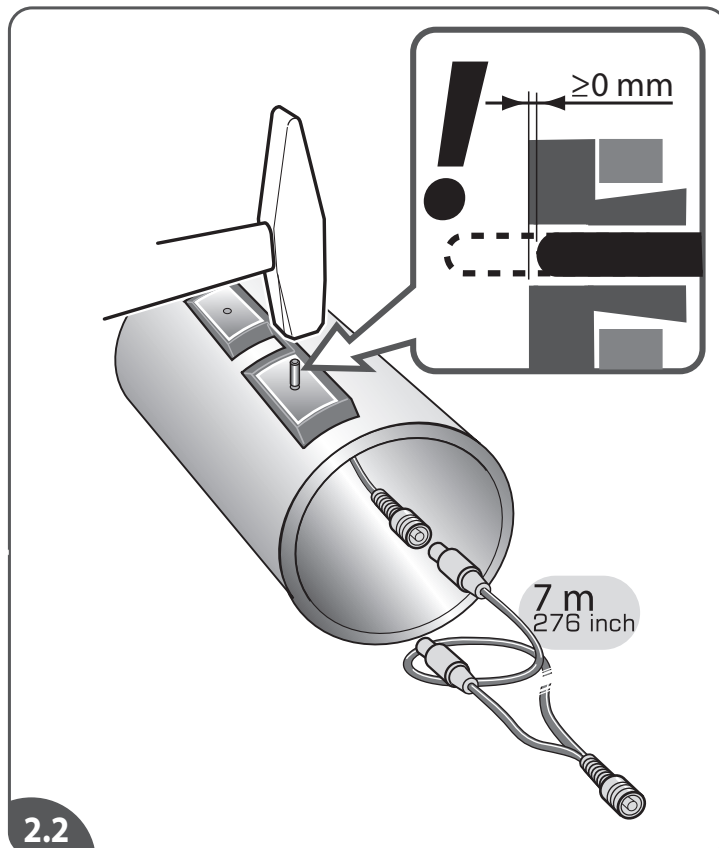
A

C0052050
C0052030

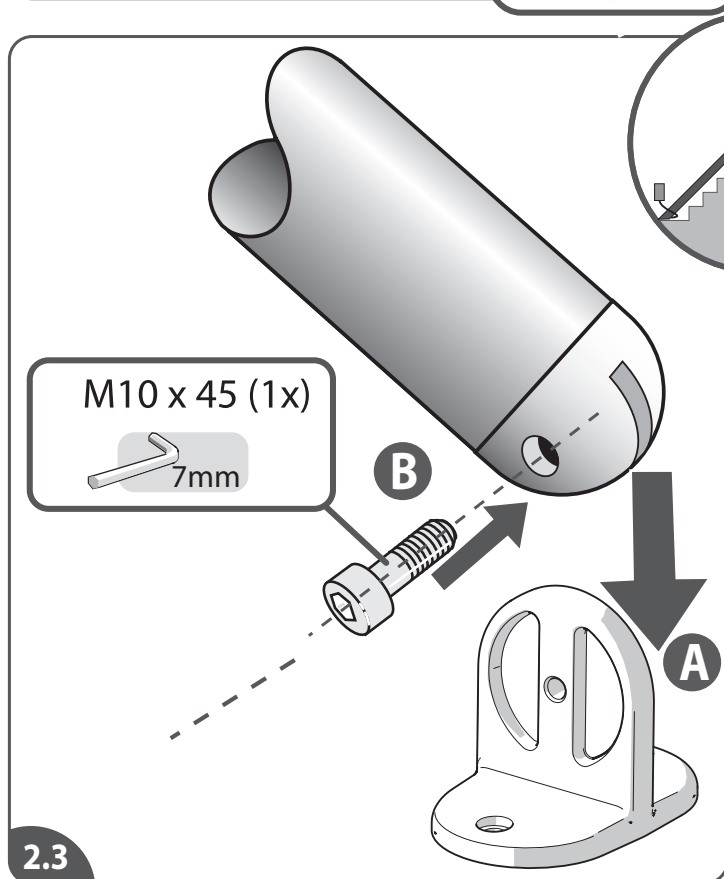
INSTALL FOLDING HINGE
INSTALLER RAIL ESCAMOTABLE

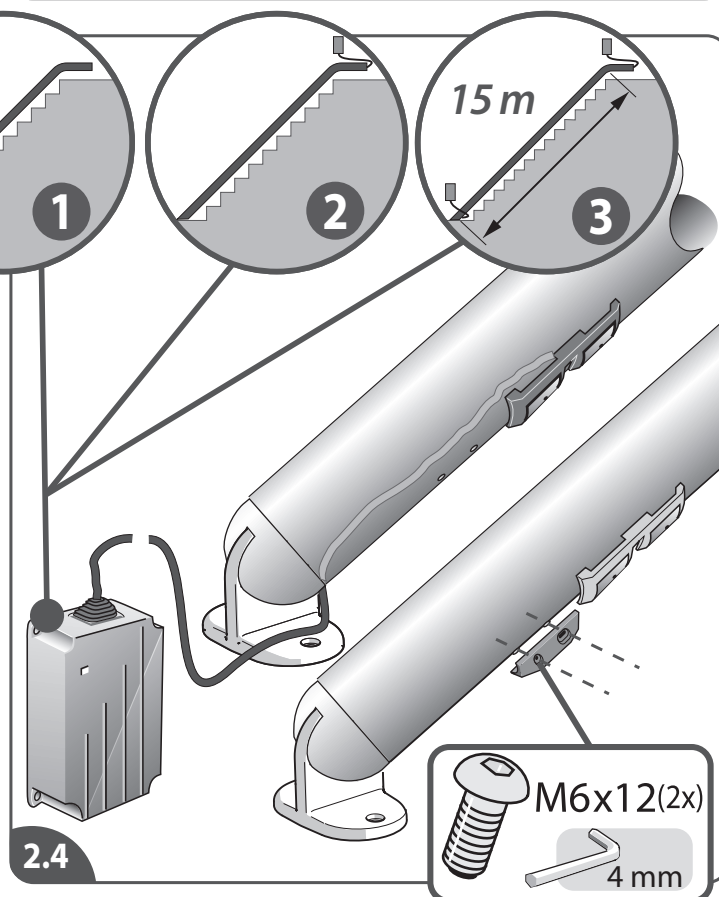
2.1

2.2

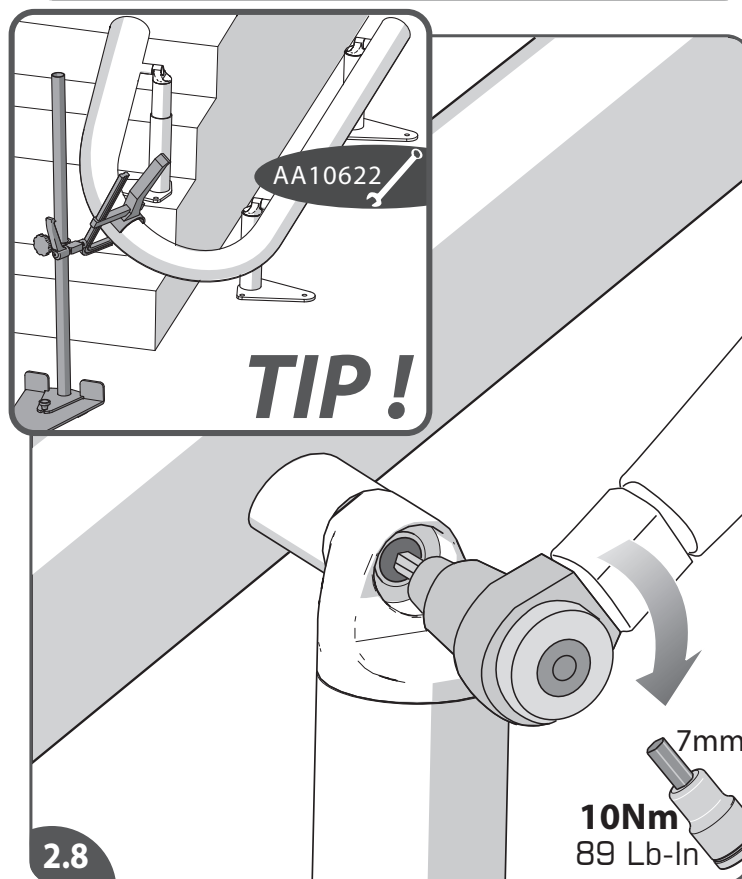
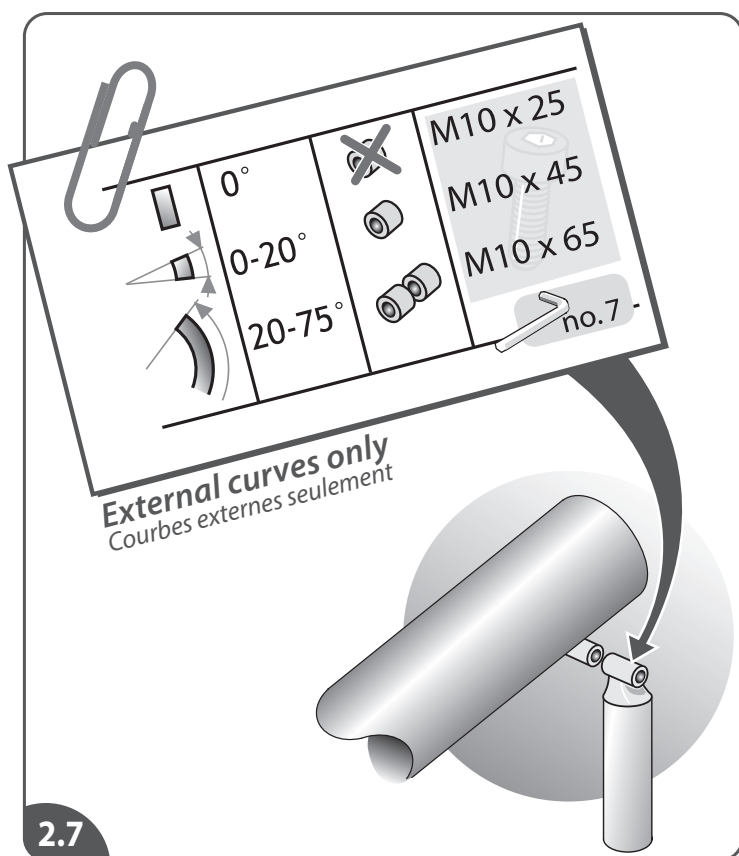
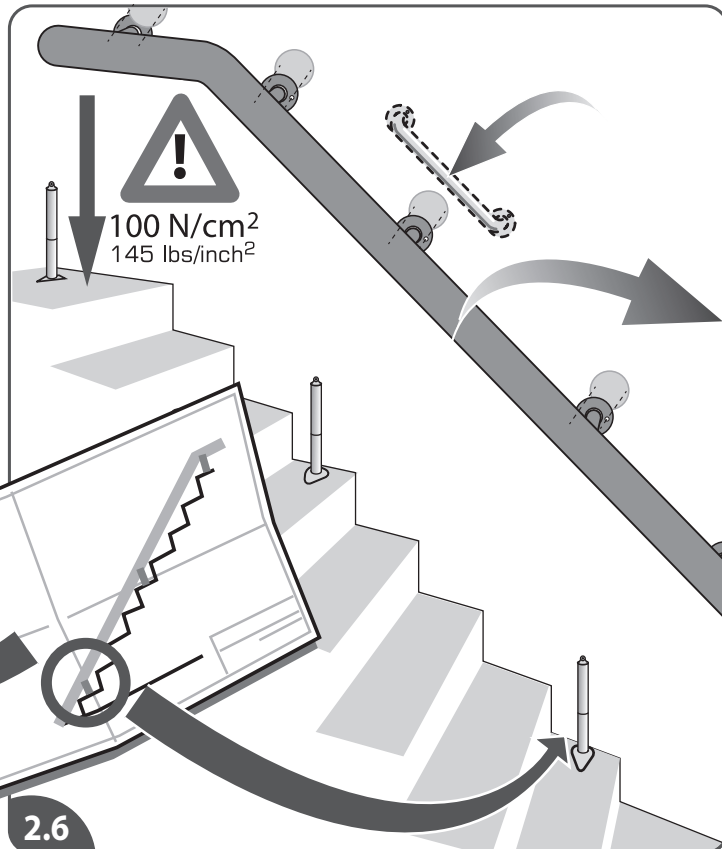
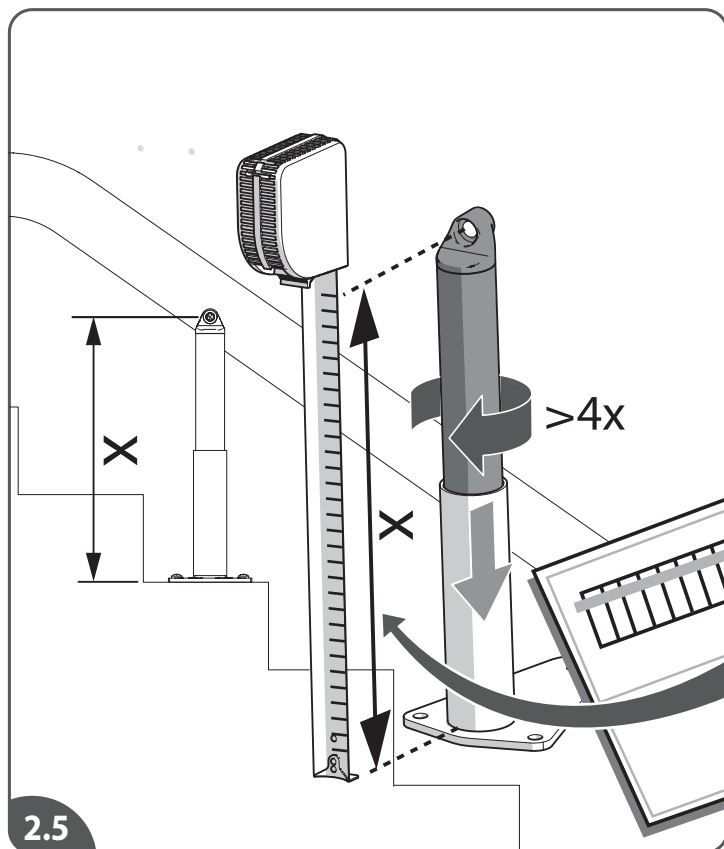
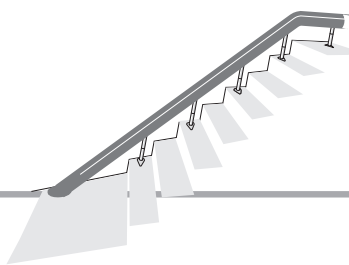
2.3

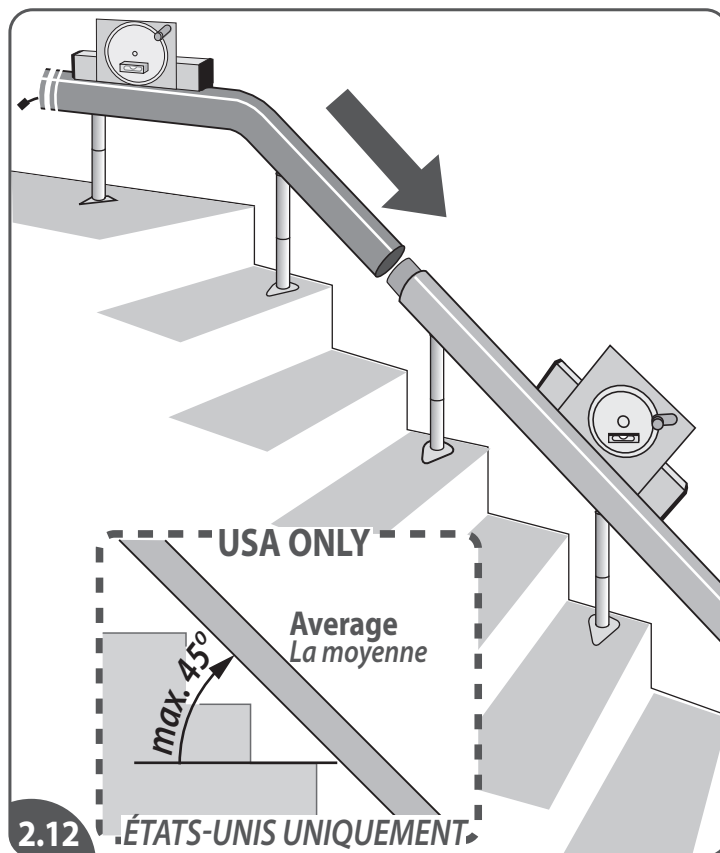
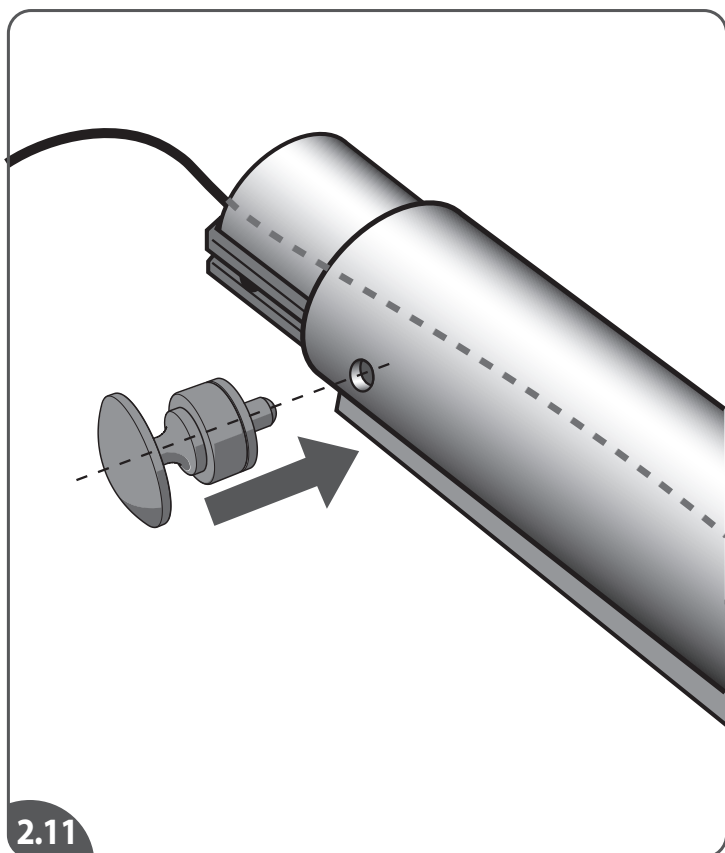
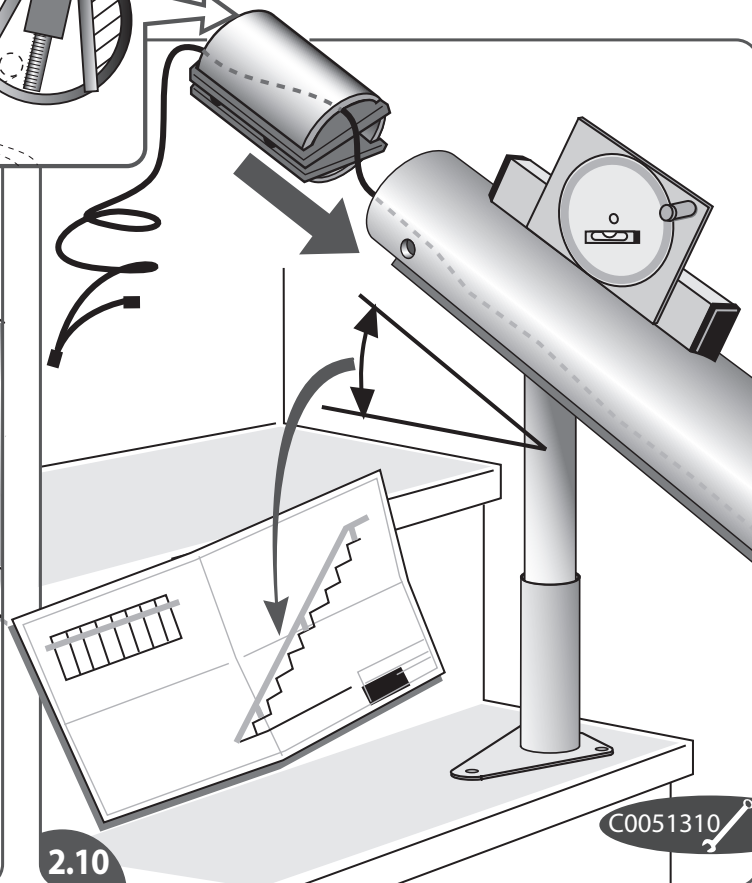
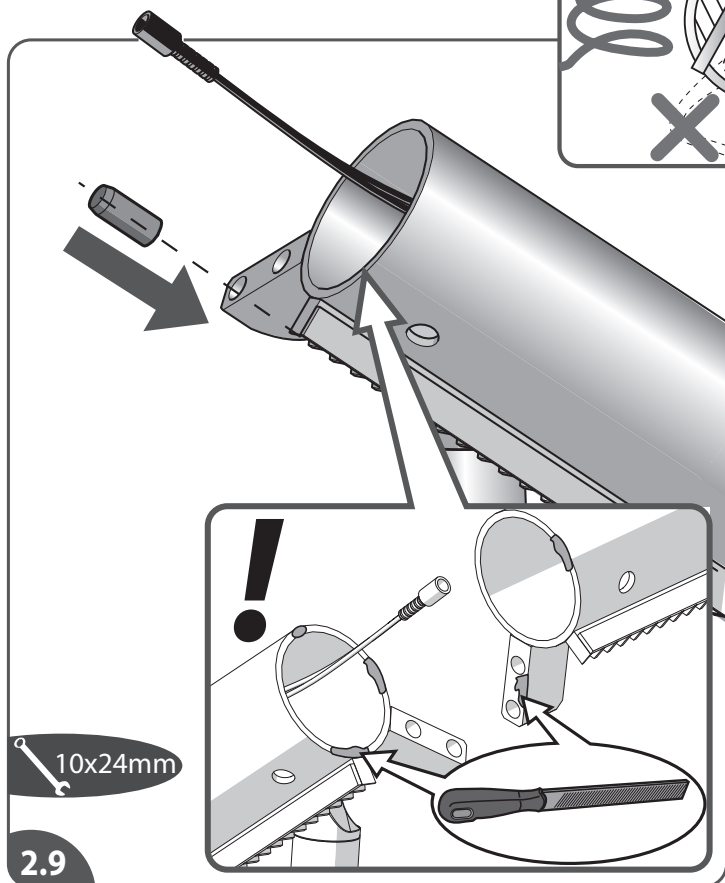
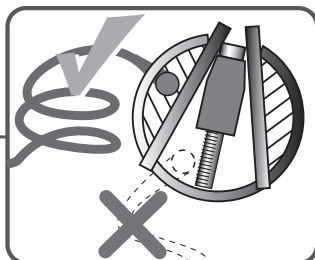
2.4

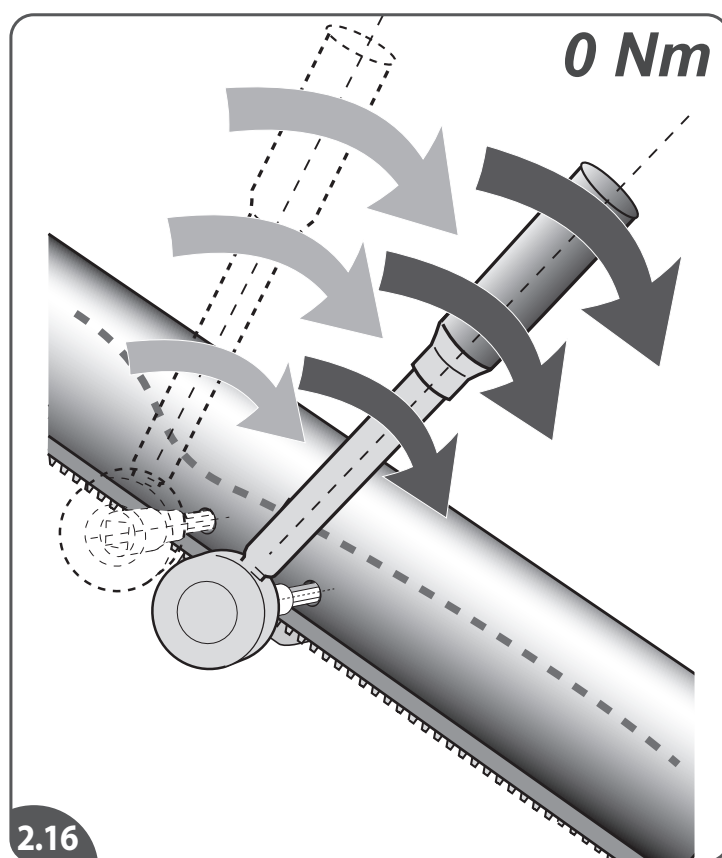
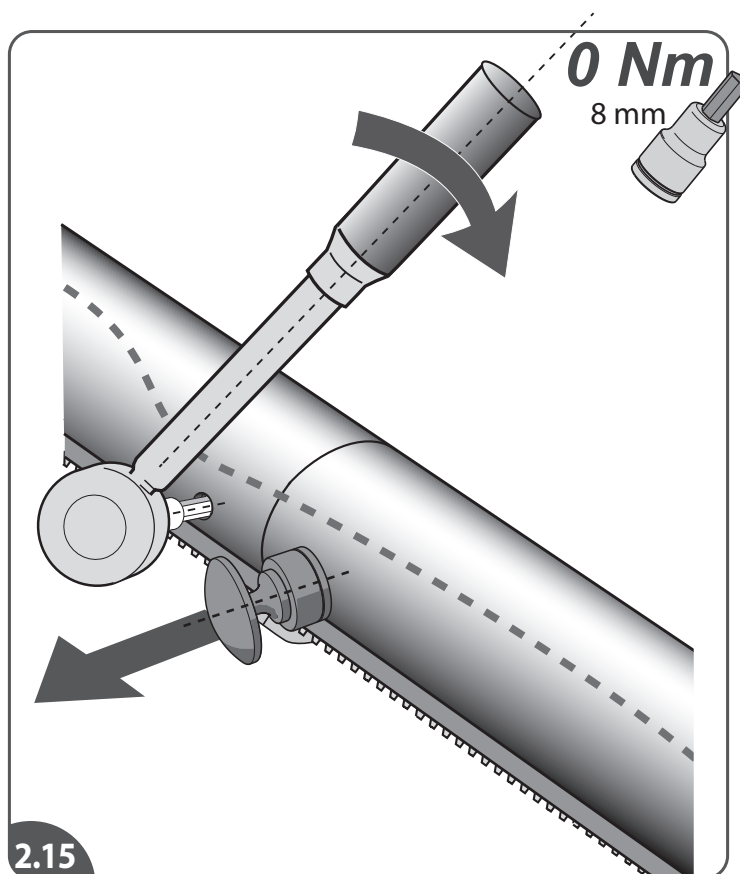
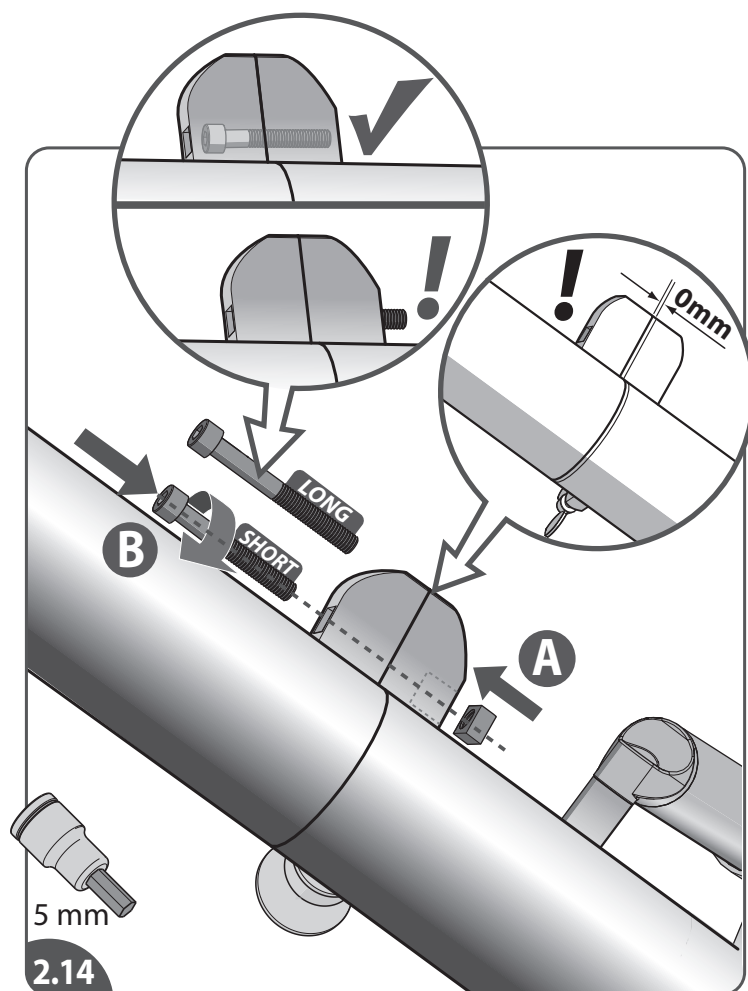
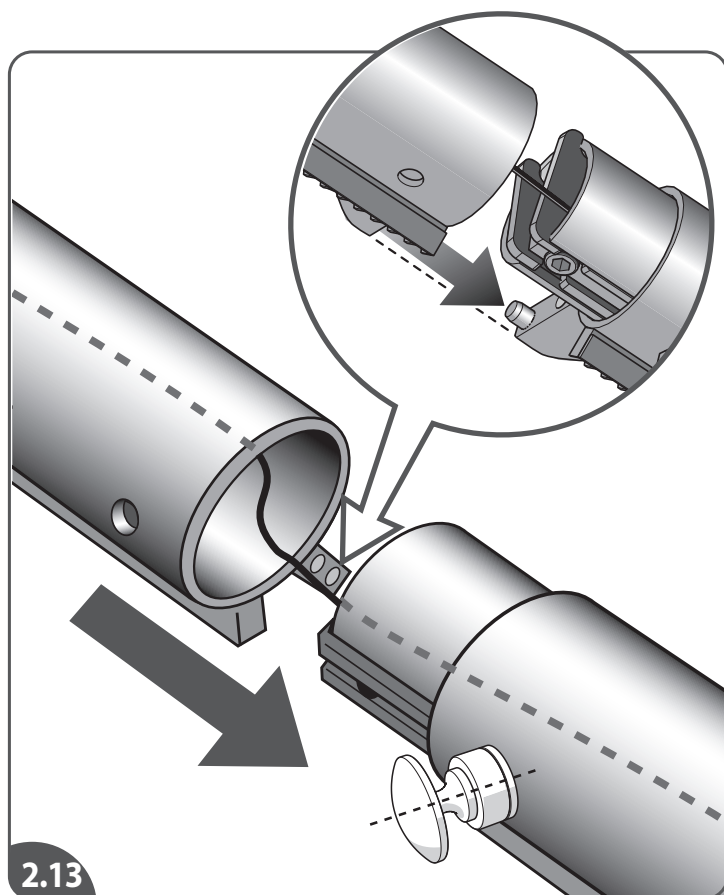
2

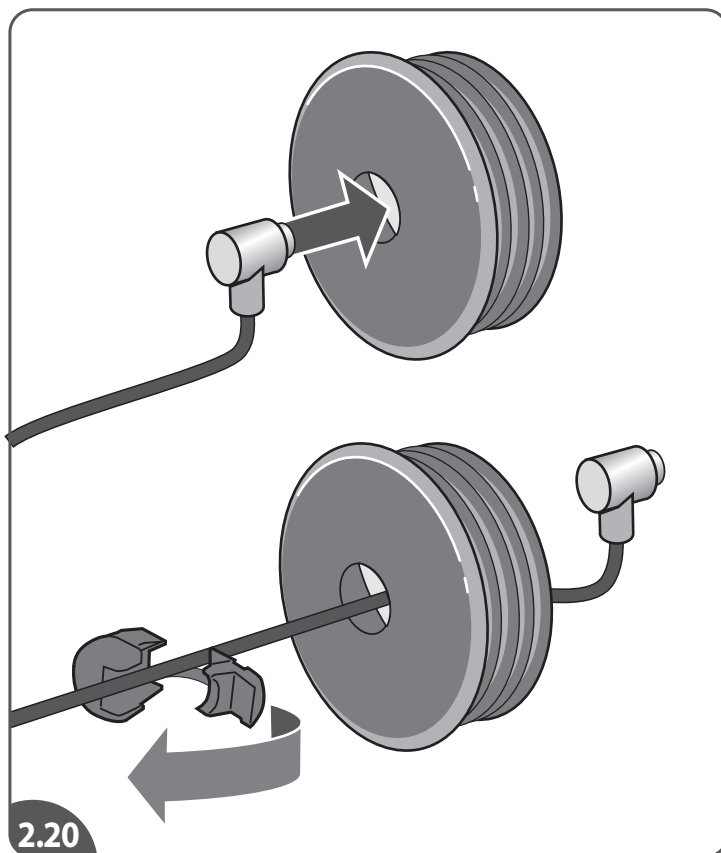
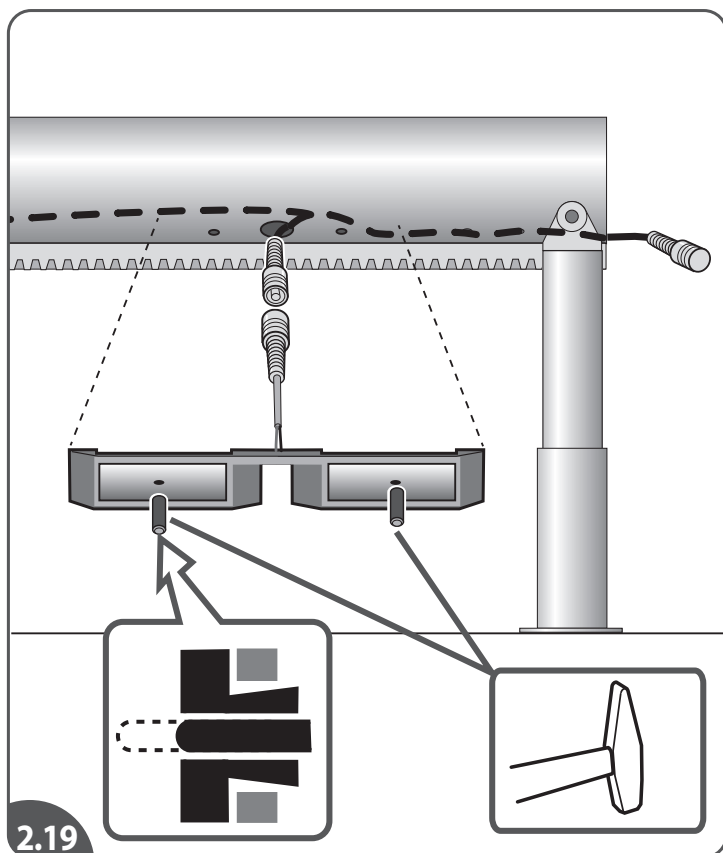
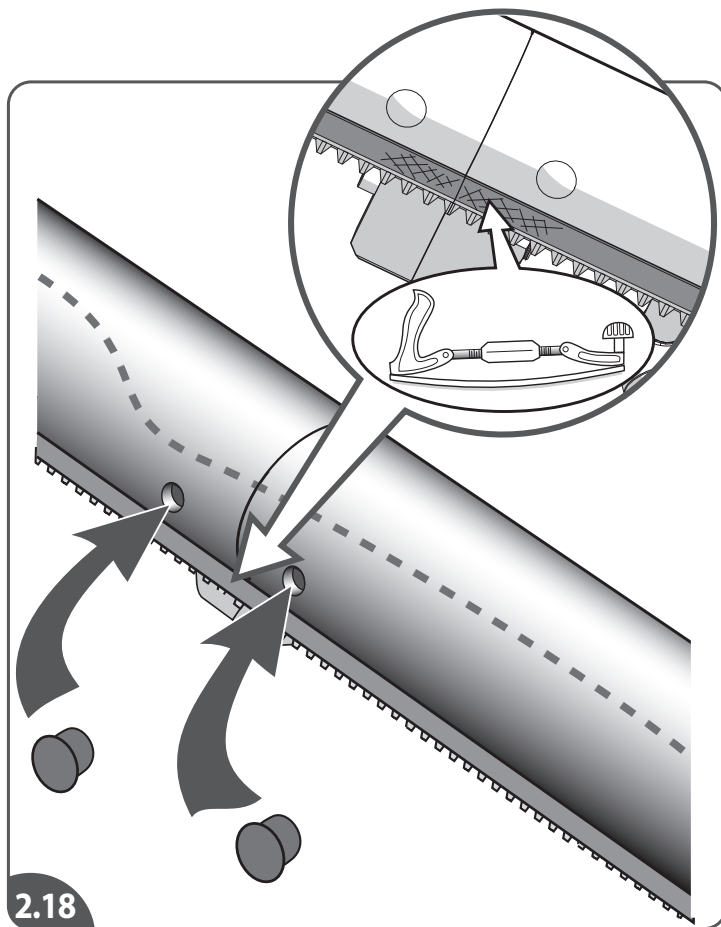
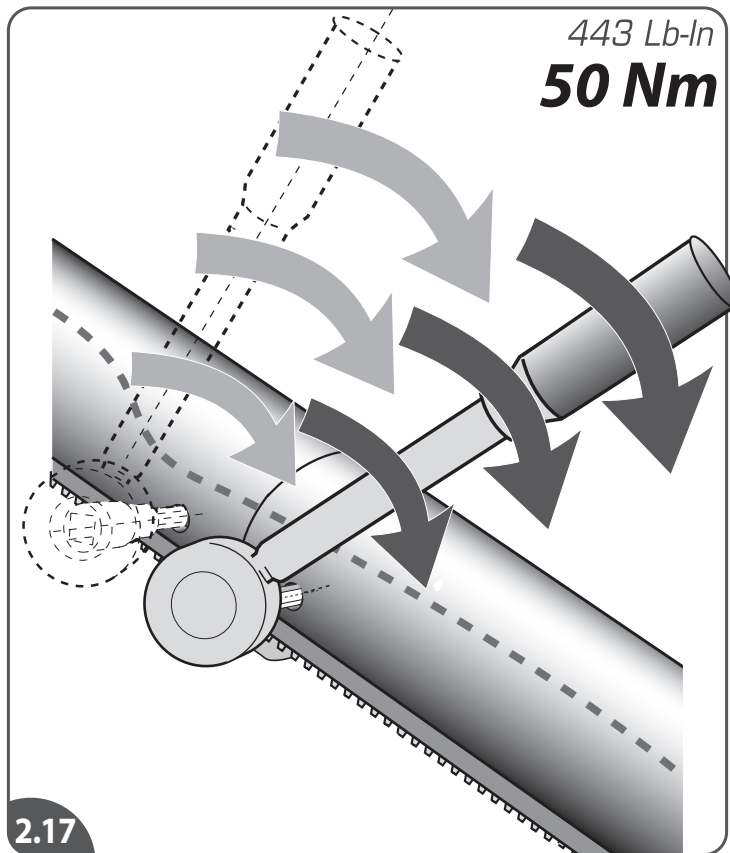
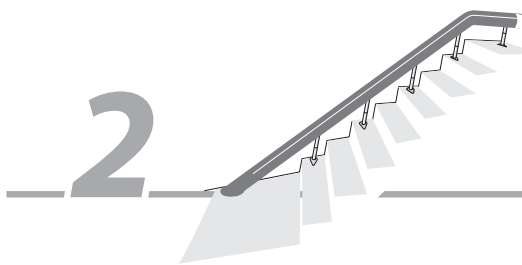
2

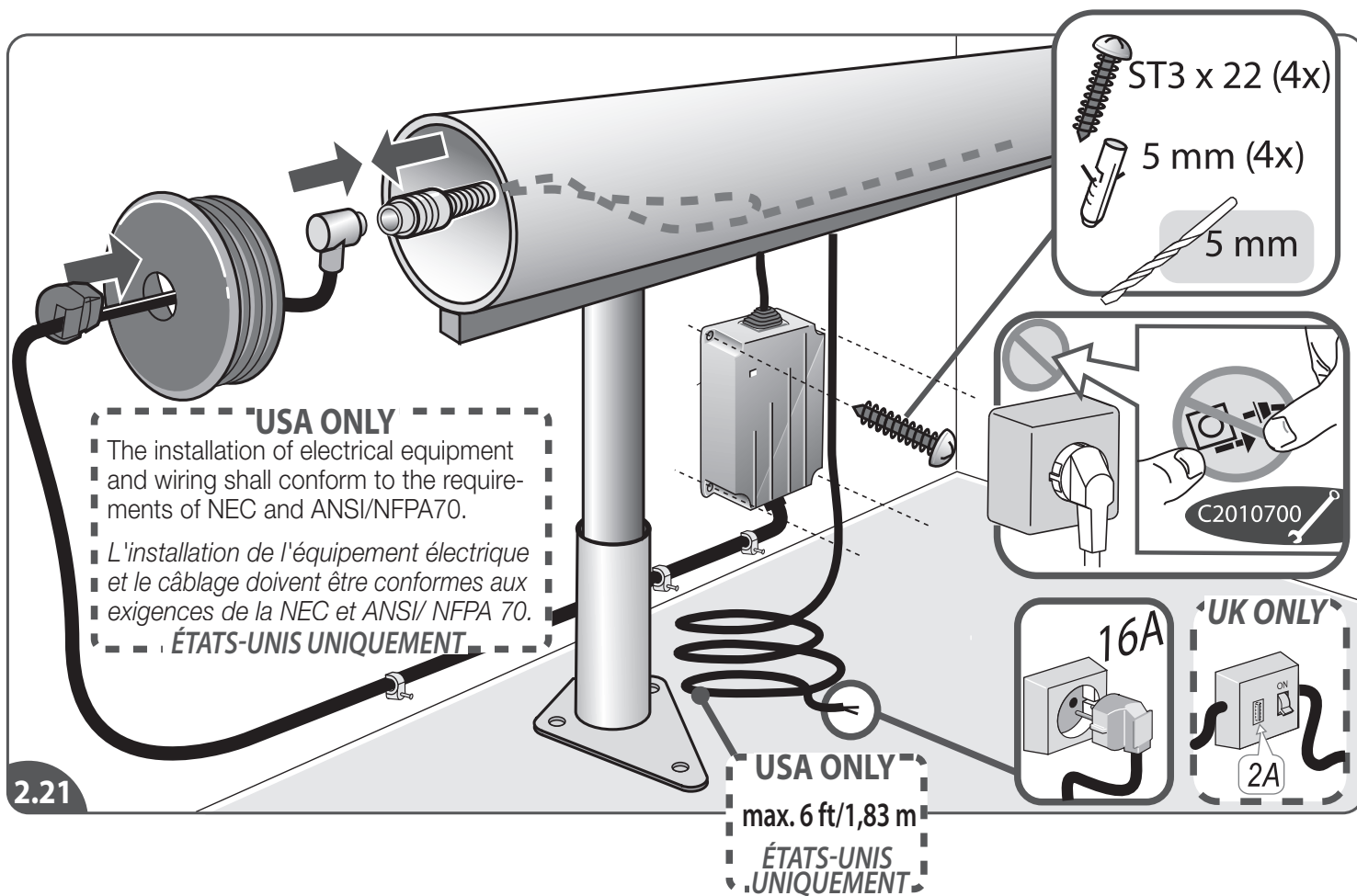
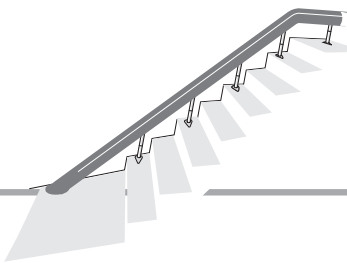
2

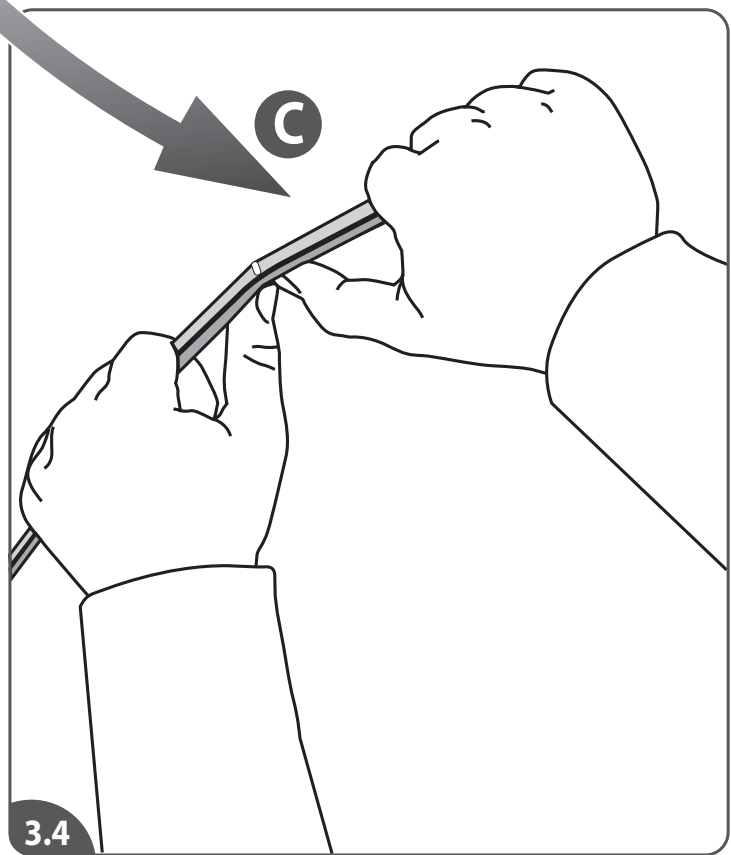
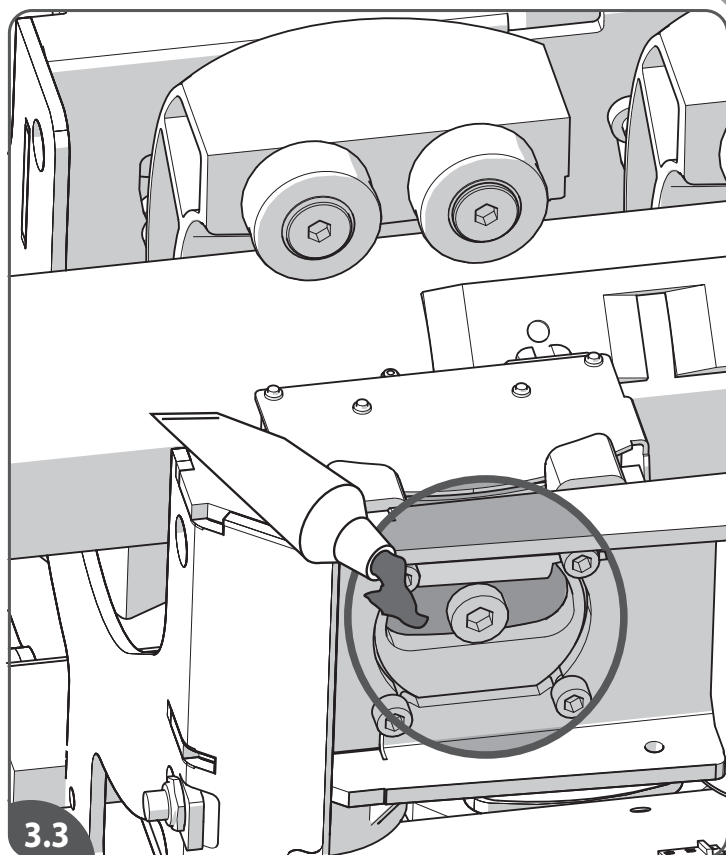
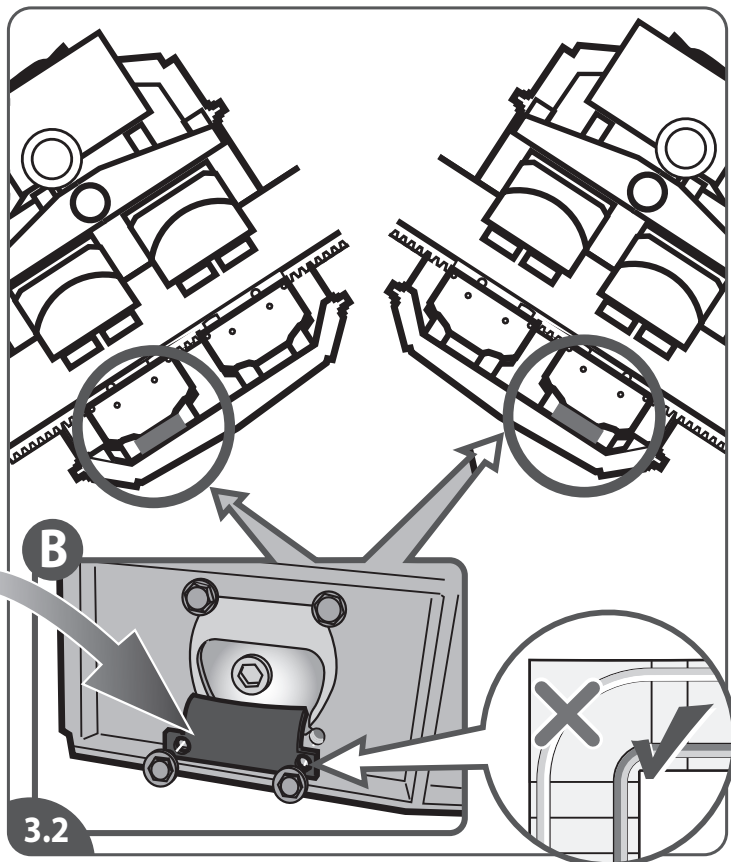
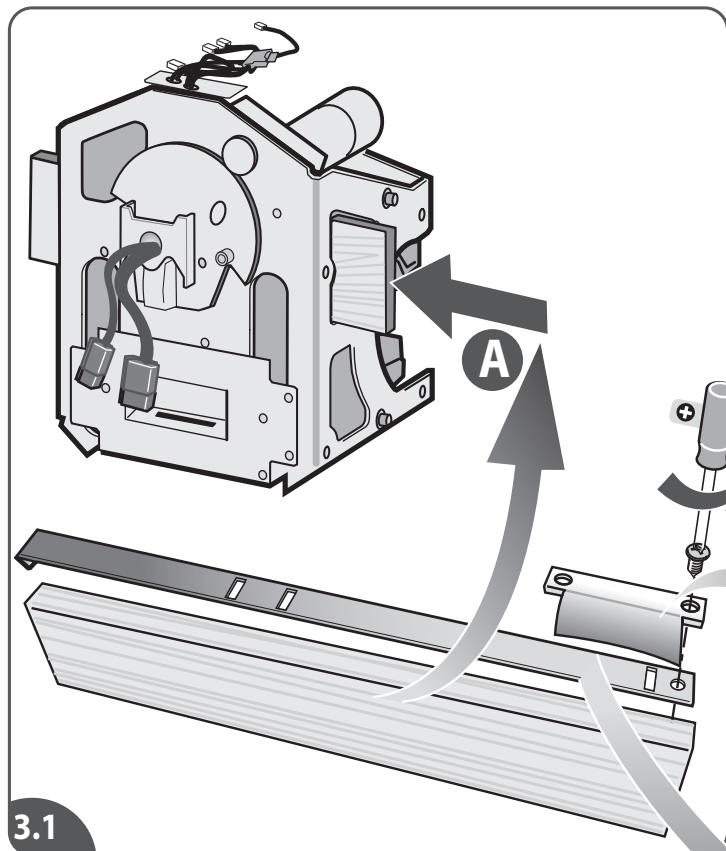
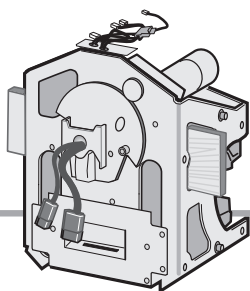
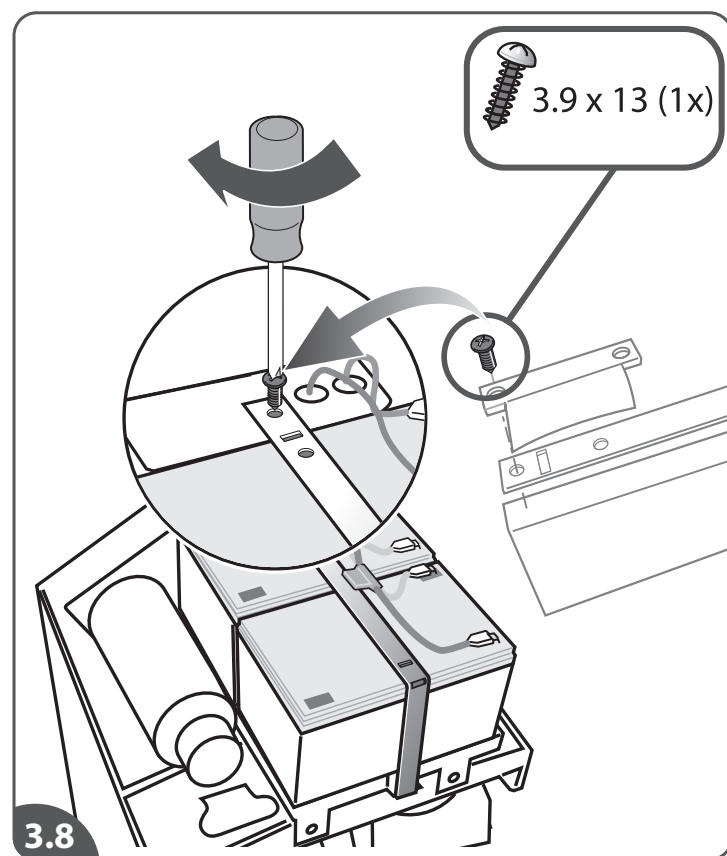
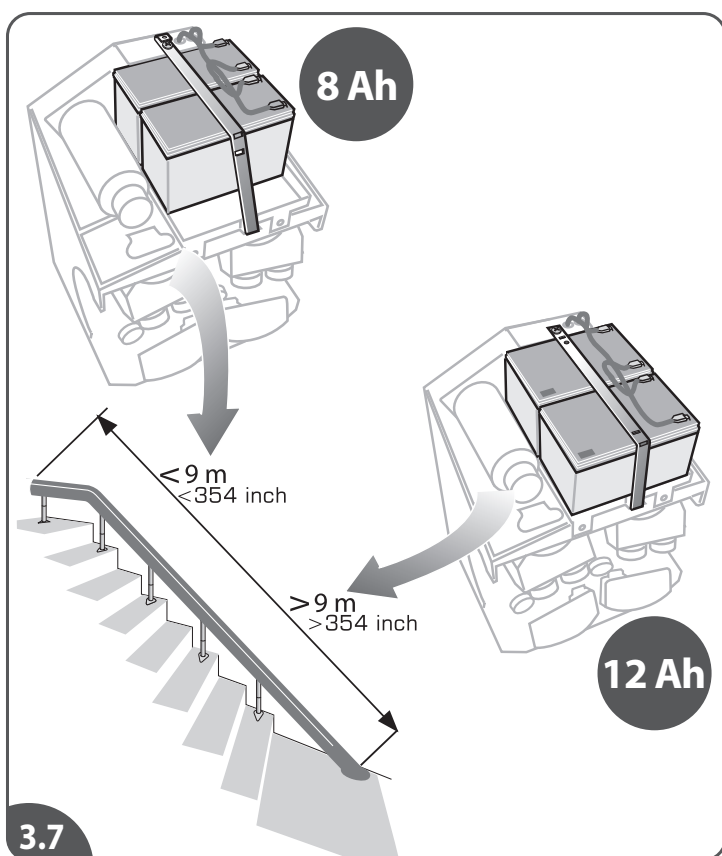
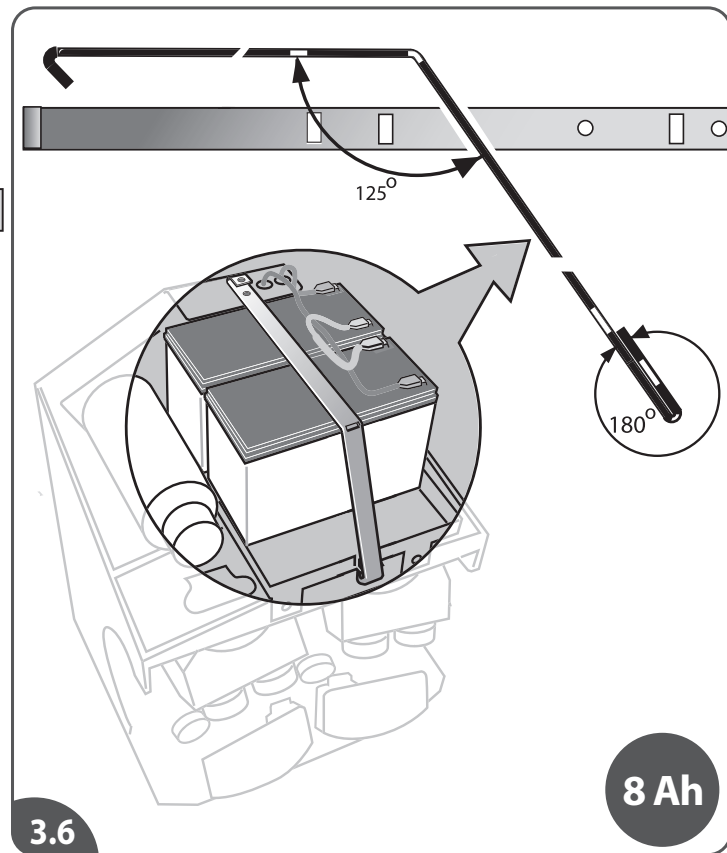
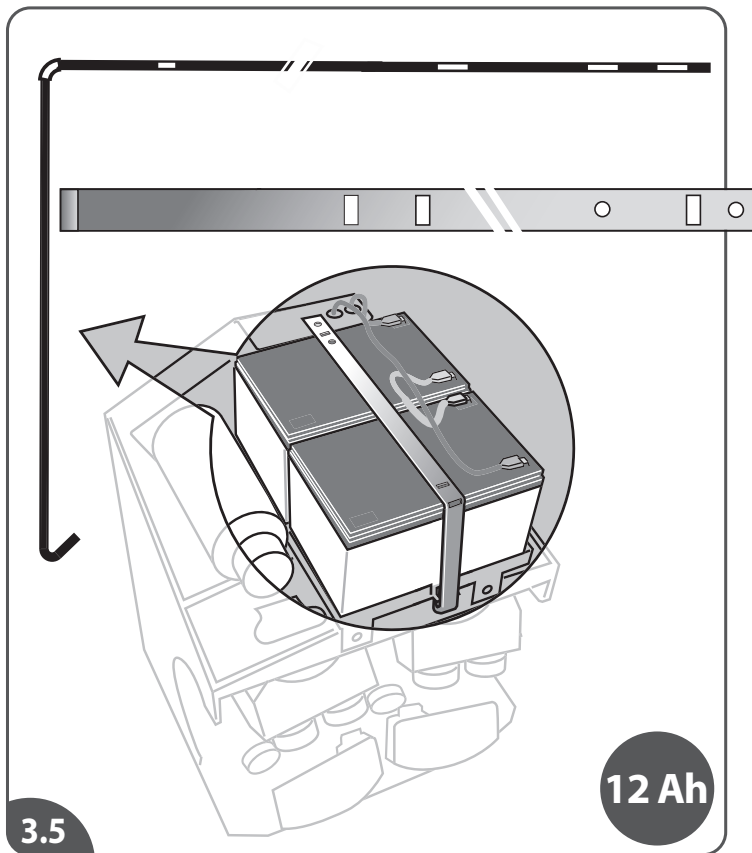
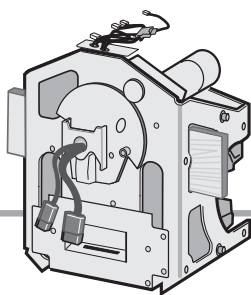
2

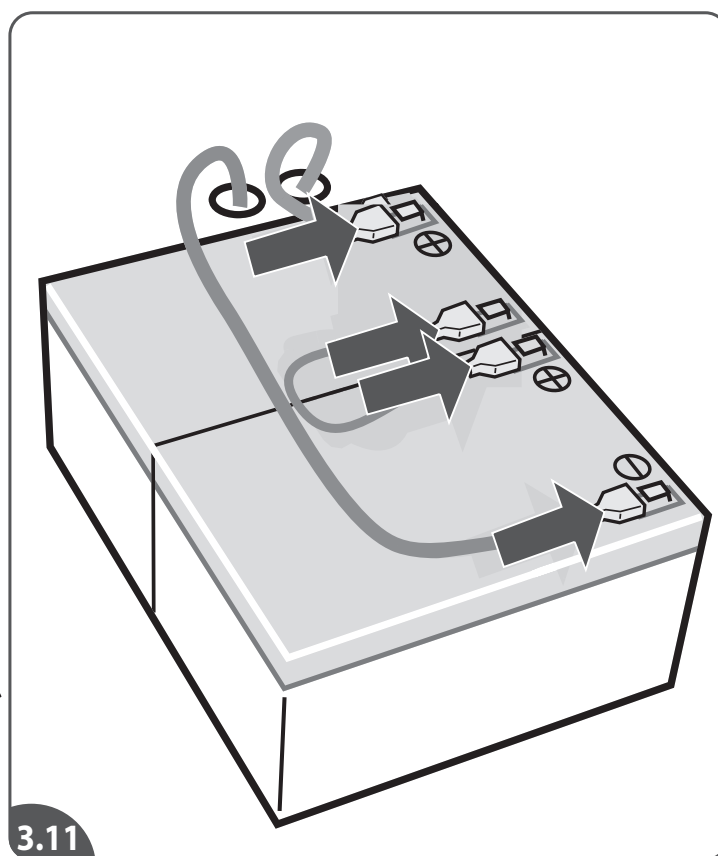
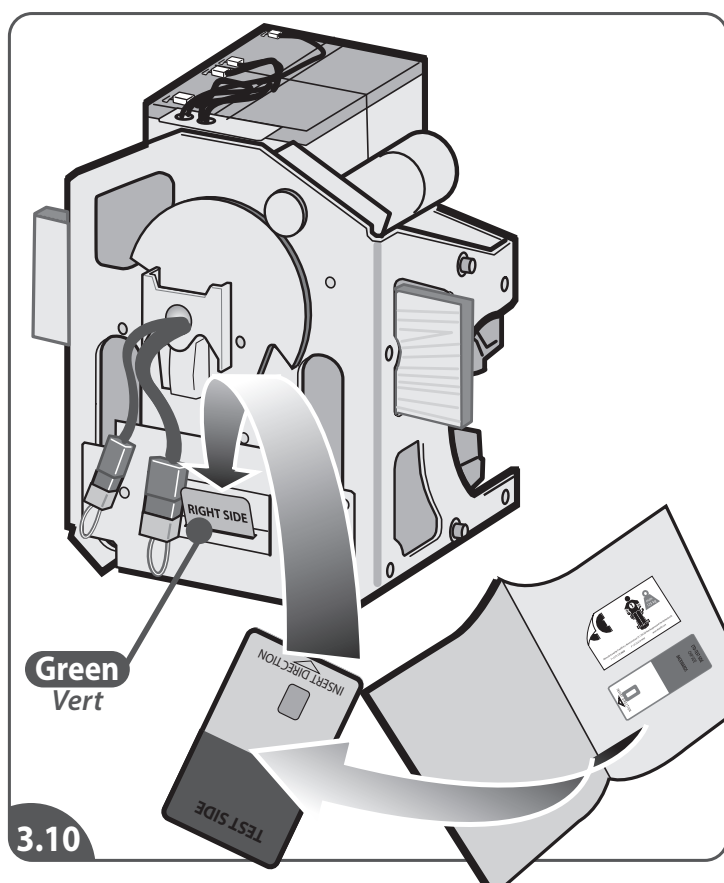
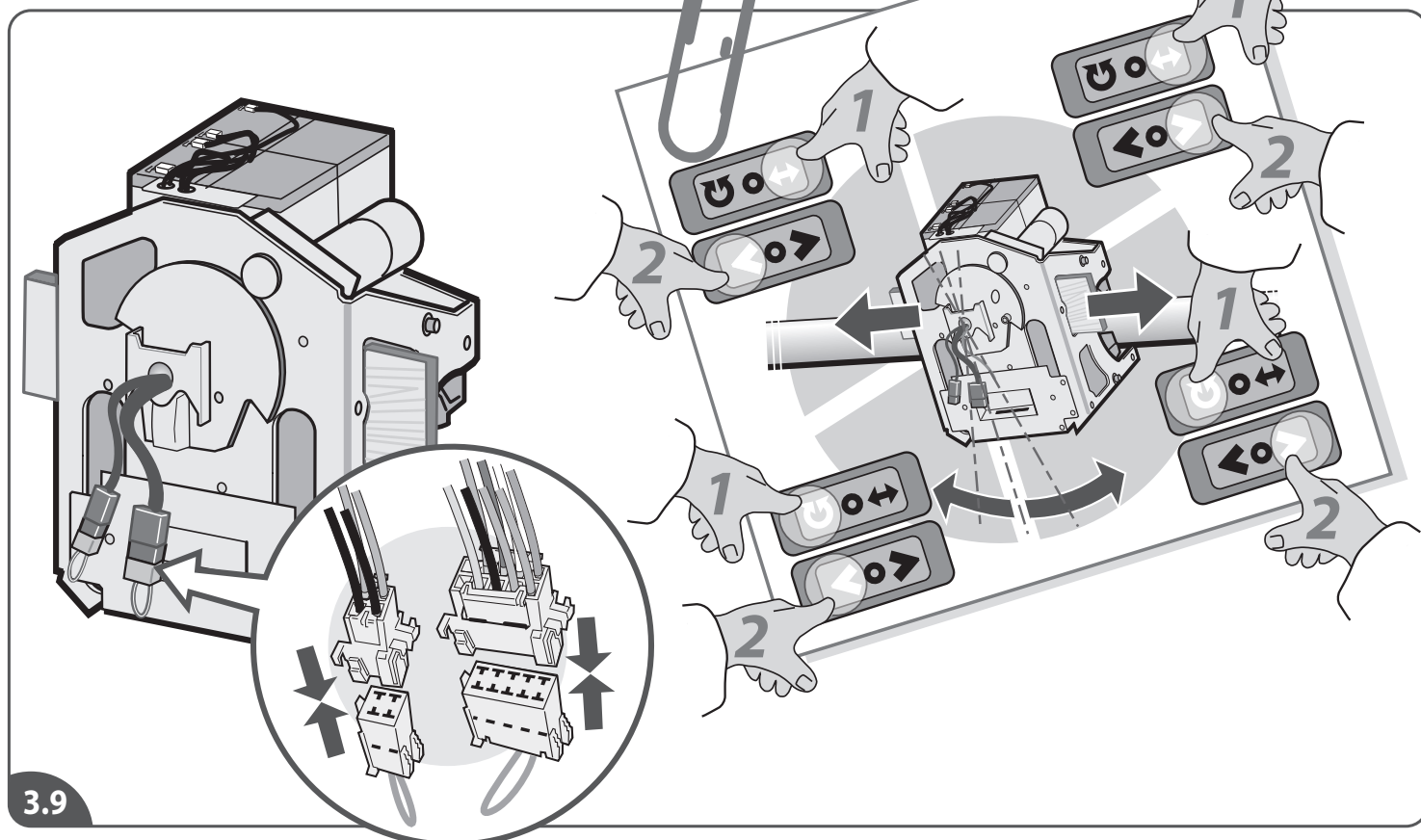
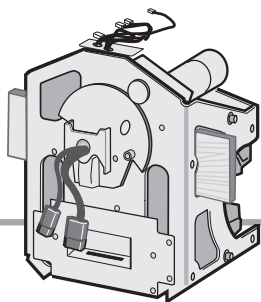
3

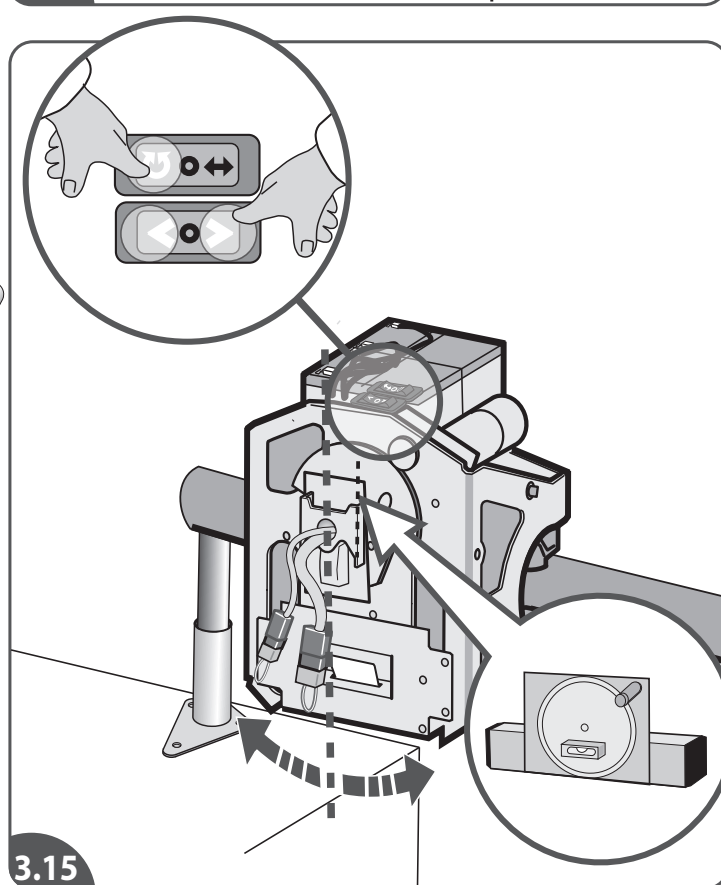
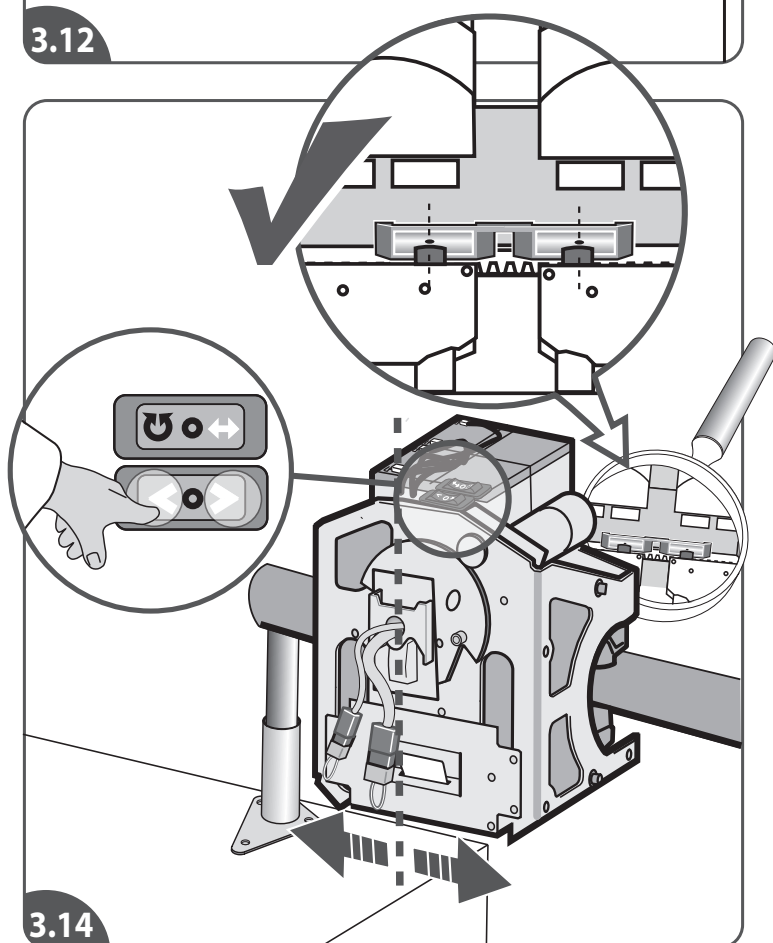
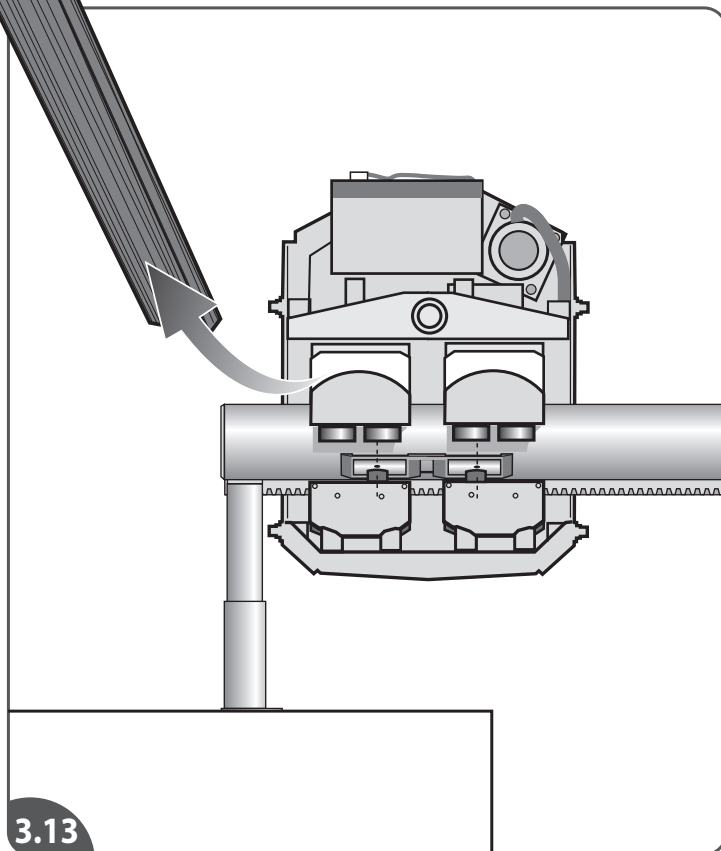
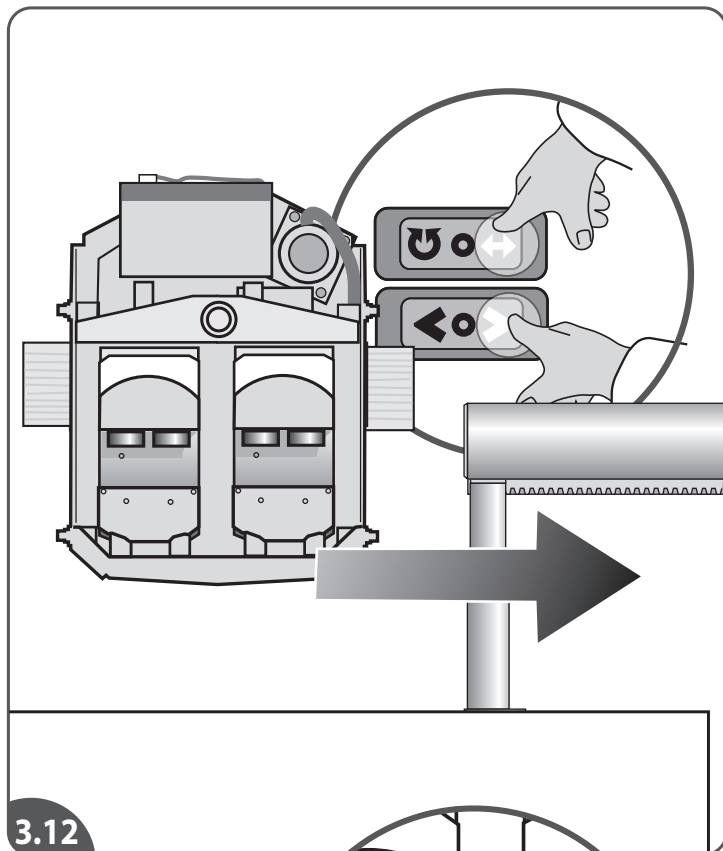
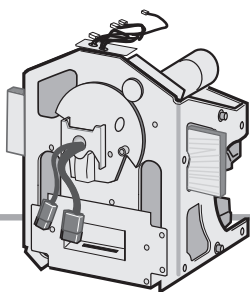
3

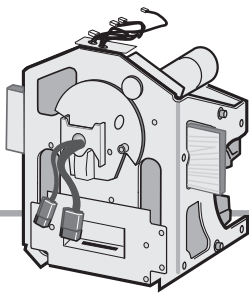
3

3

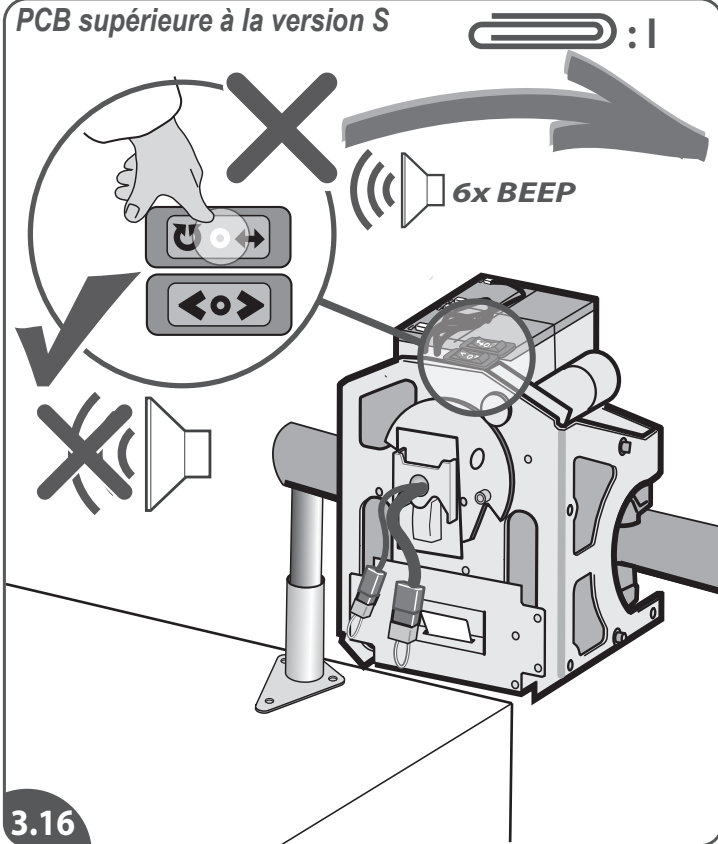
3

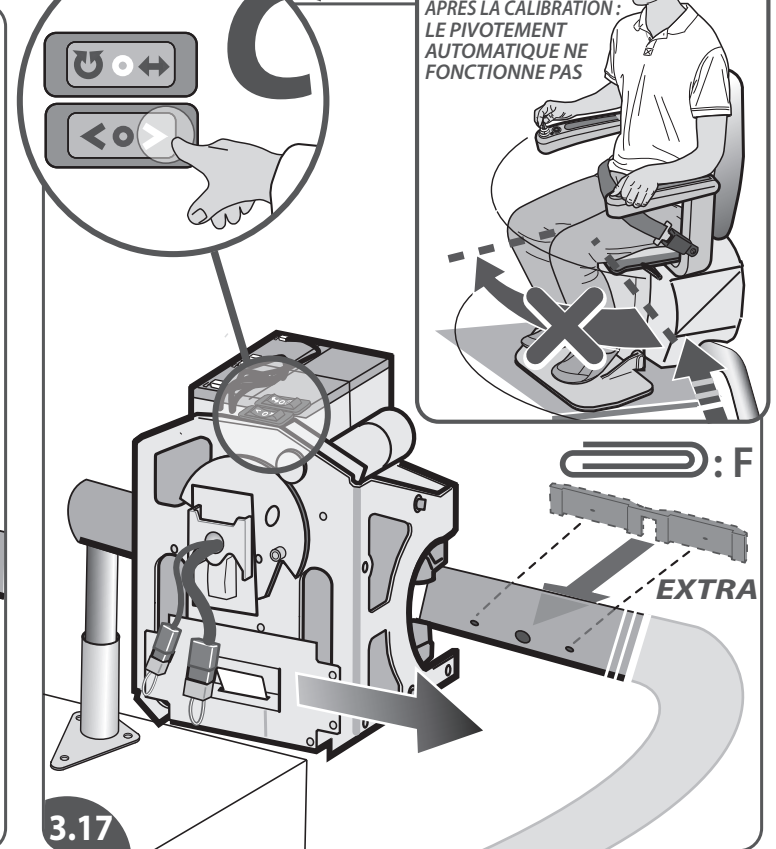
PCB higher than version S

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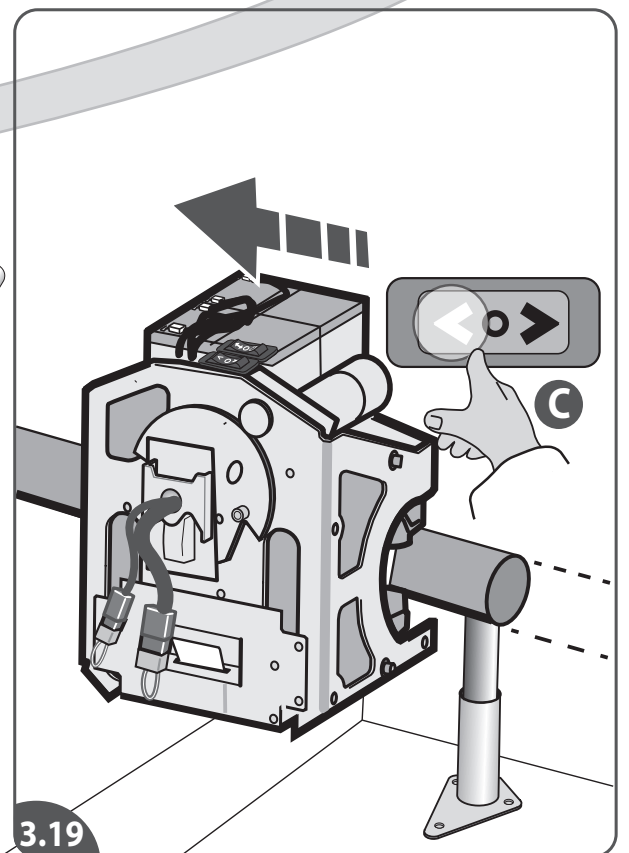
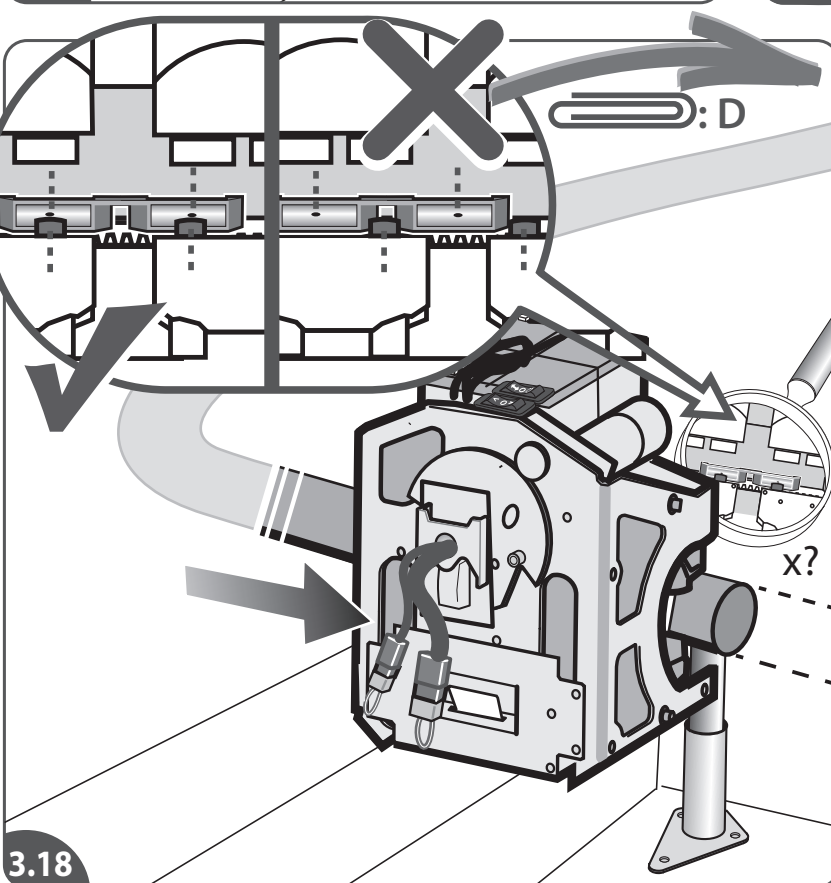
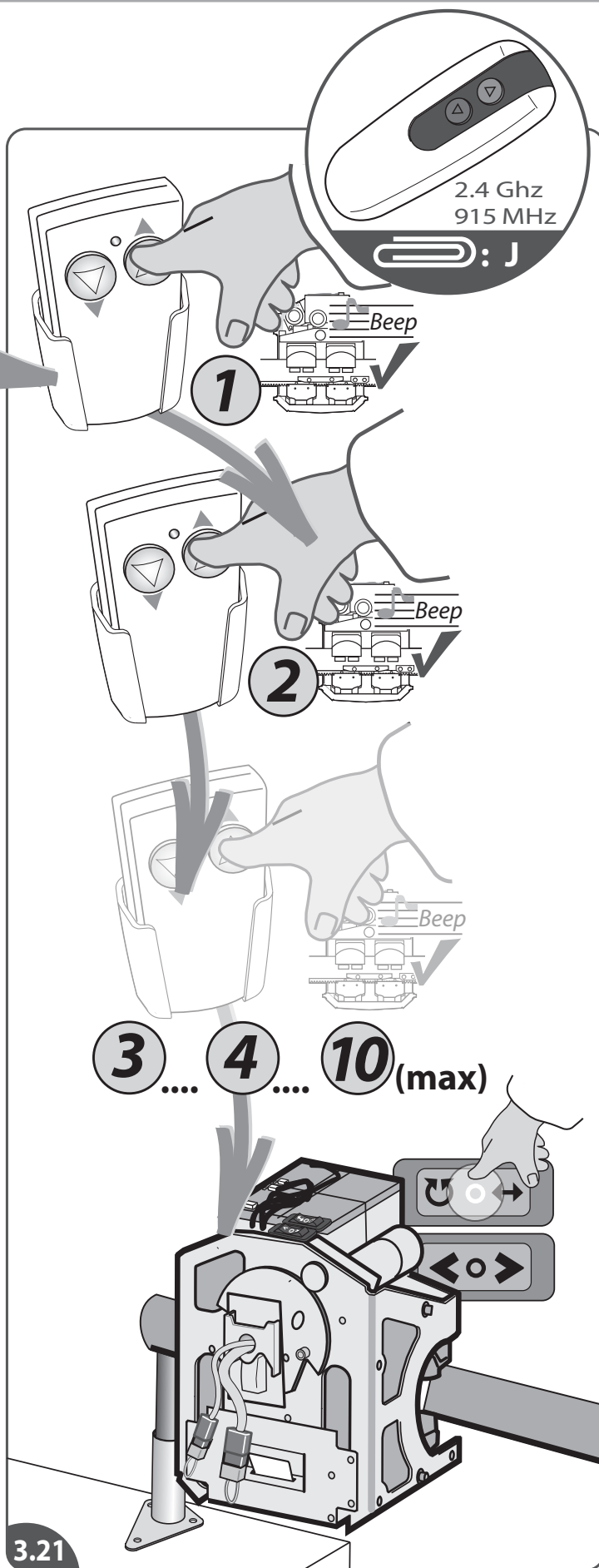
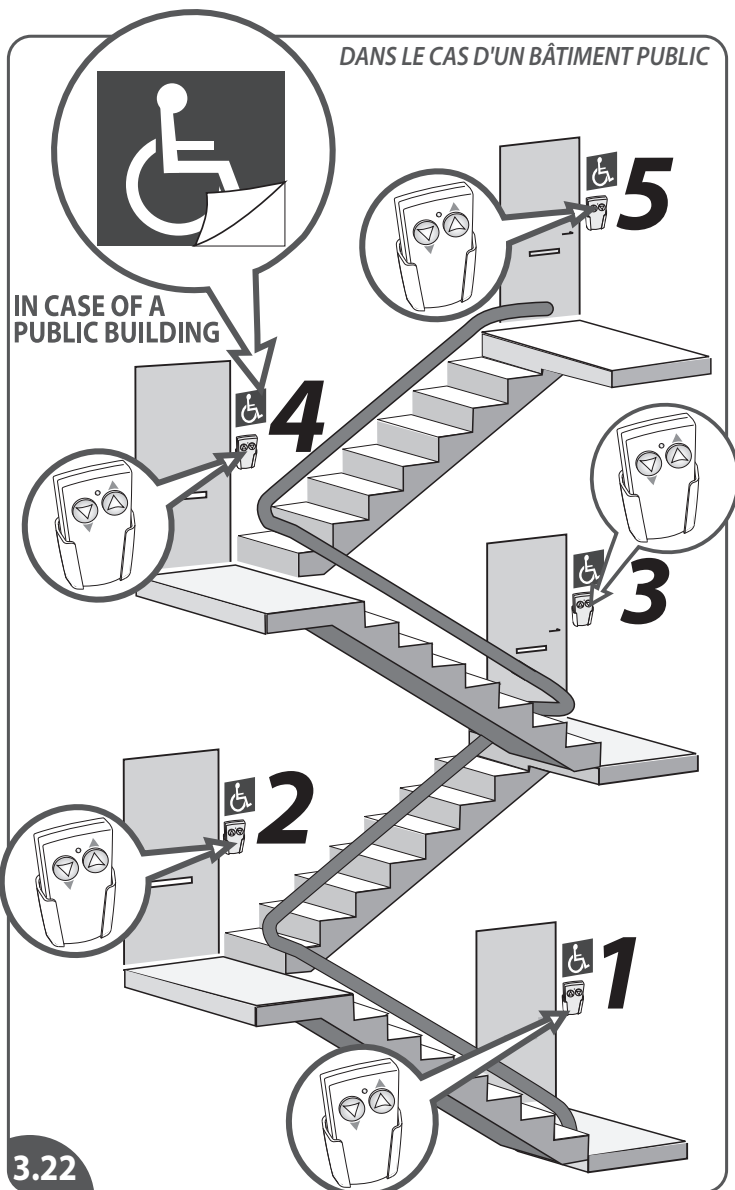
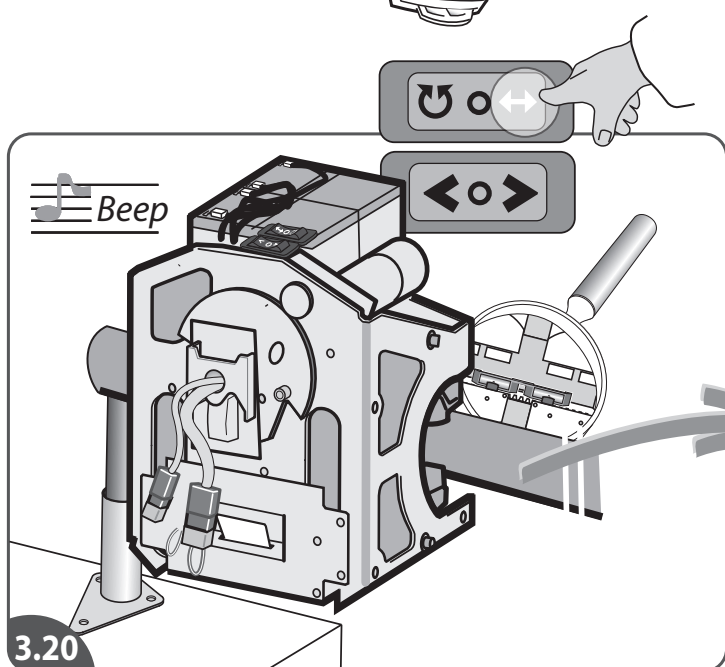
FIRST RIDE AFTER
CALIBRATION: THE
AUTOSWIVEL WILL
NOT FUNCTION

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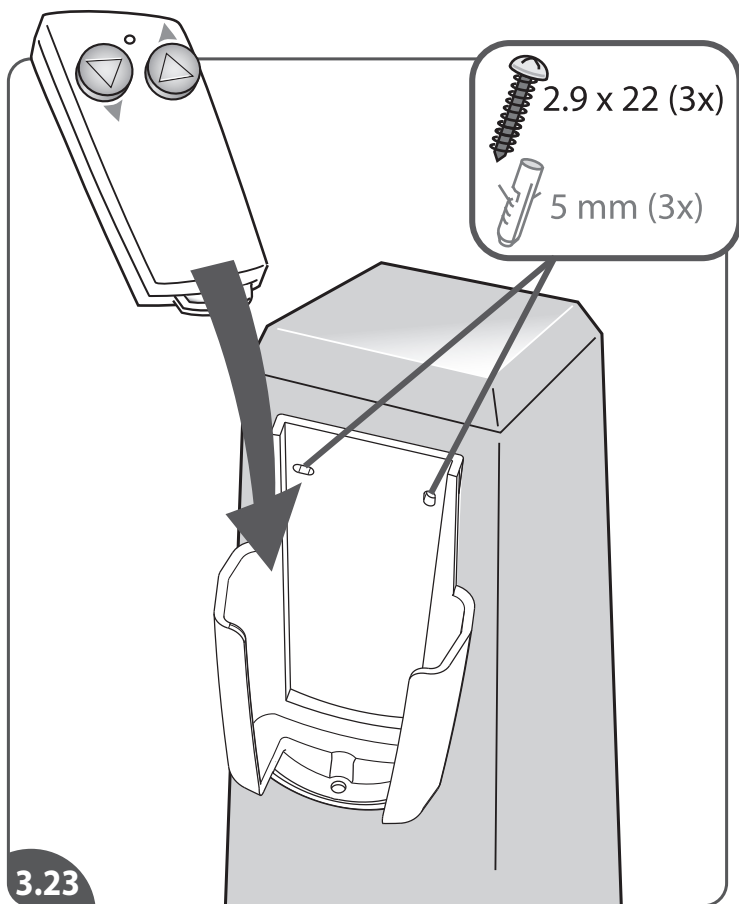
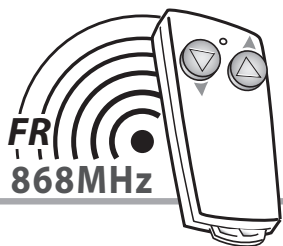
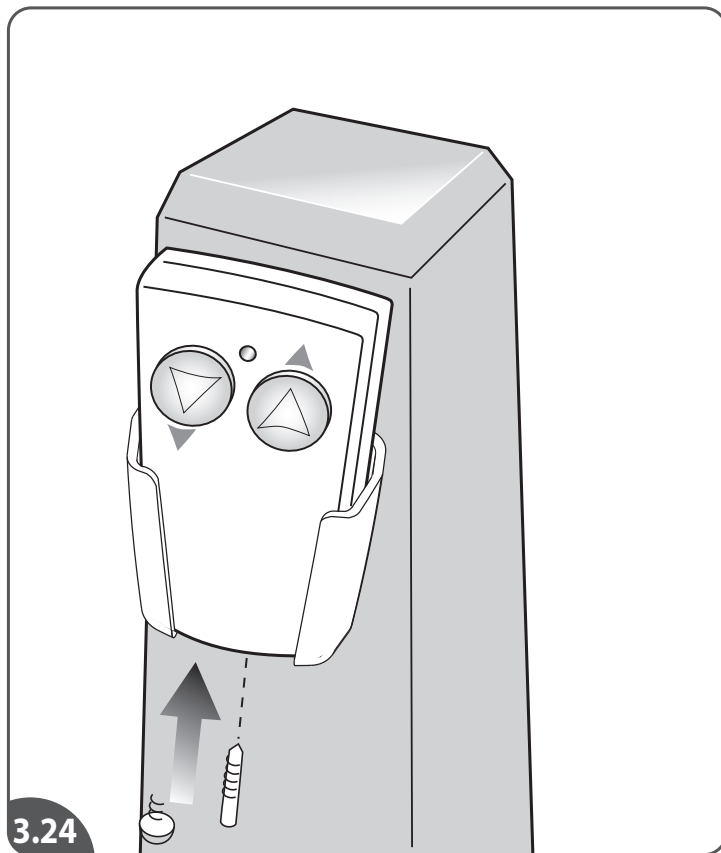
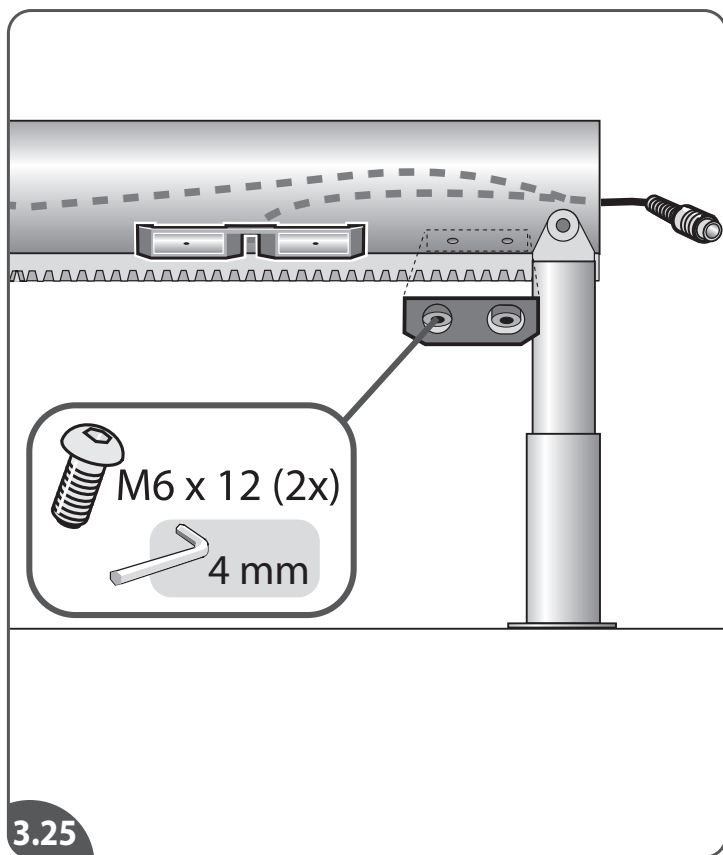
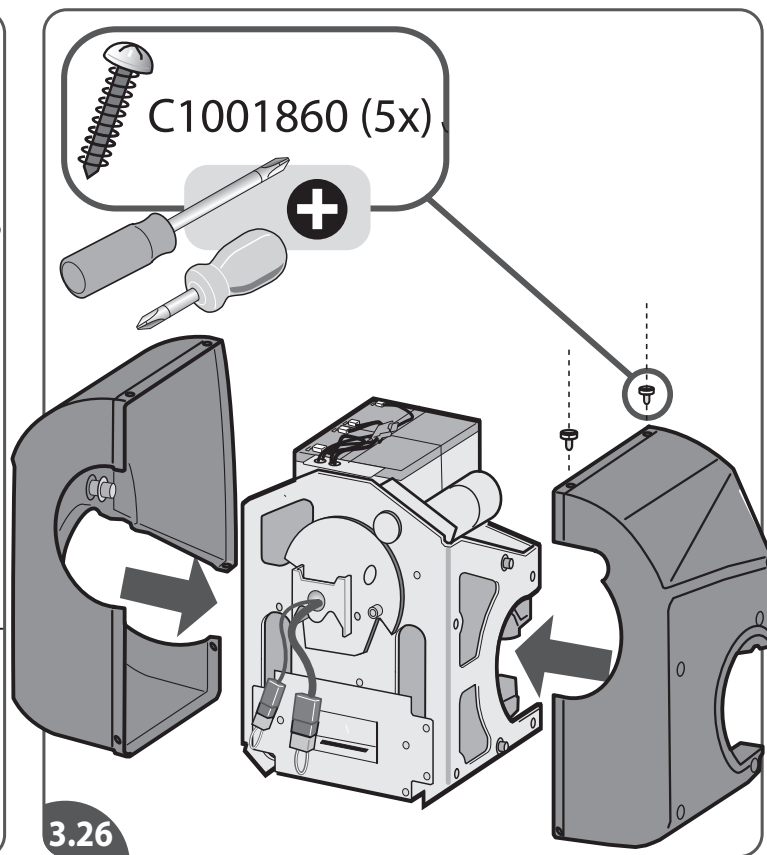
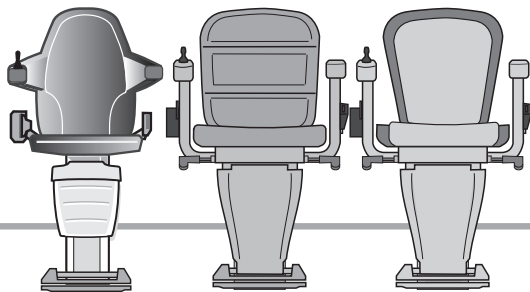
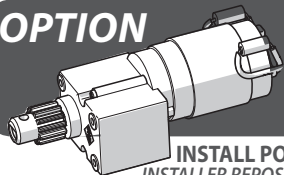
4

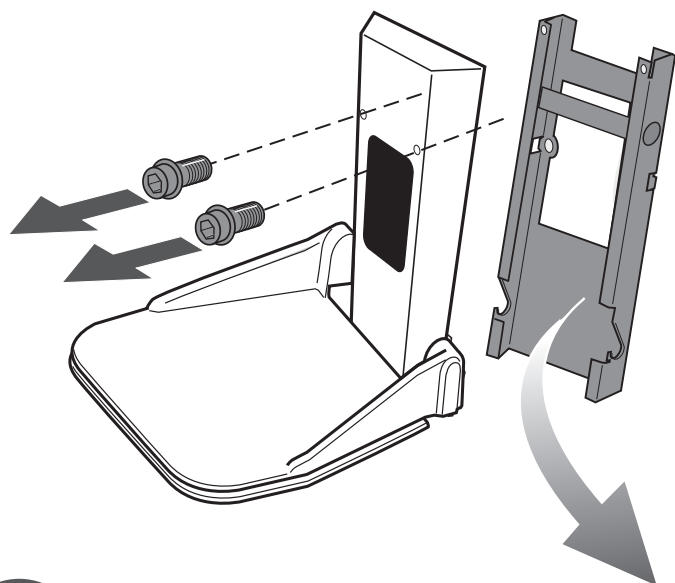
OPTION

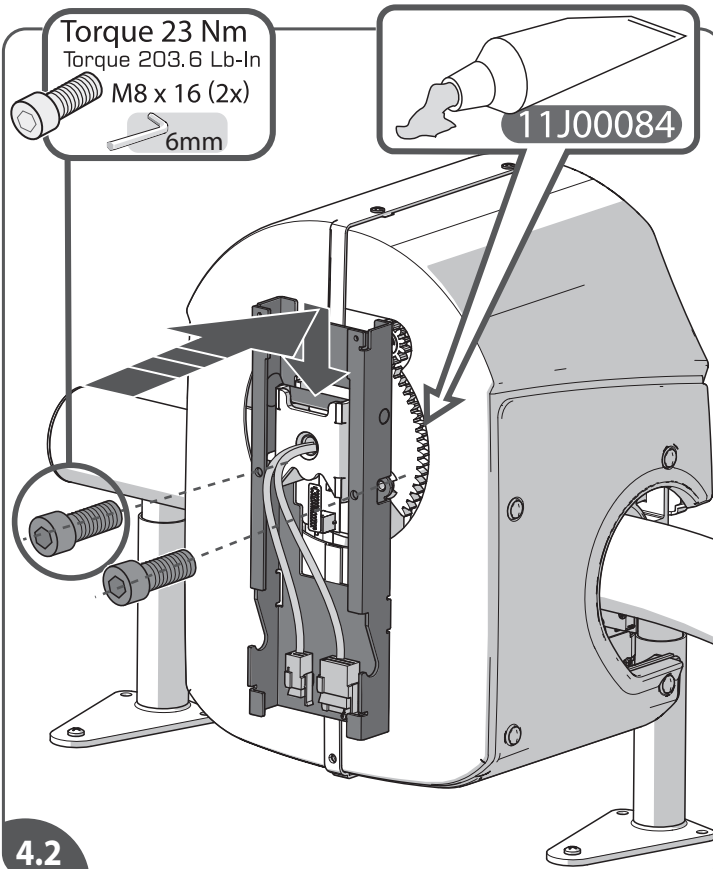
INSTALL POWERED FOOTREST
INSTALLER REPOSE-PIEDS AUTOMATIQUE

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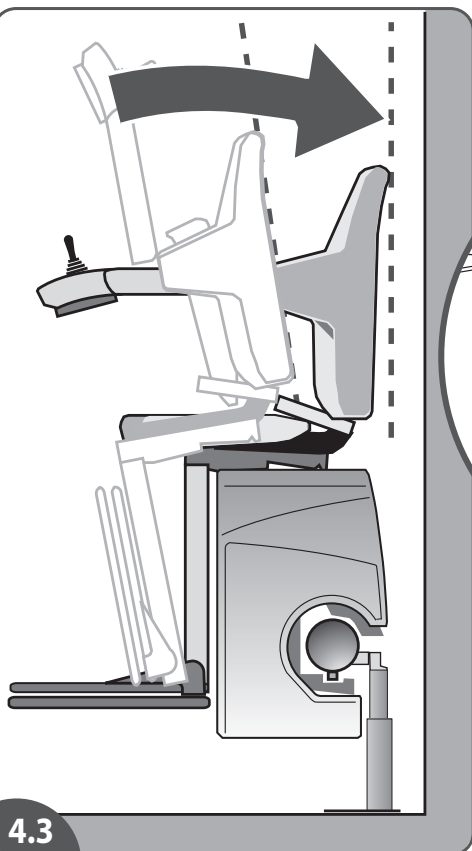
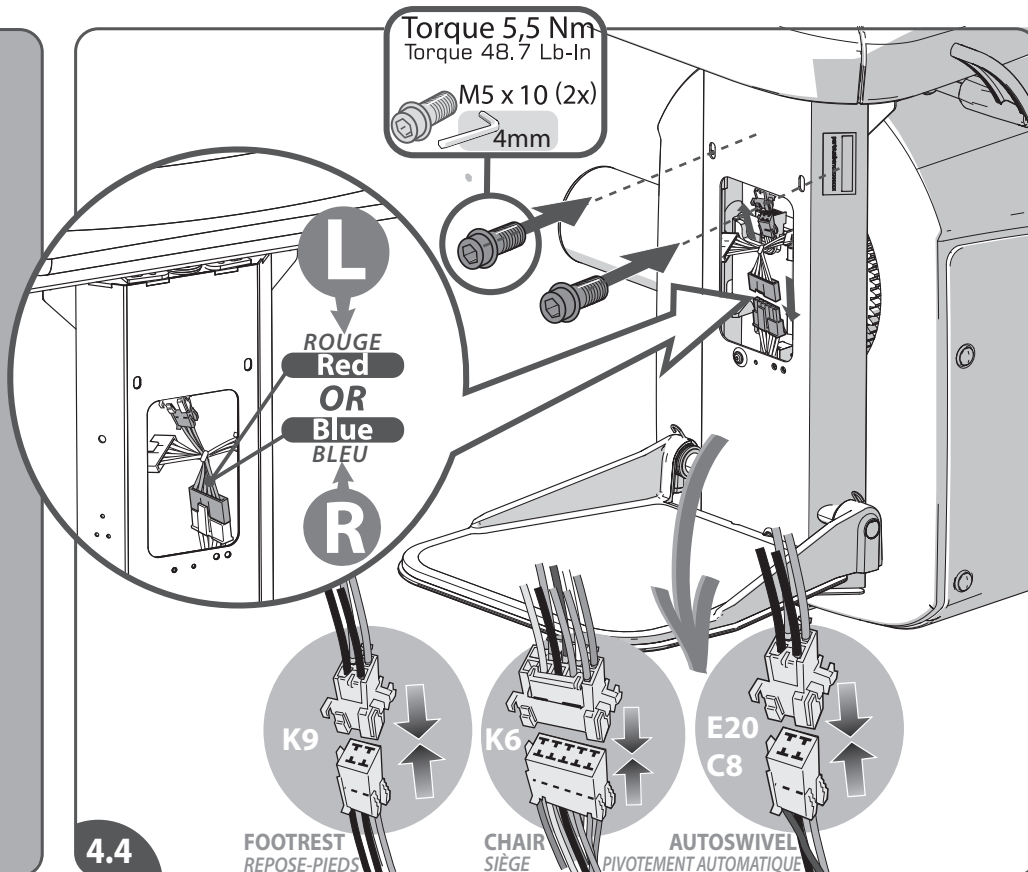
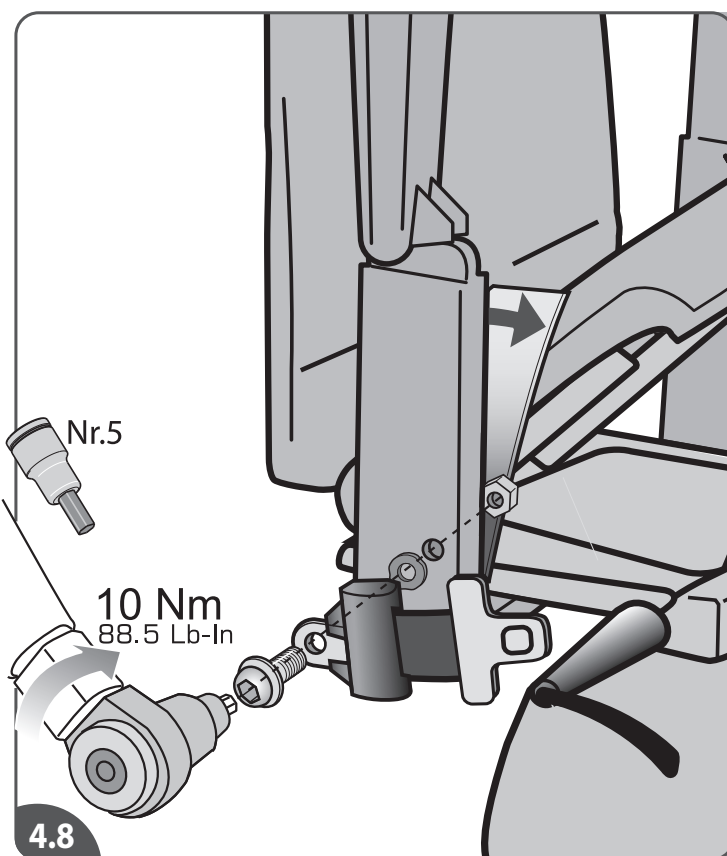
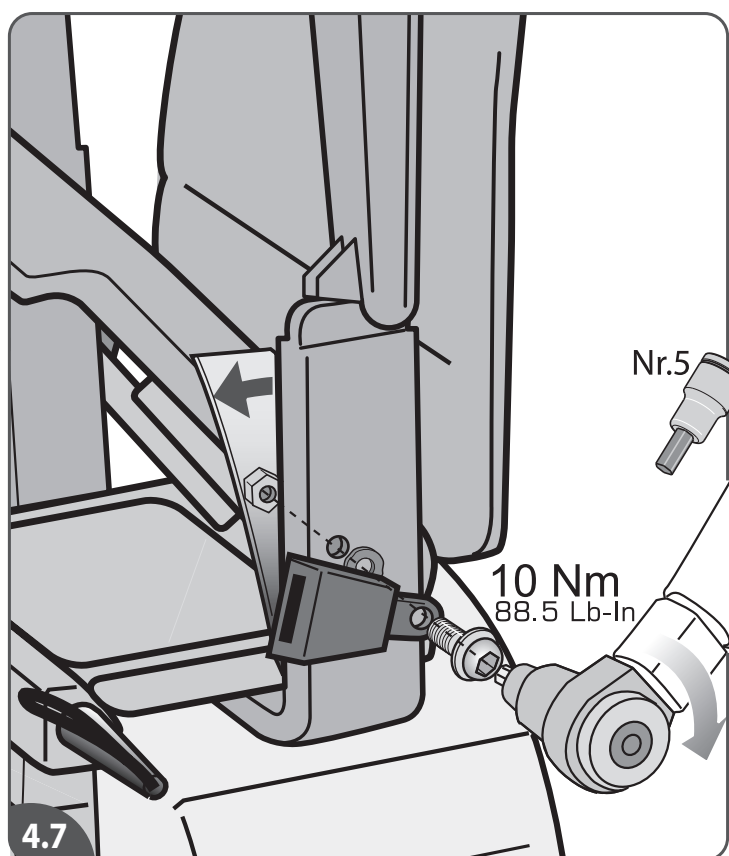
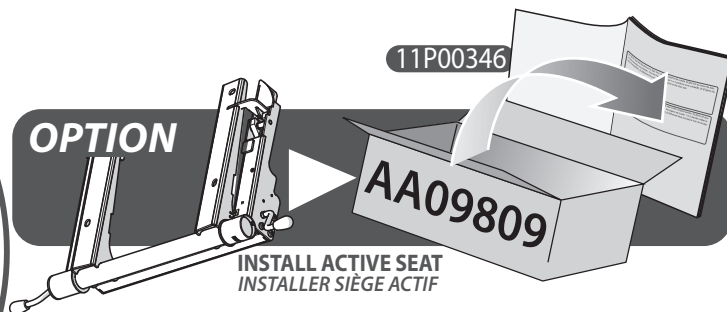
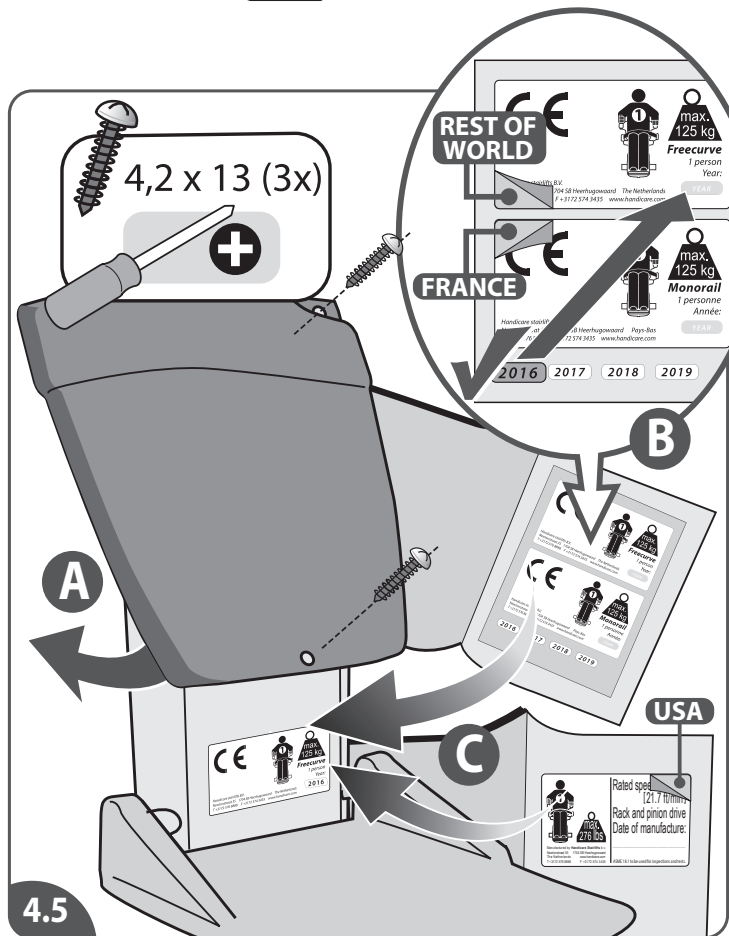
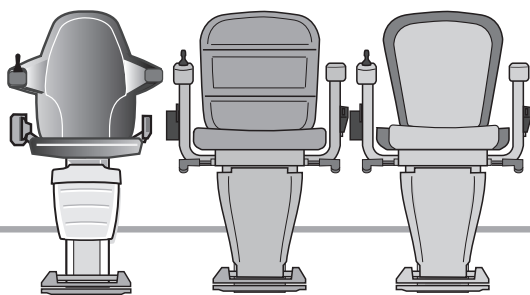
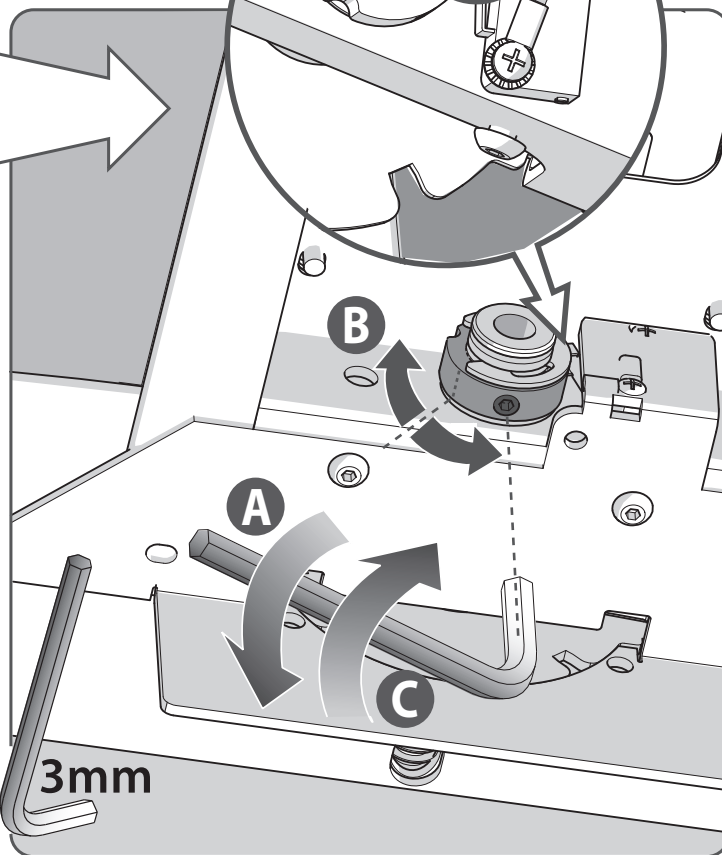
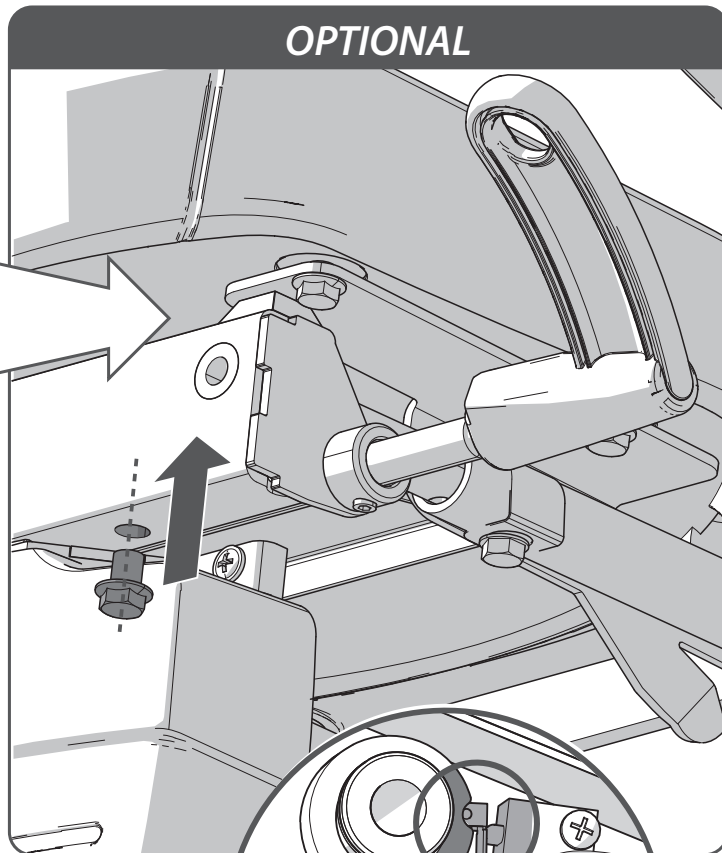
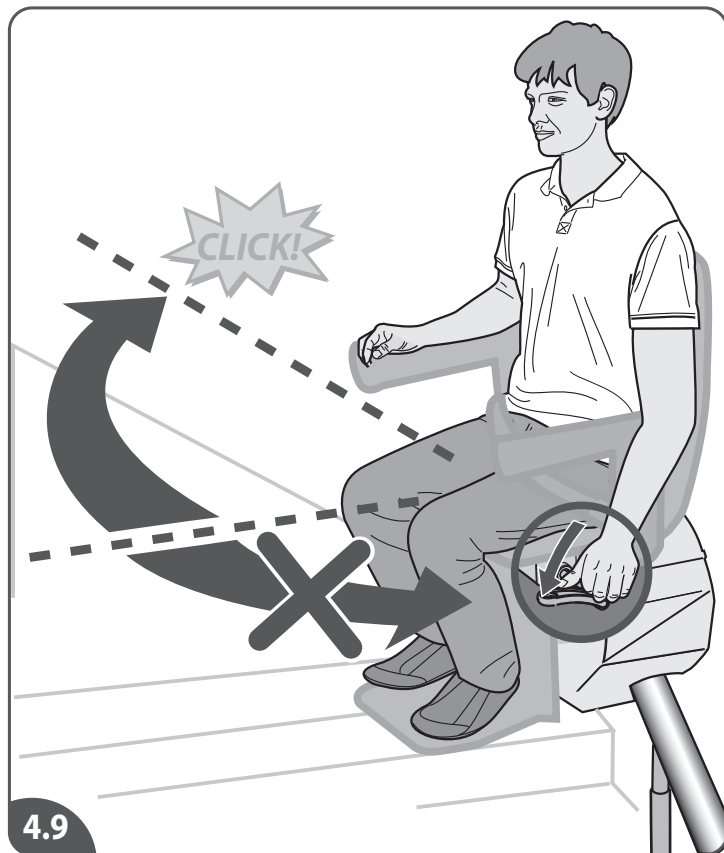
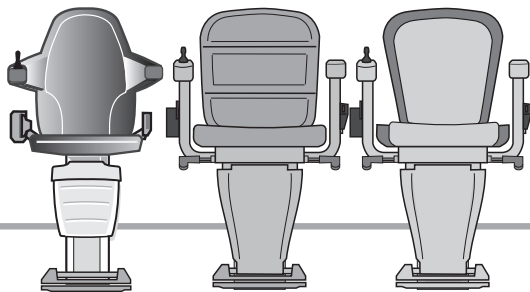
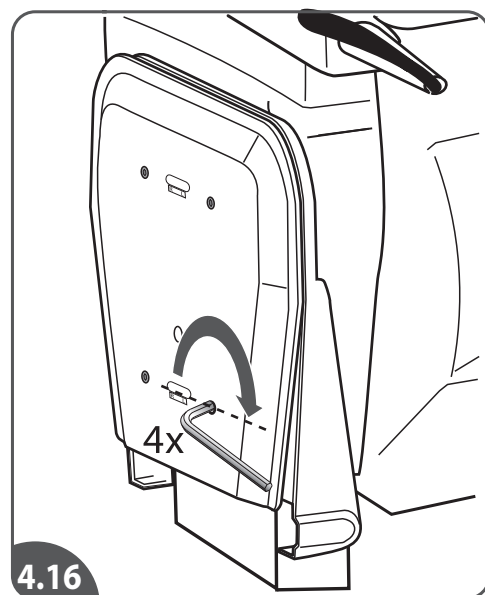
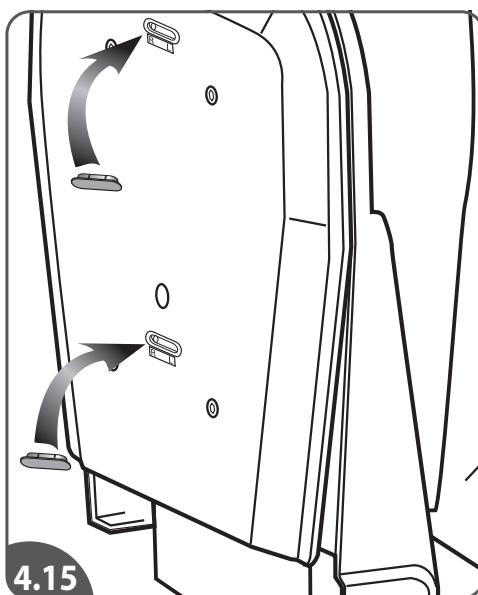
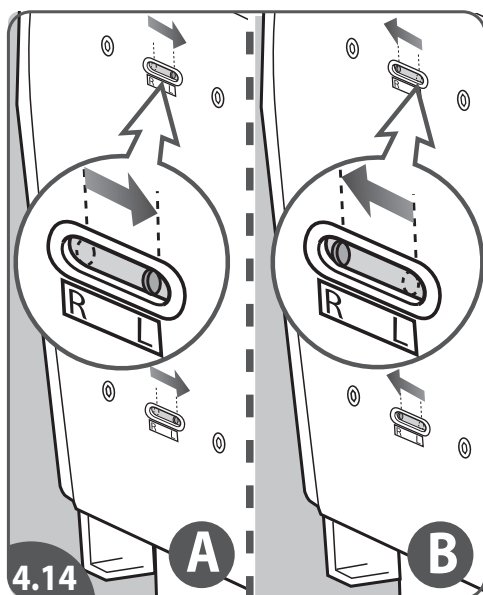
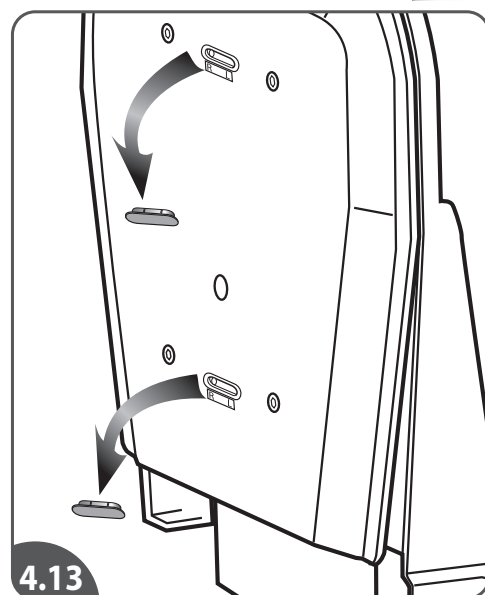
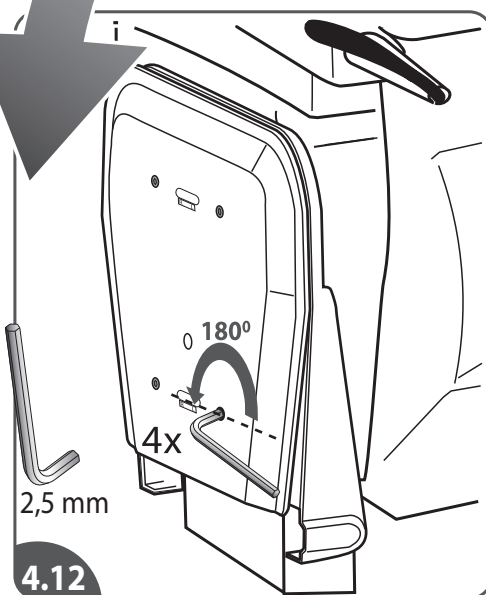
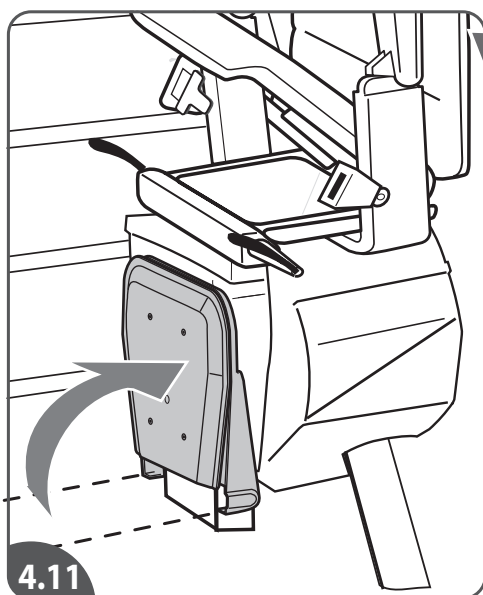
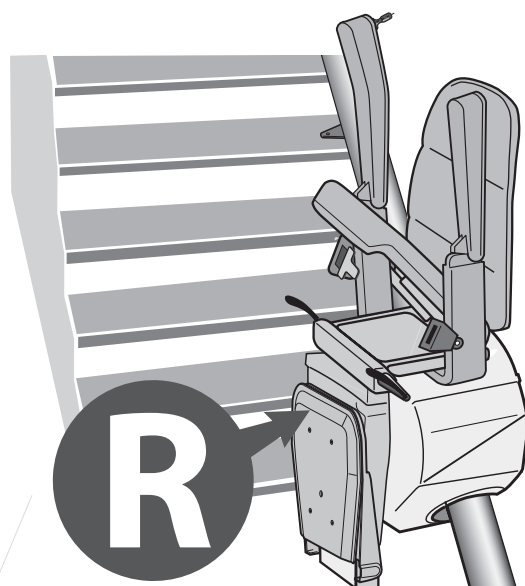
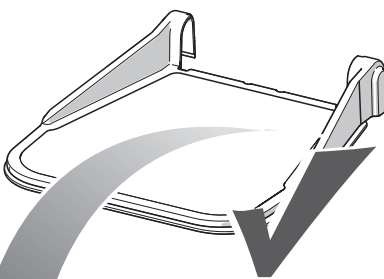
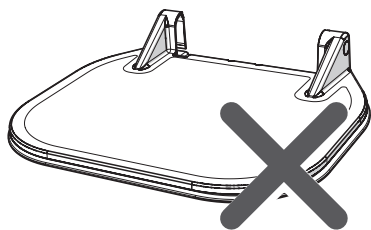
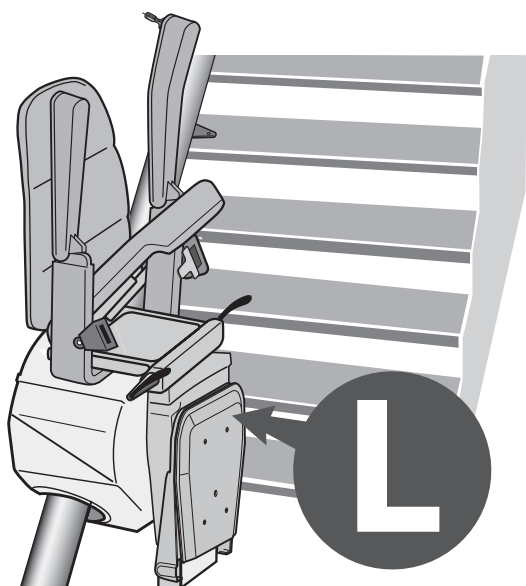
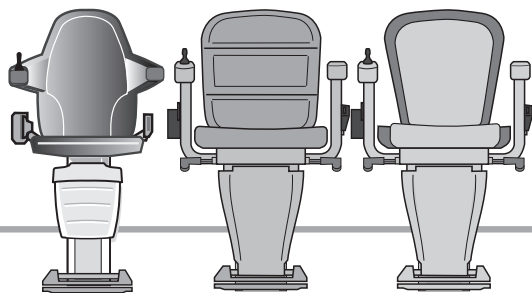
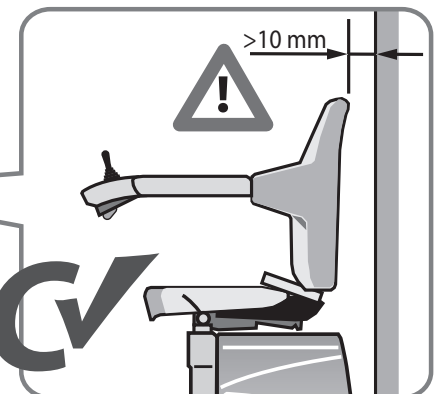
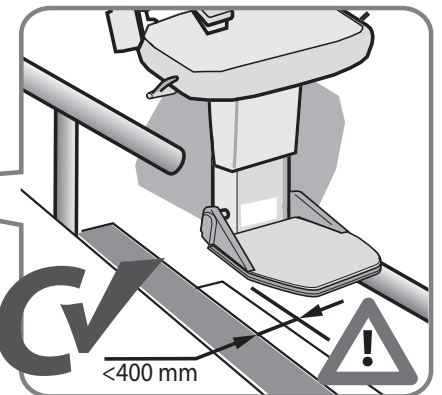
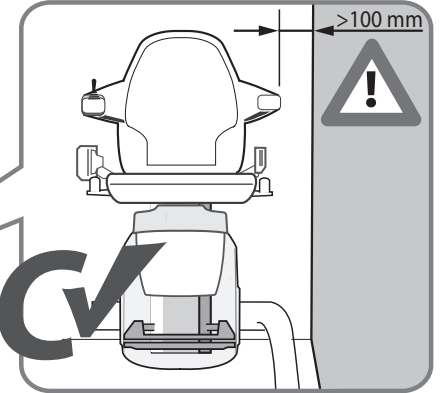
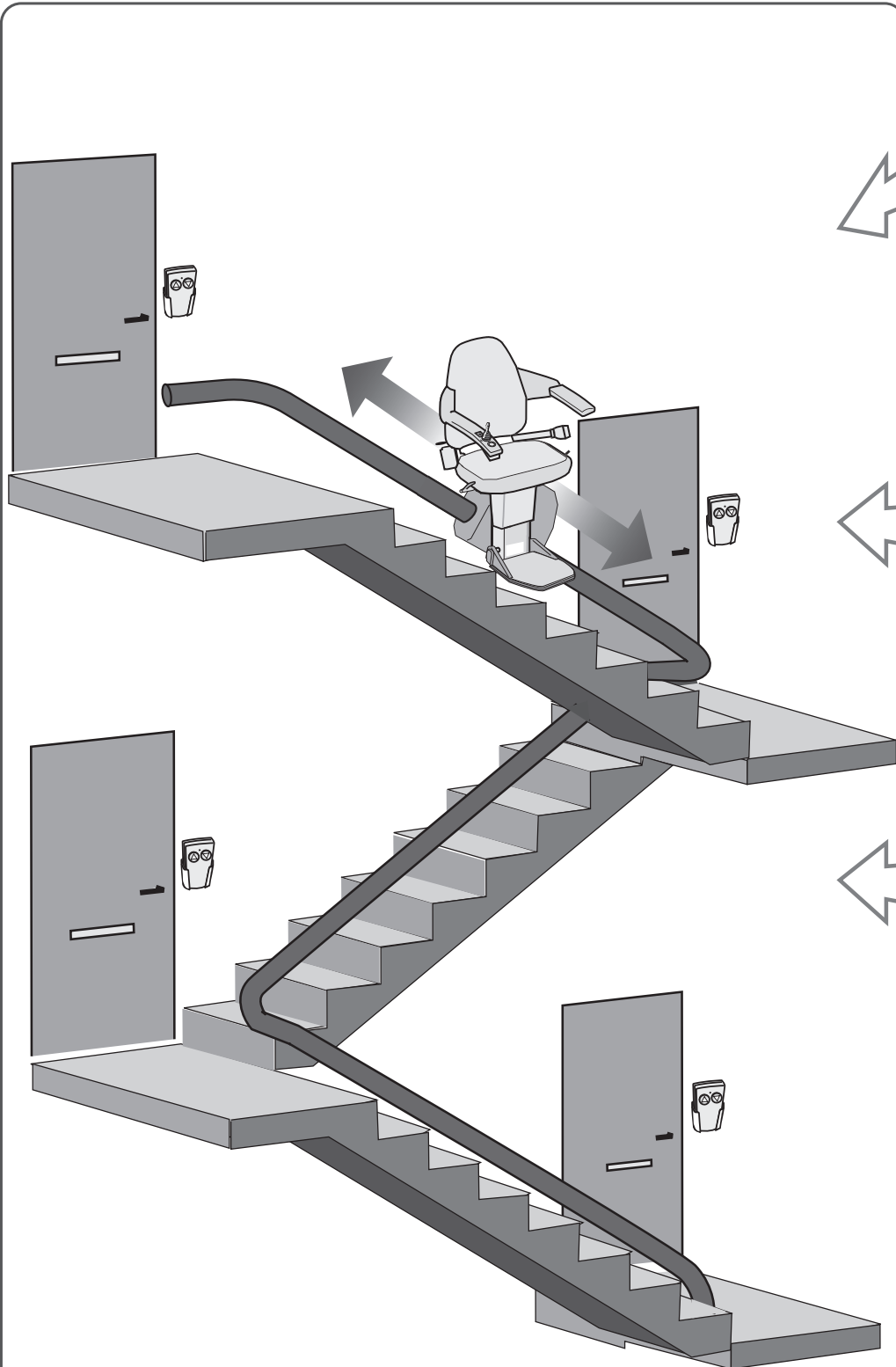
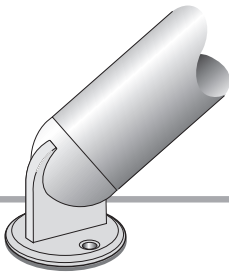
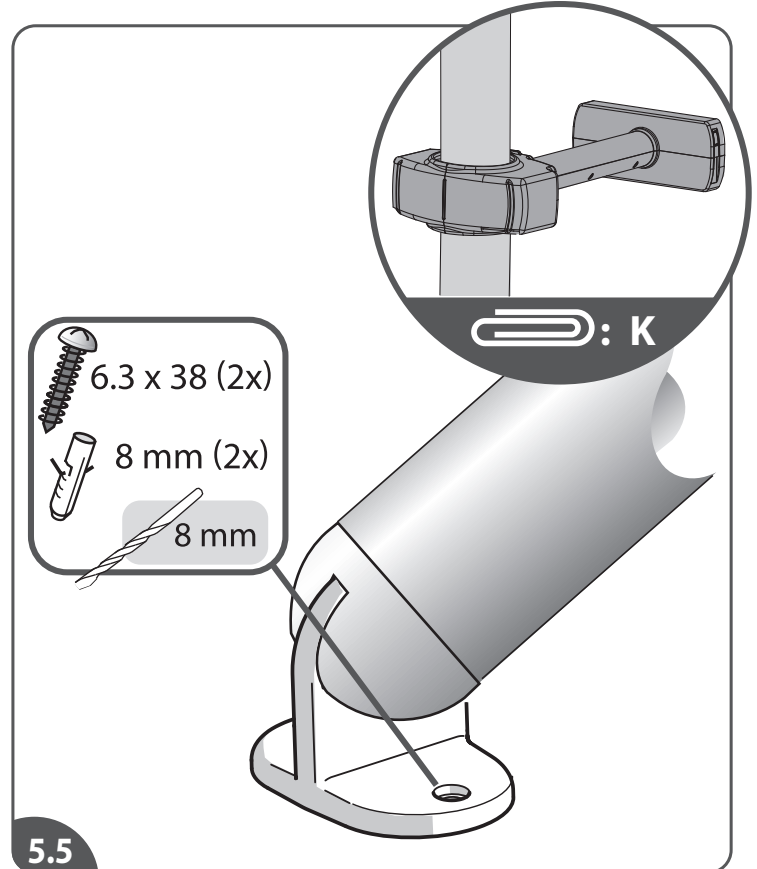
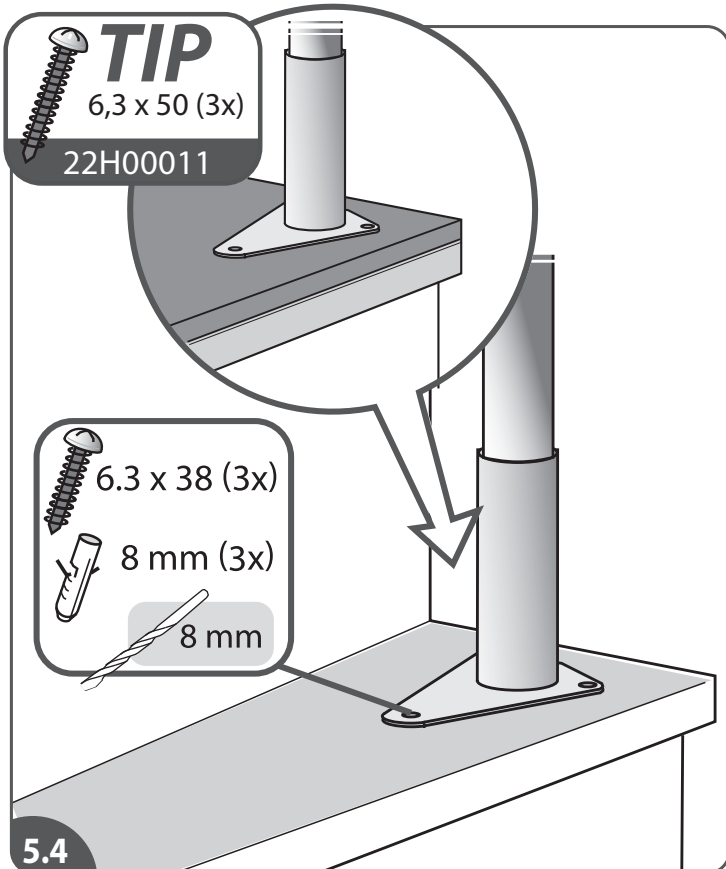
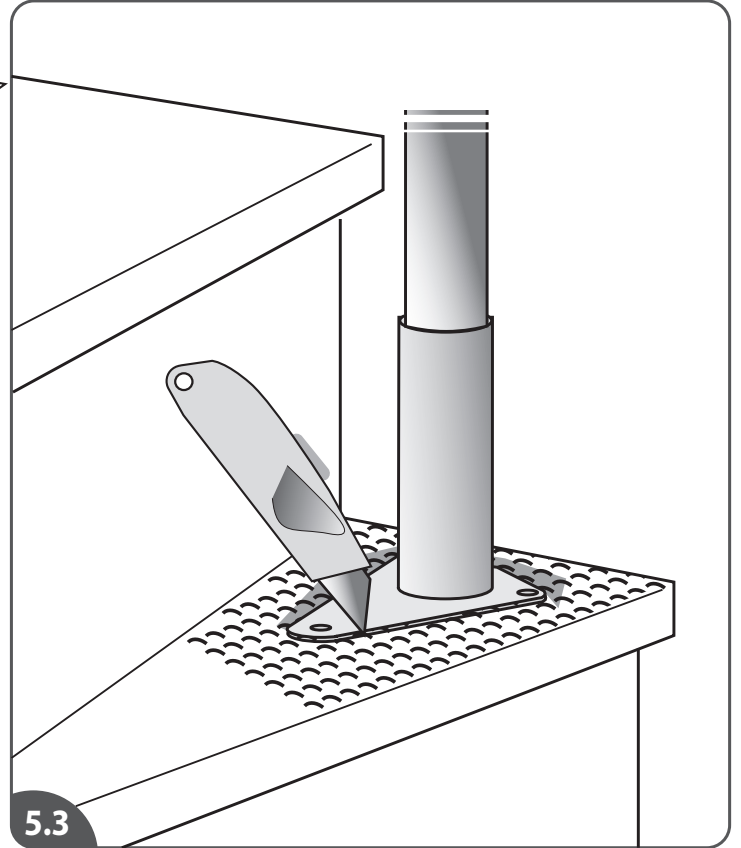
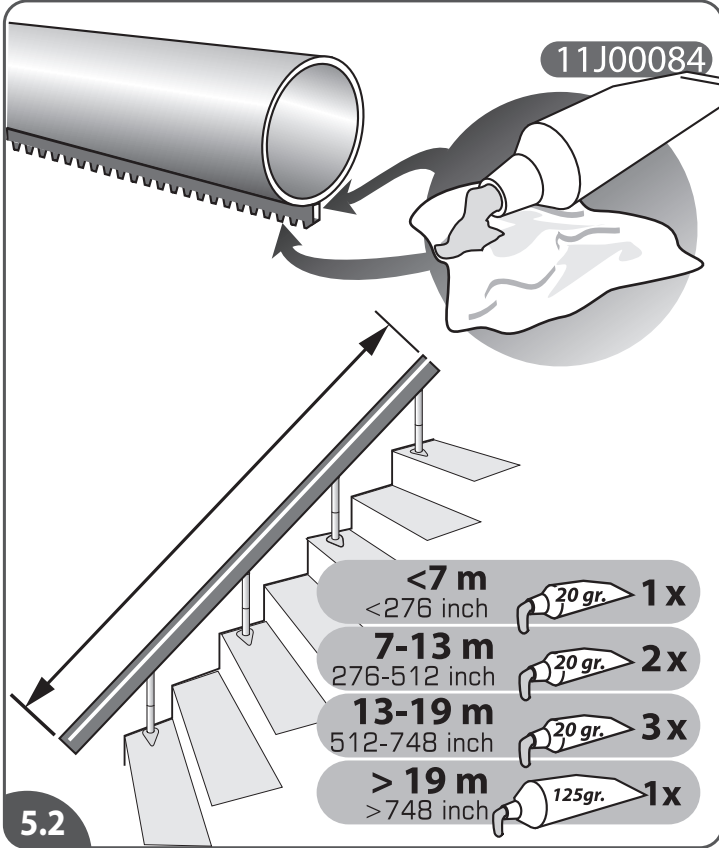
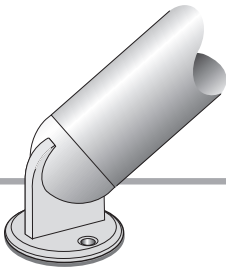
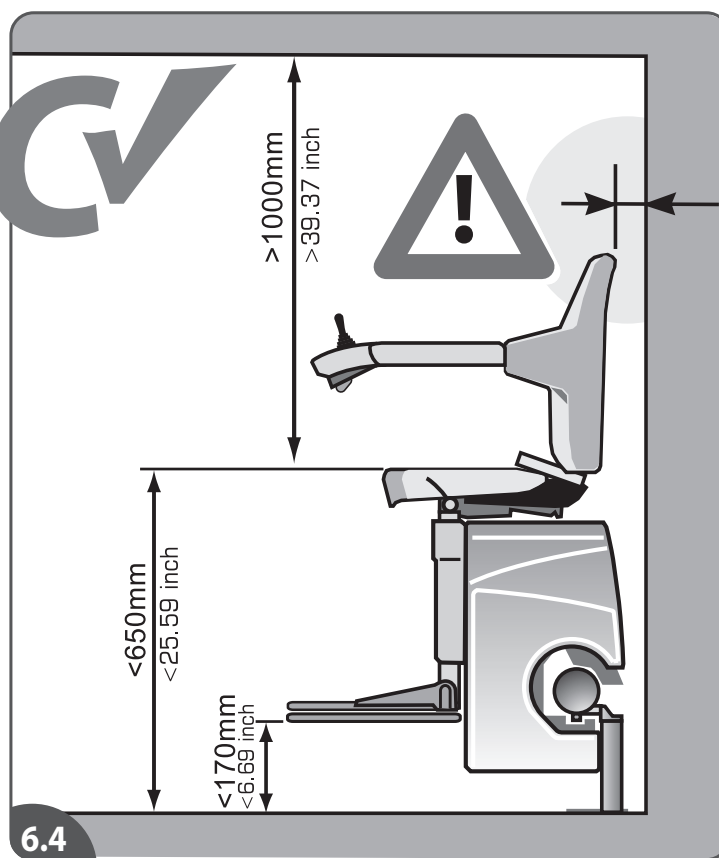
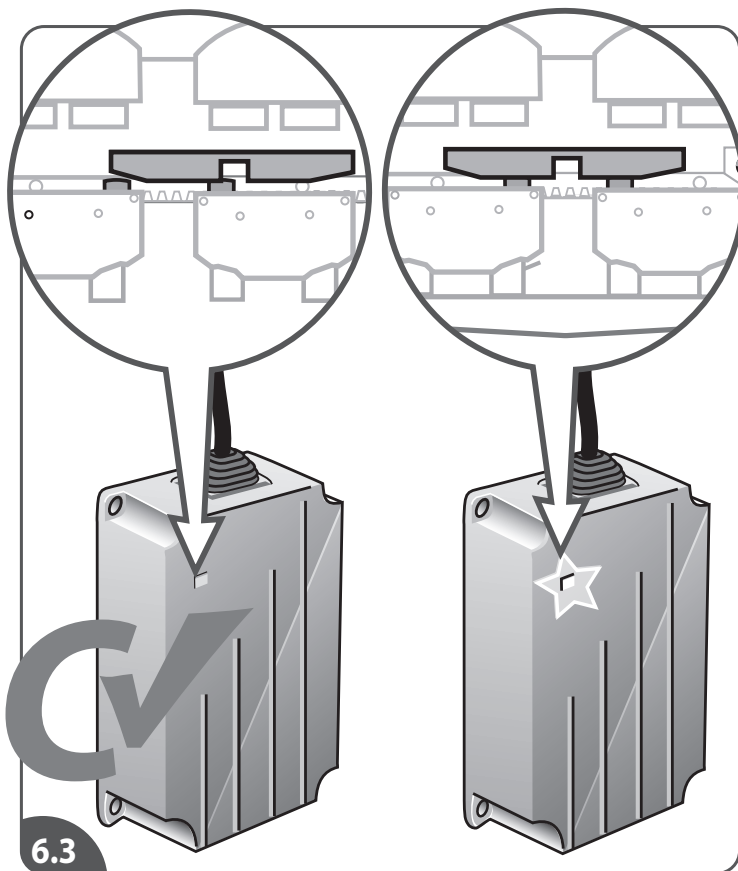
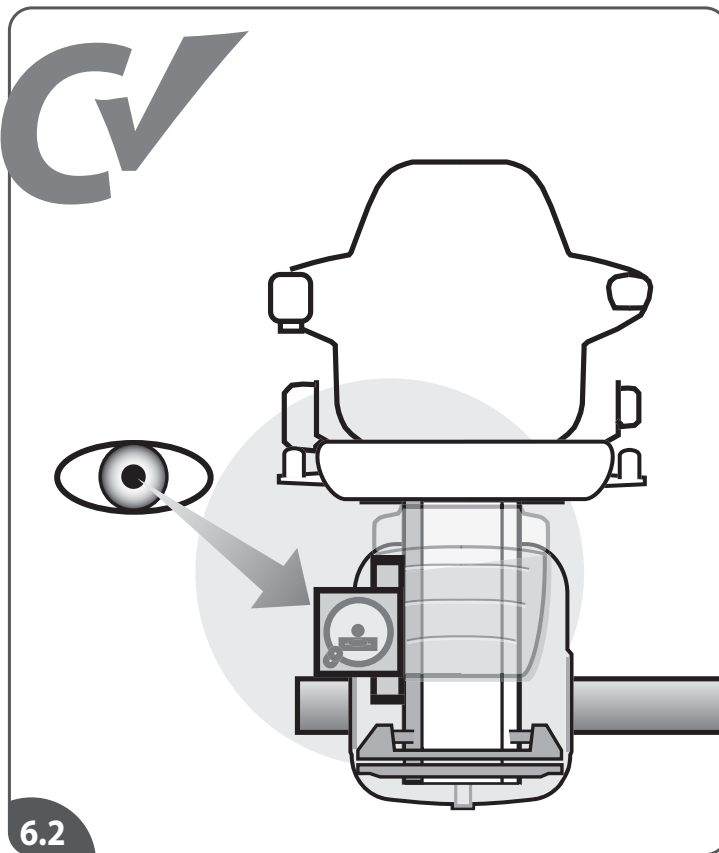
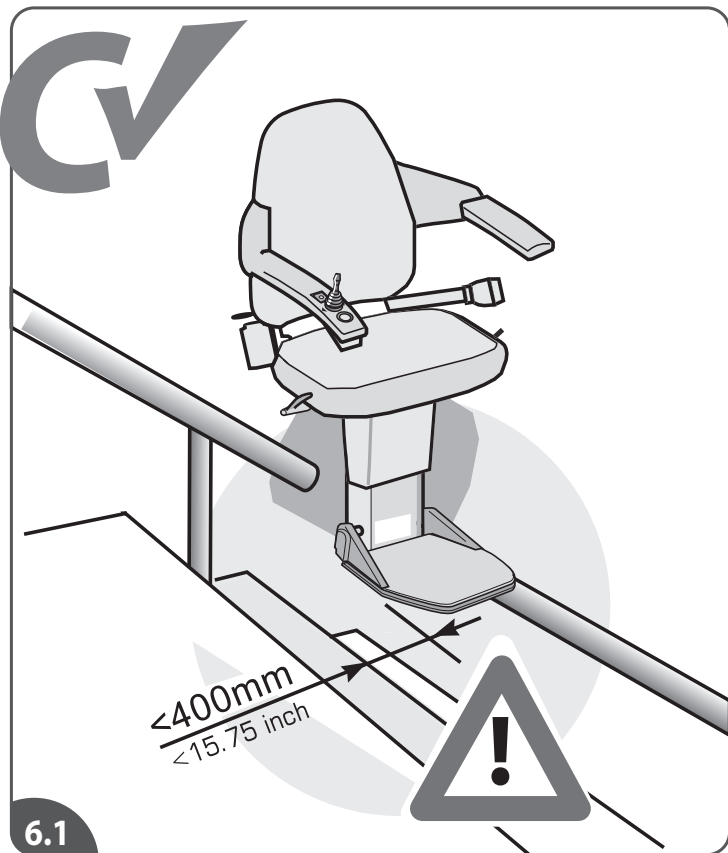
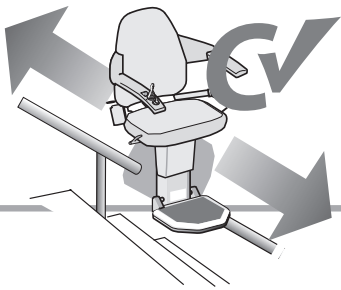
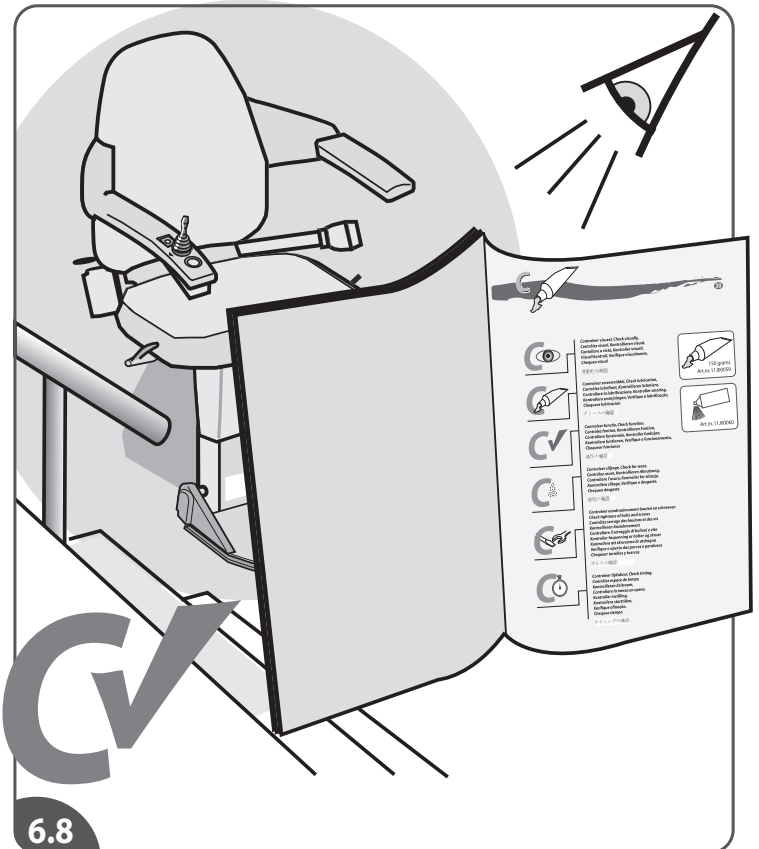
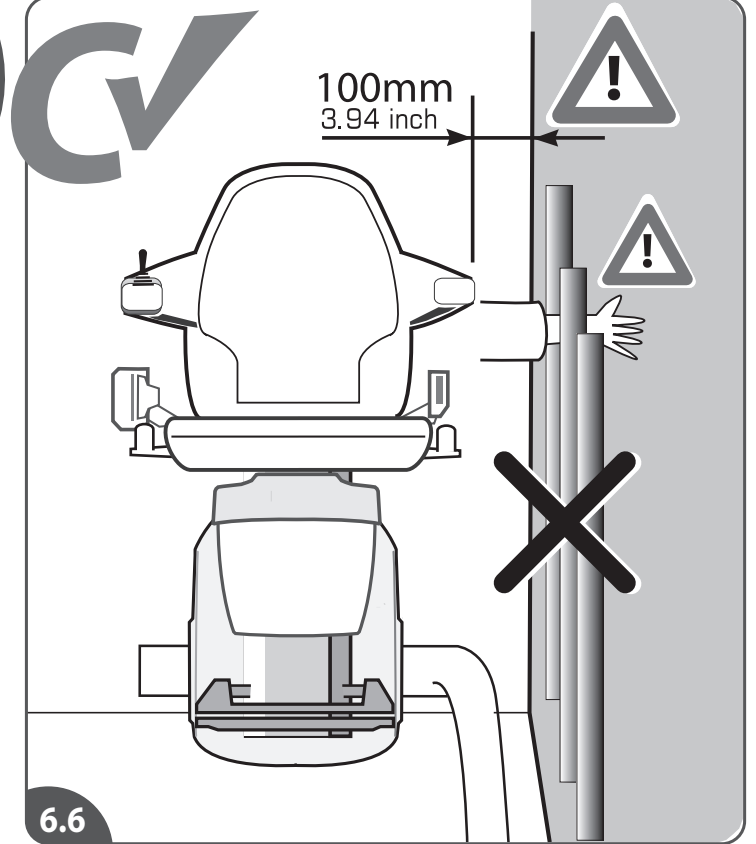
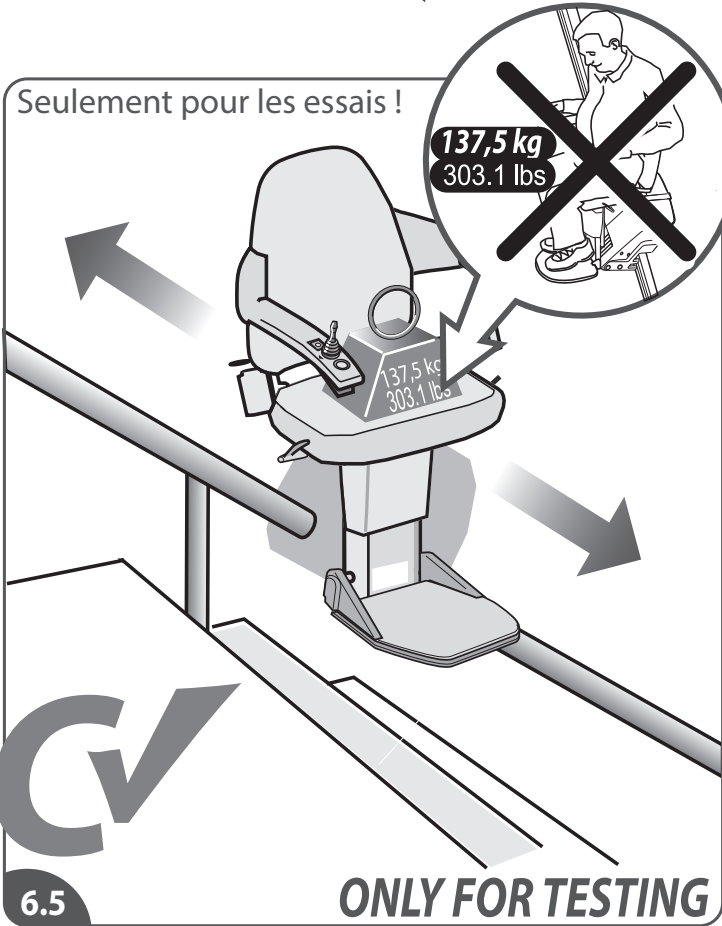
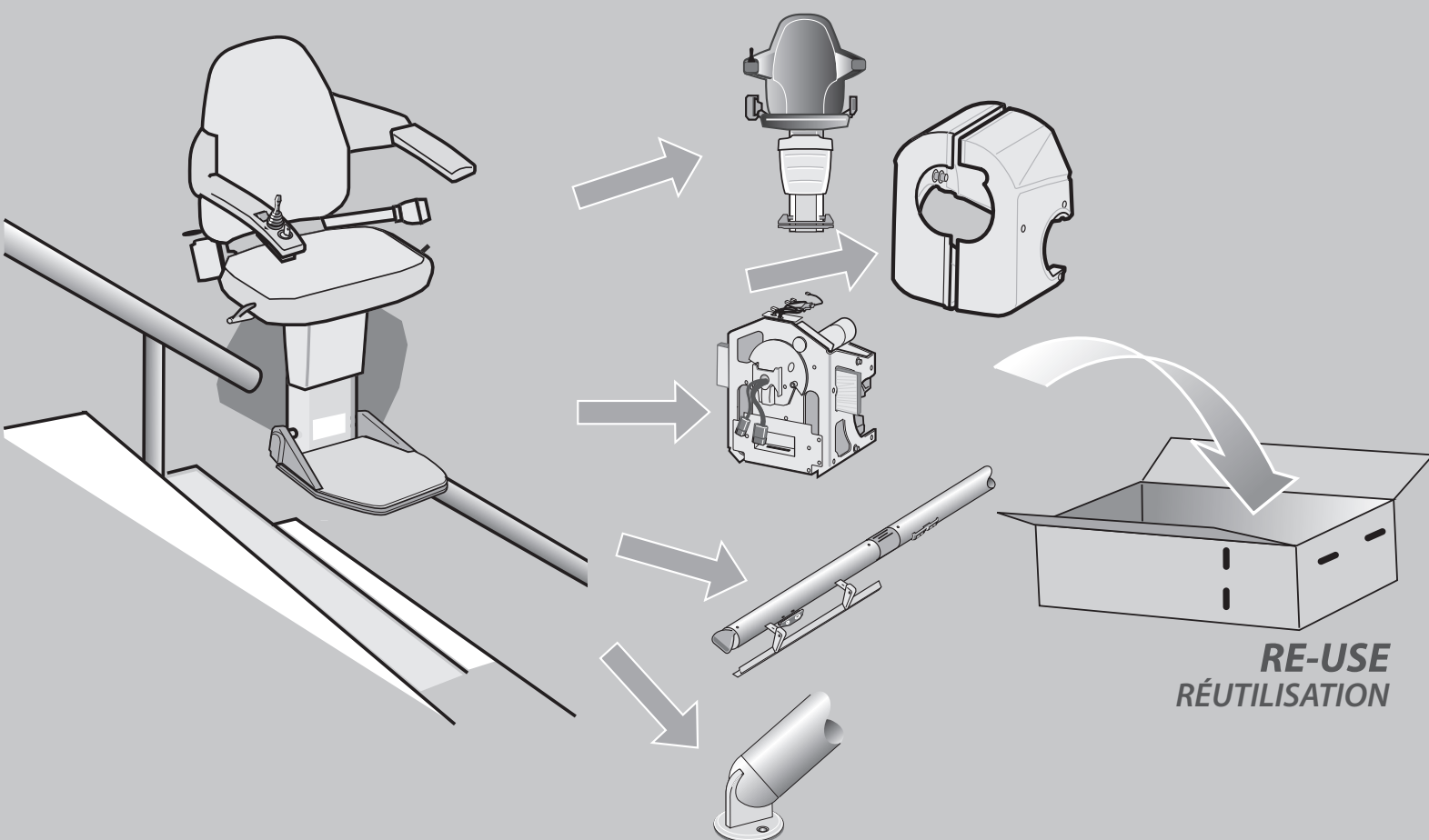
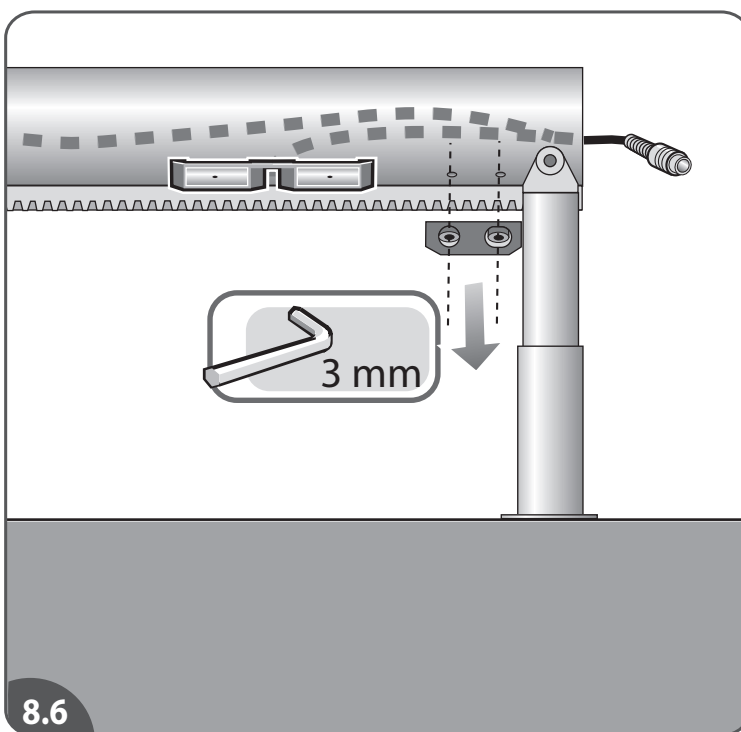
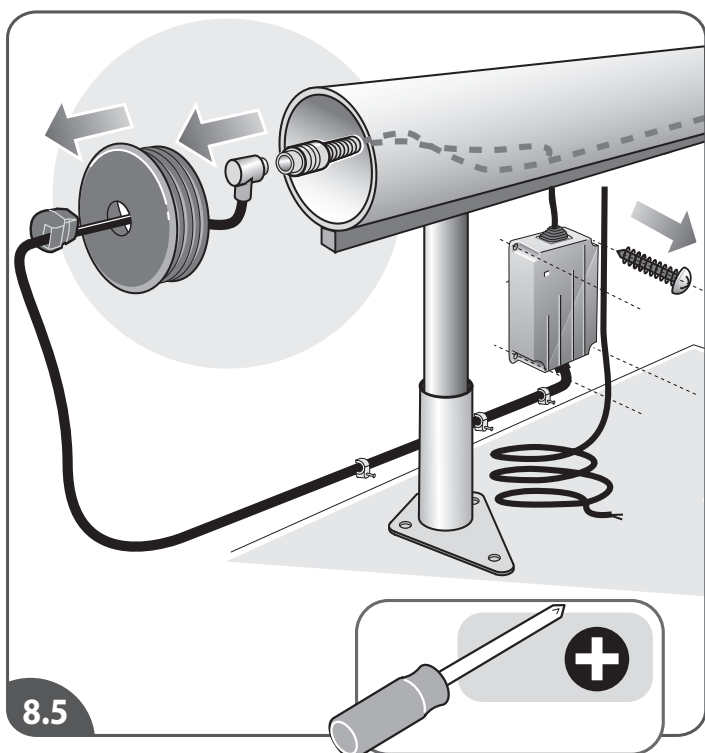
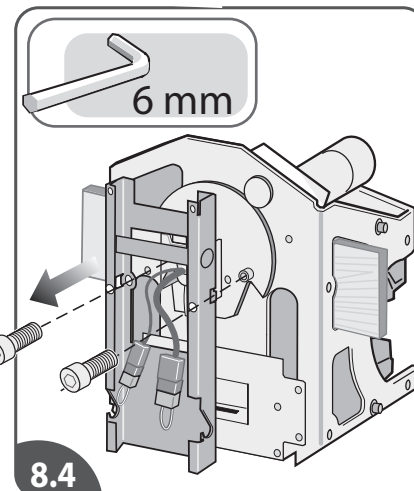
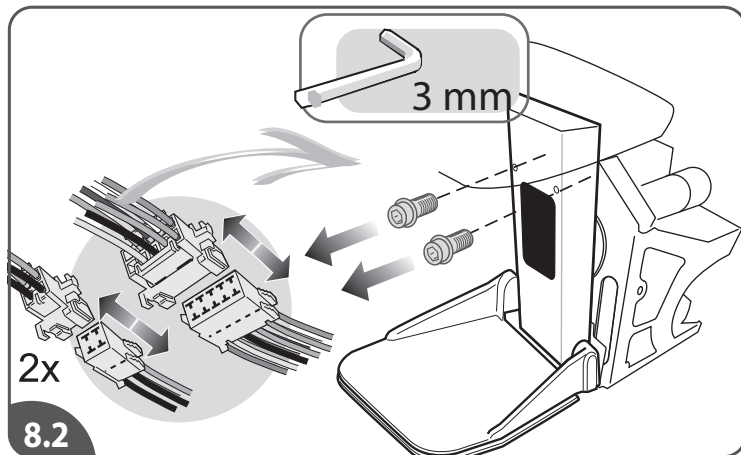
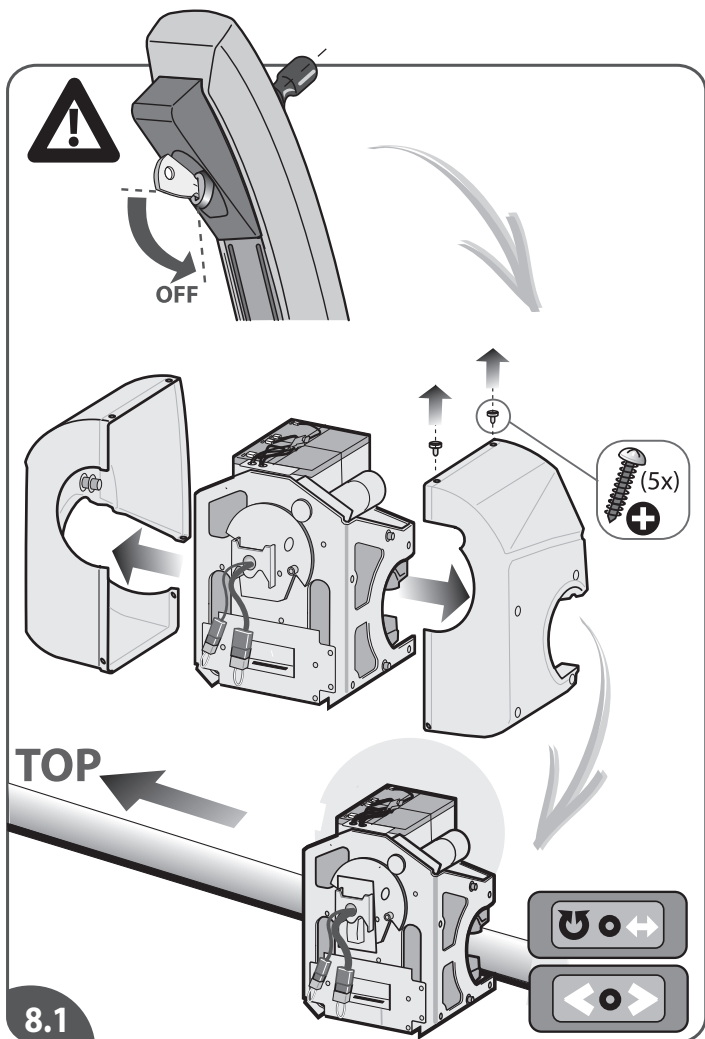
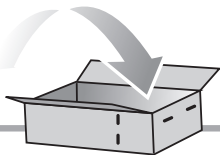
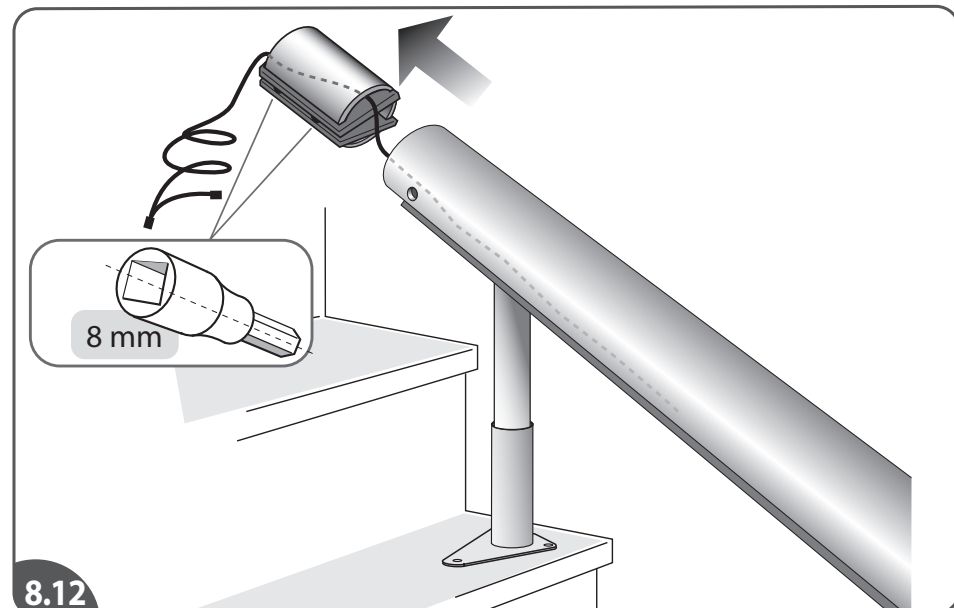
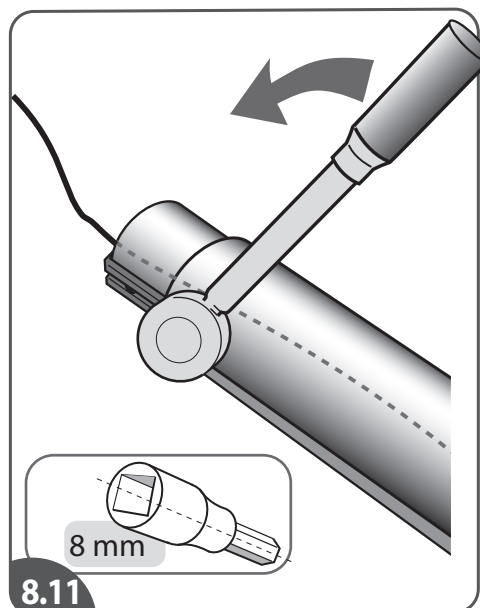
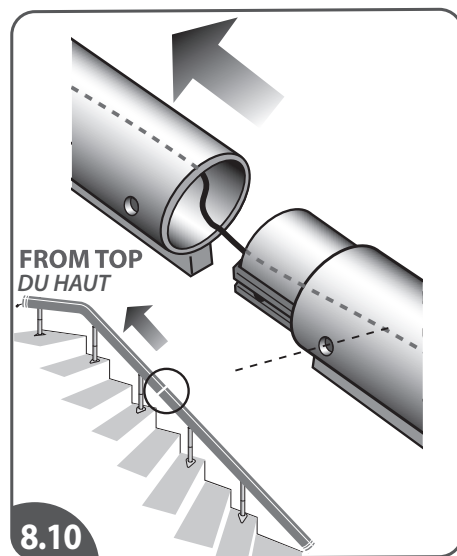
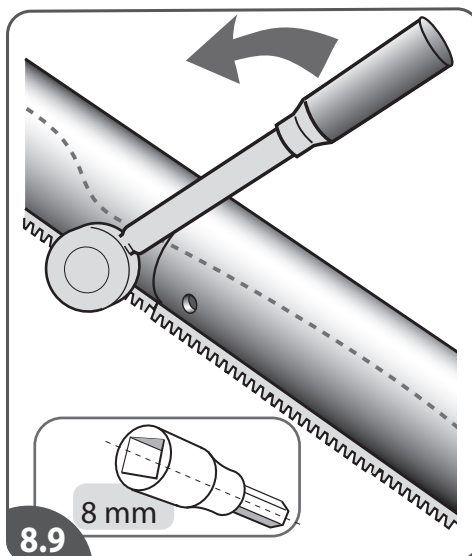
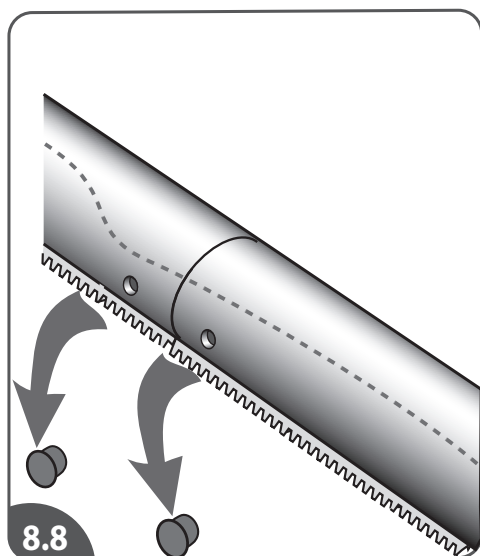
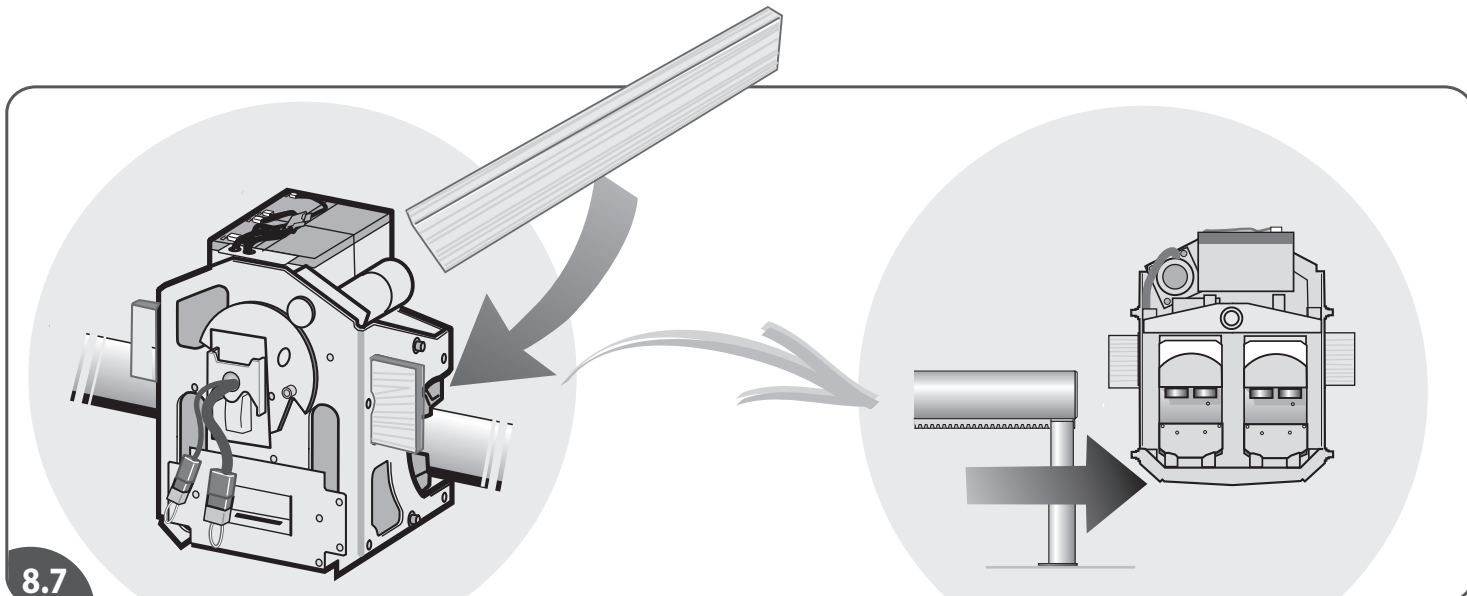
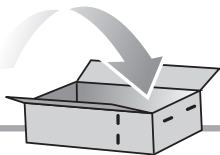
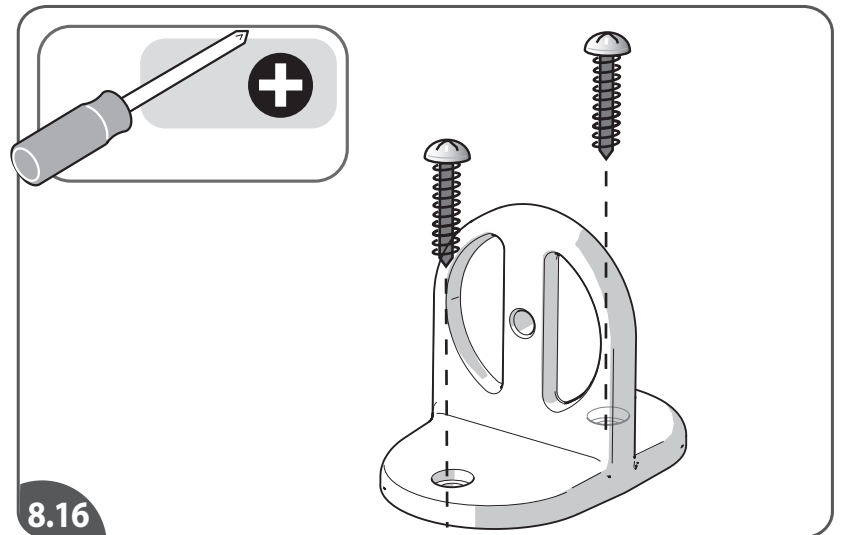
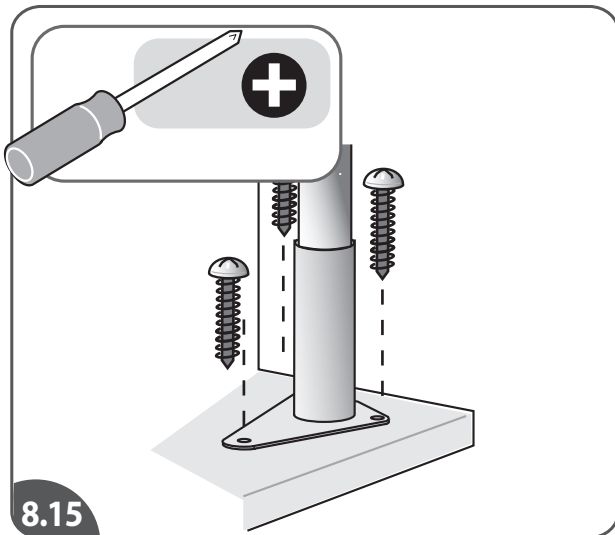
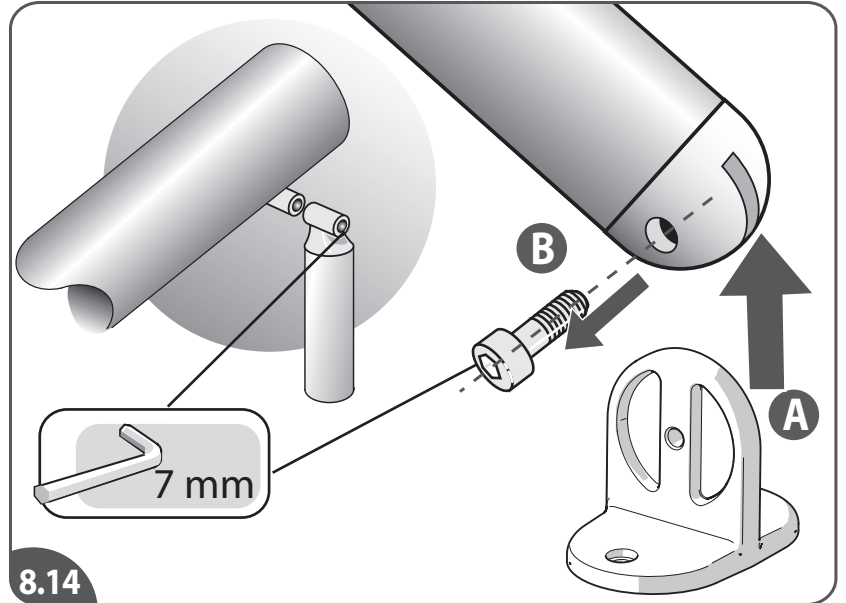
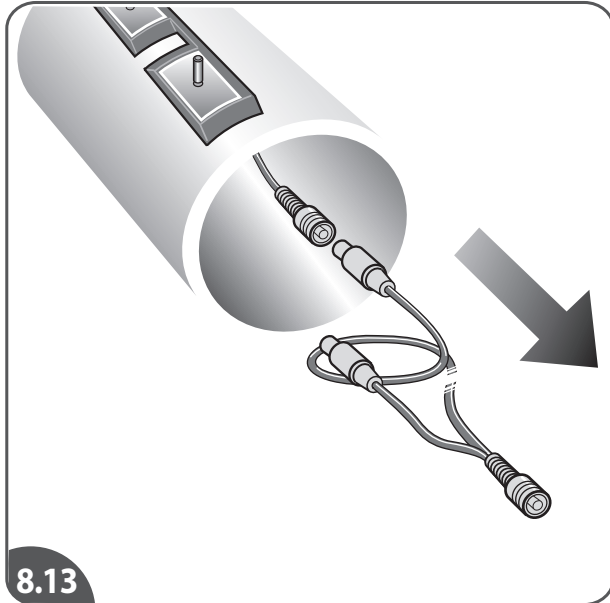
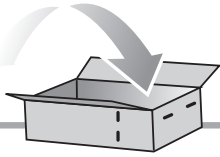
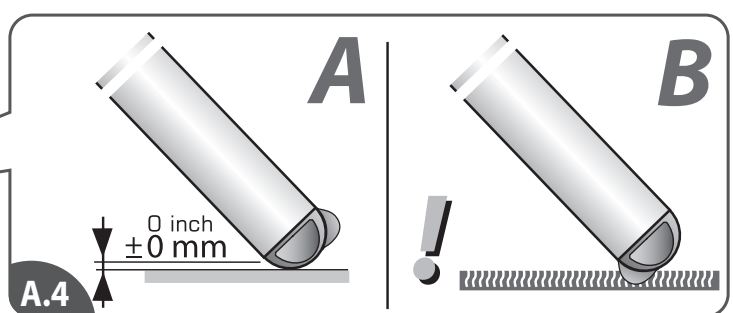
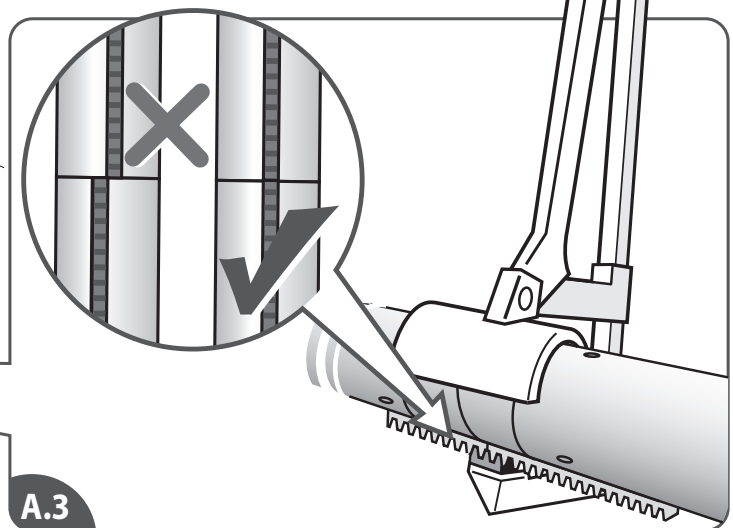
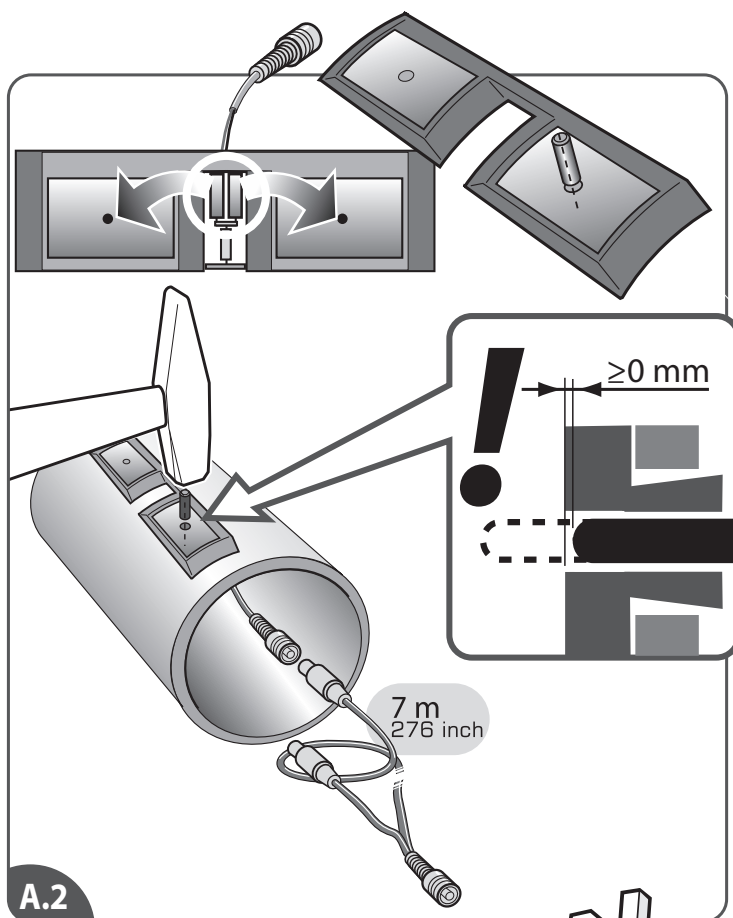
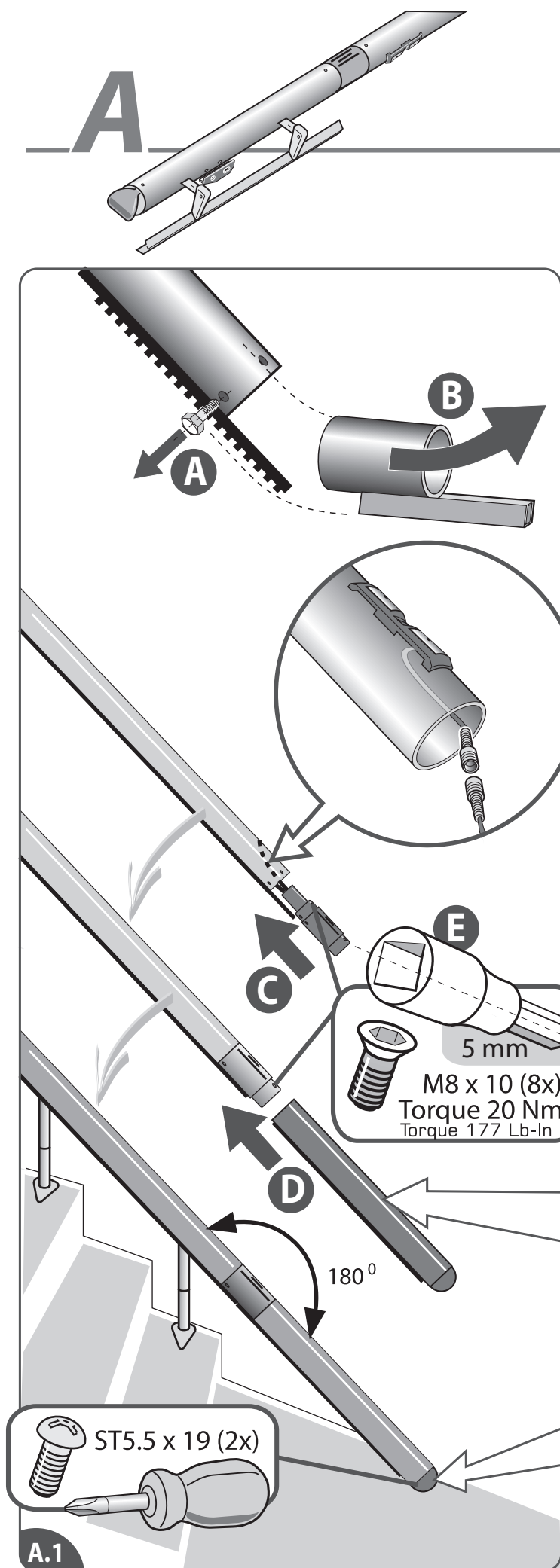
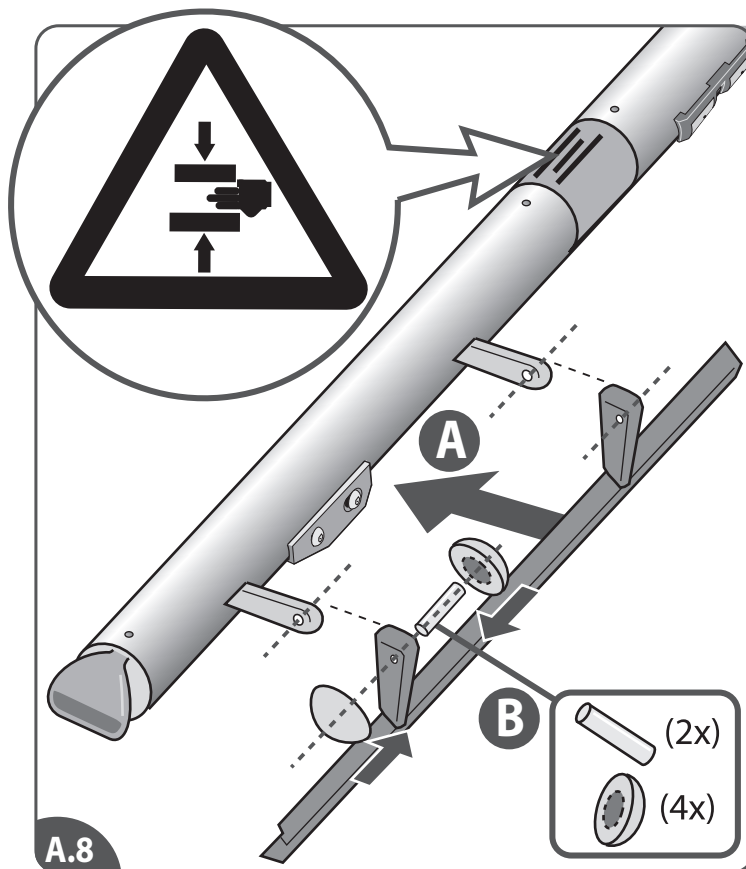
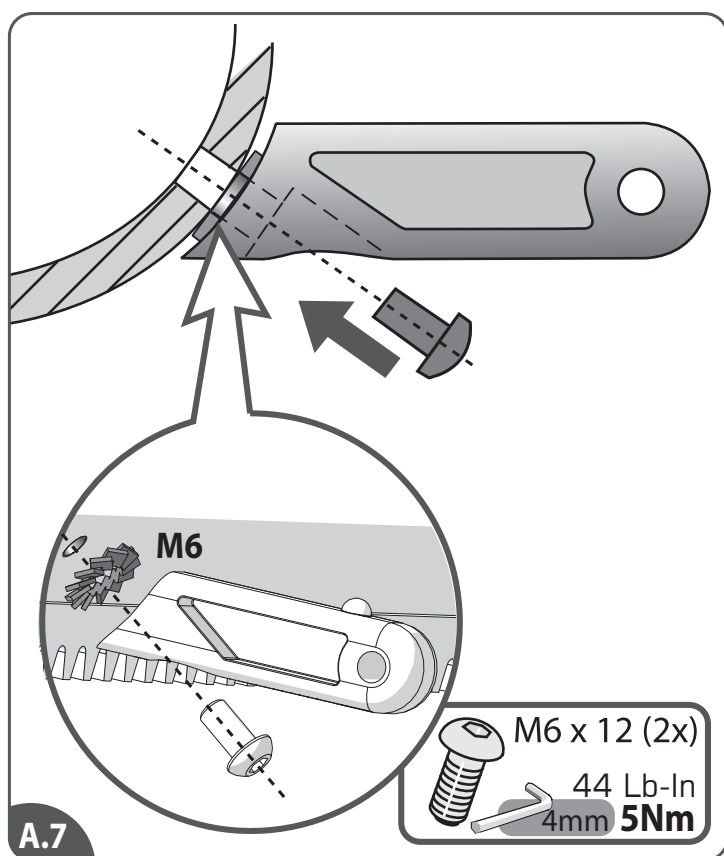
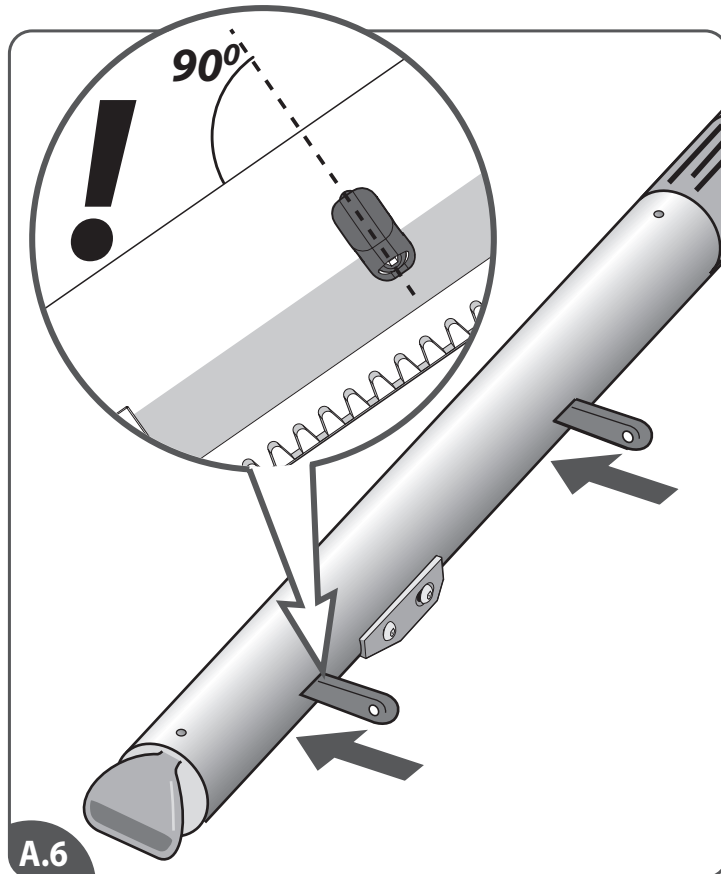
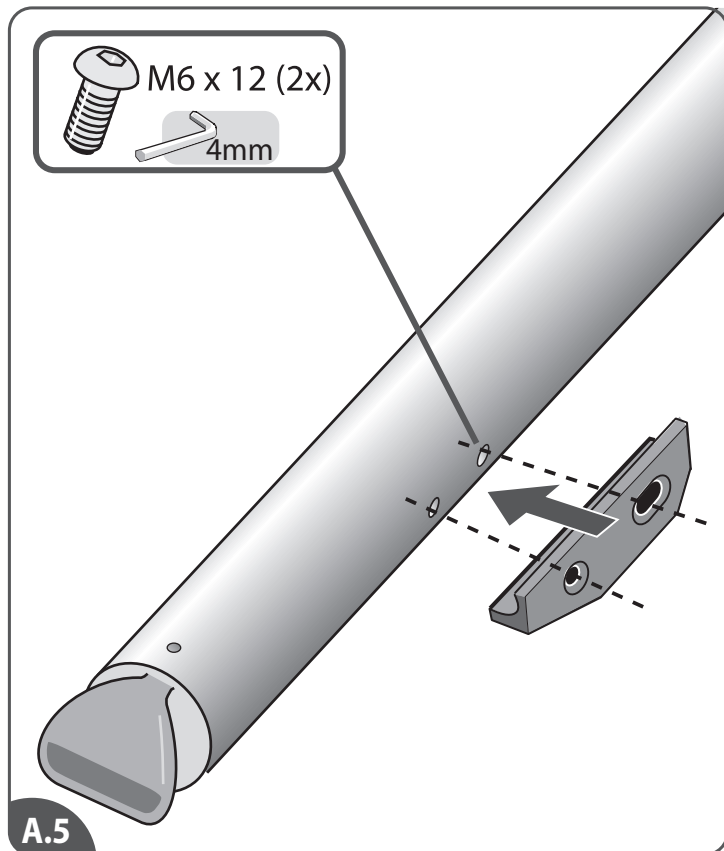
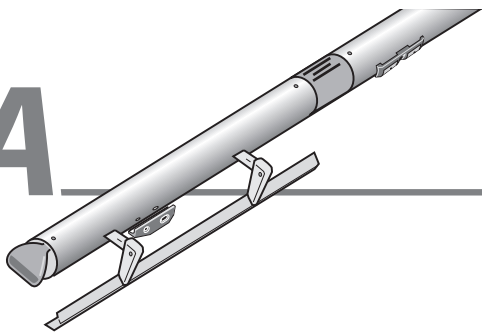
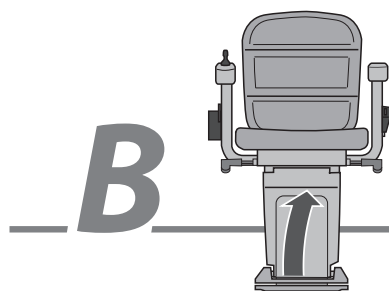
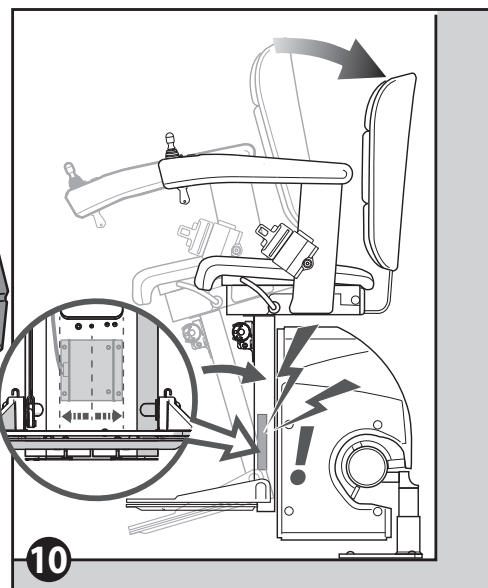
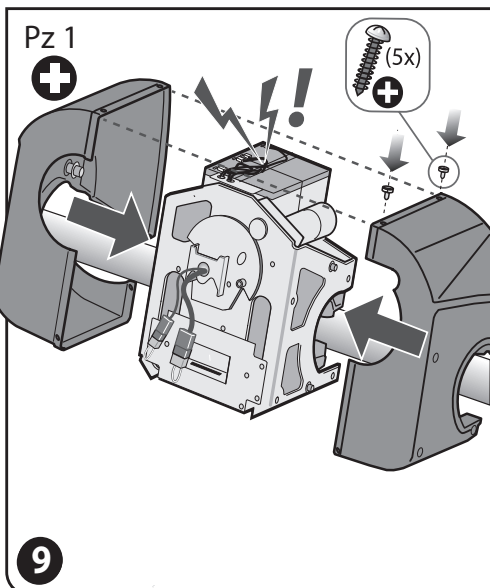
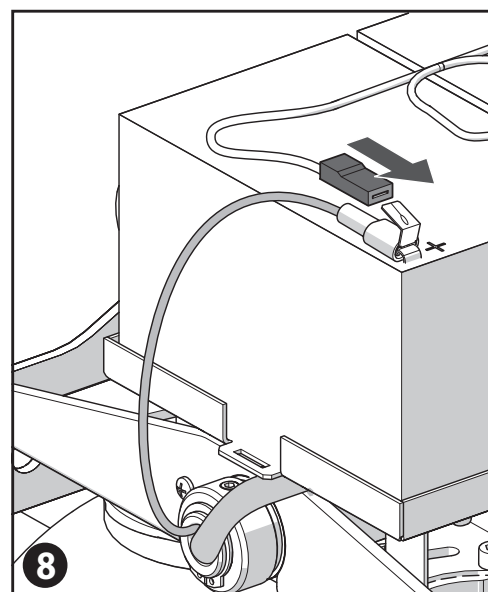
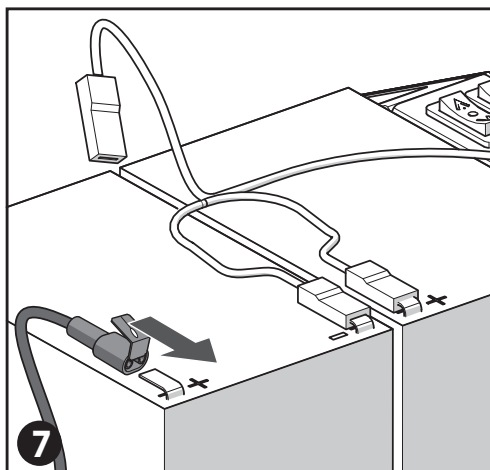
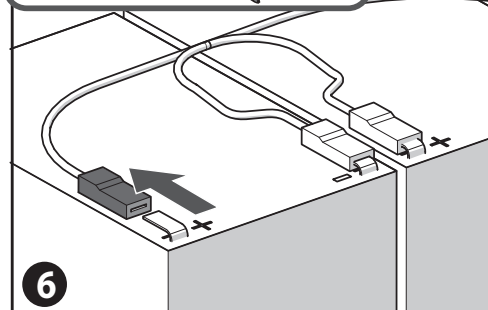
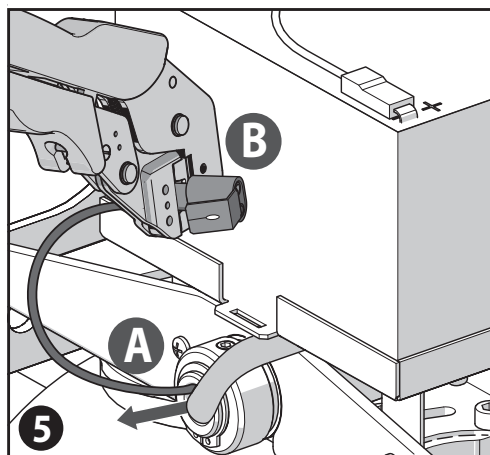
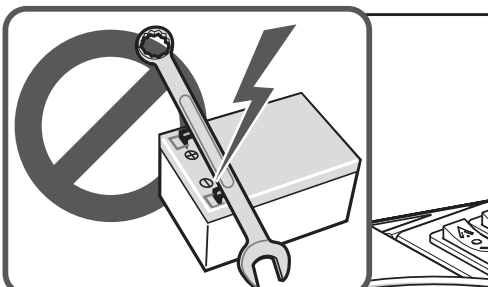
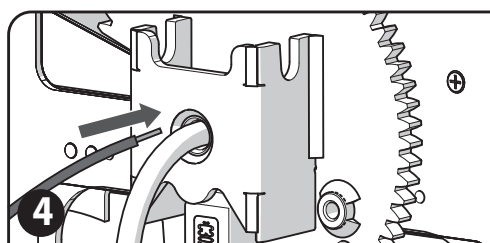
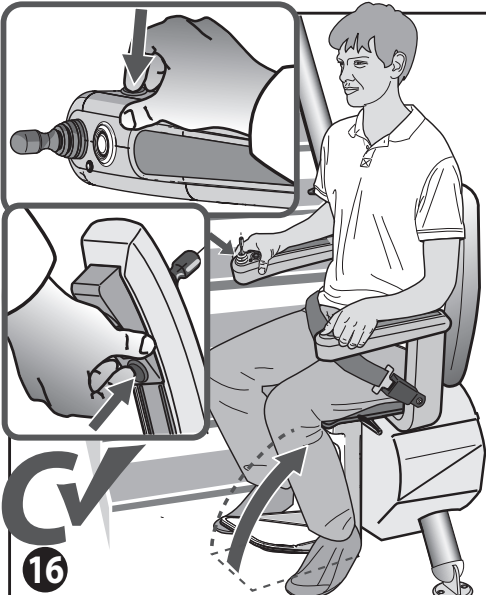
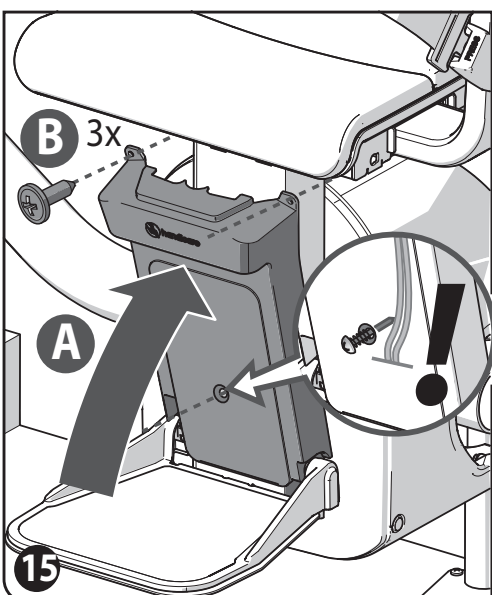
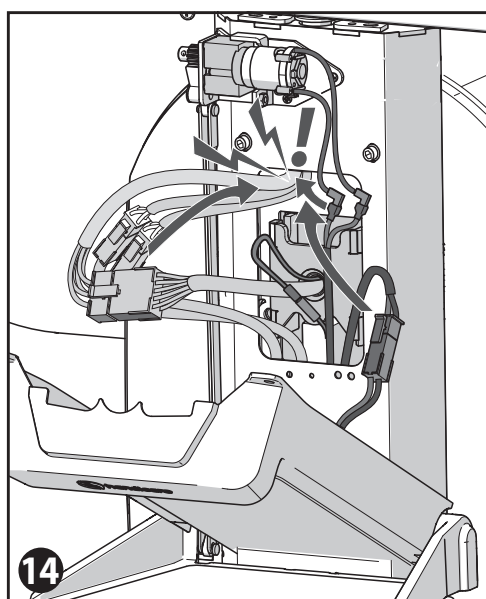
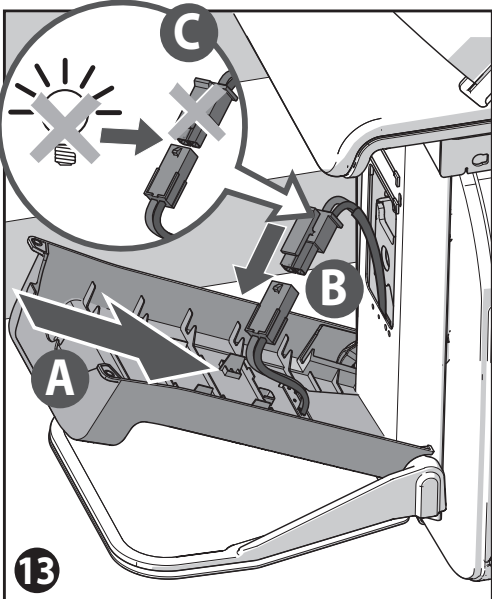
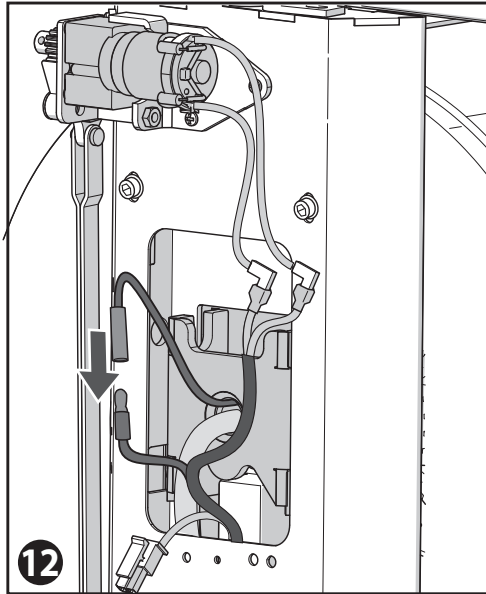
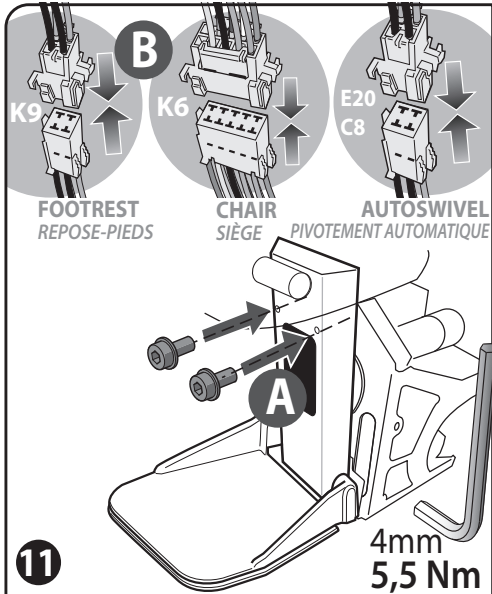
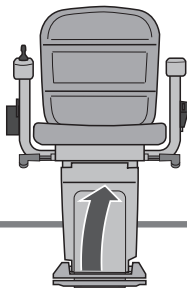
UK [5]Guide the wire far enough through the axle{A}. Use a pair of pliers to push the special connector onto the wire{B}. [6]Disconnect the connector from the positive terminal. [7]Connect the separate wire to the positive battery terminal. [8]Attach the battery wire to the special connector. [9]Refit the covers onto the power pack. Mind the wires on top of the battery. [10]Refit the seat onto the power pack. Be careful with the printed circuit board and the wiring on the back of the frame.

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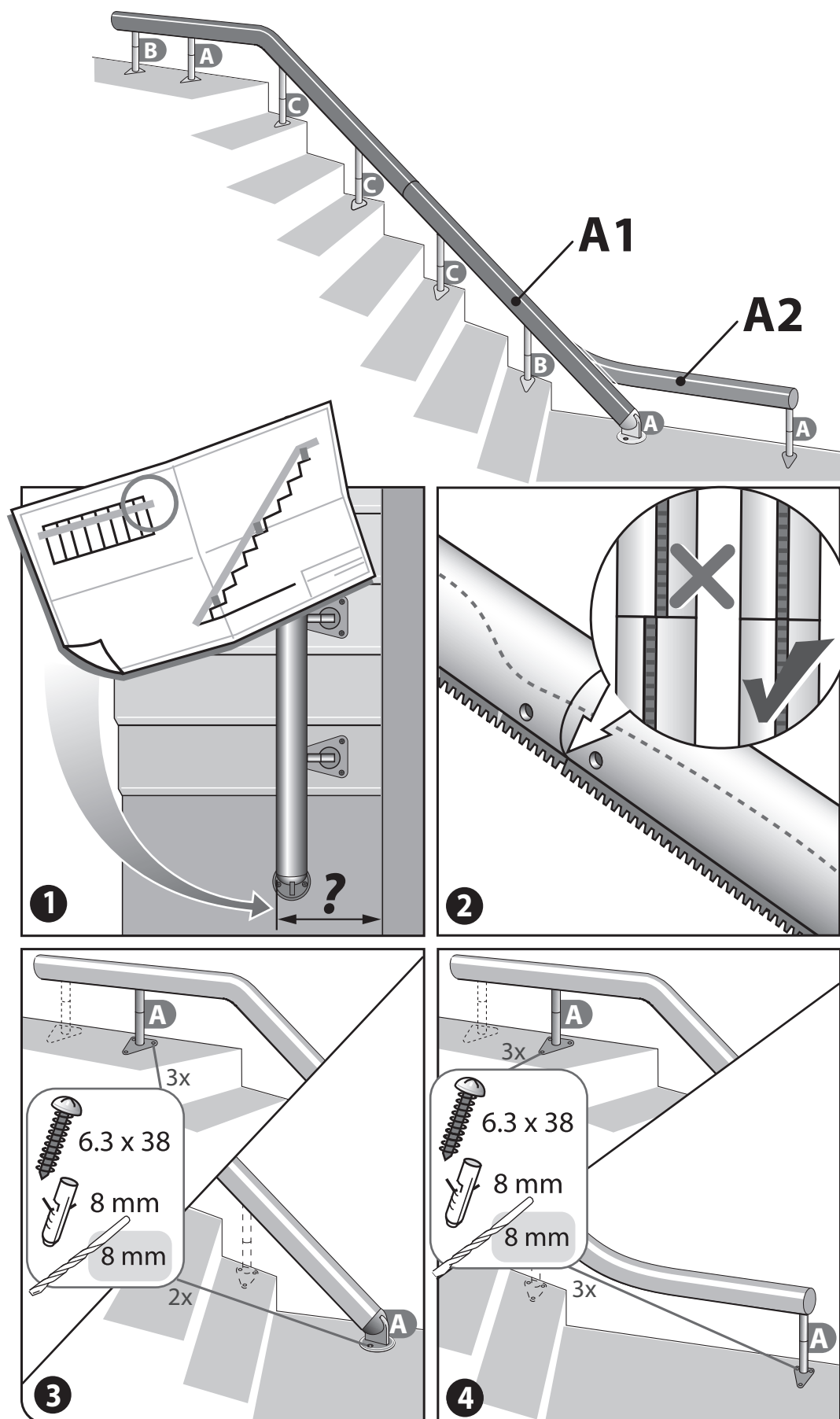
UK [11]Screw the seat to the power pack{A}. Reconnect the seat connectors{B}. [12]Connect the separate wire to the wire of the printed circuit board for the powered footrest. [13]Refit the cover{A} and connect the connectors for the lights{B}. If your customer does not want lights, do not connect these connectors{C}. [14]Carefully arrange all cables and connectors in the frame. [15]Close the cover{A} and tighten the screws{B}. Make sure that the bottom screw does not damage the wires. [16]Test the powered footrest and lights for correct operation.

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C A-RAILS [ADDITIONAL] A-RAILS [SUPPLÉMENTAIRES]

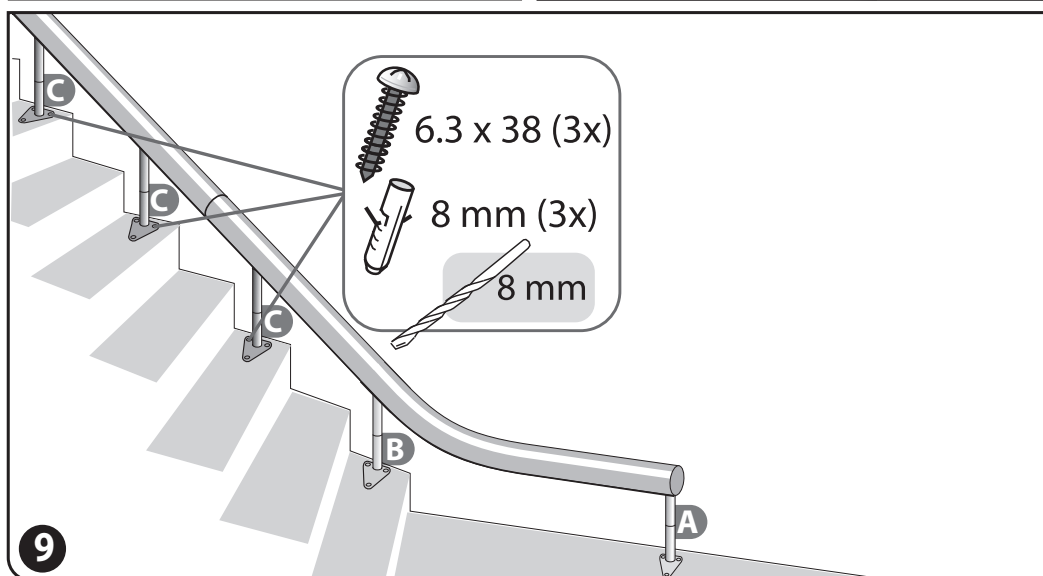
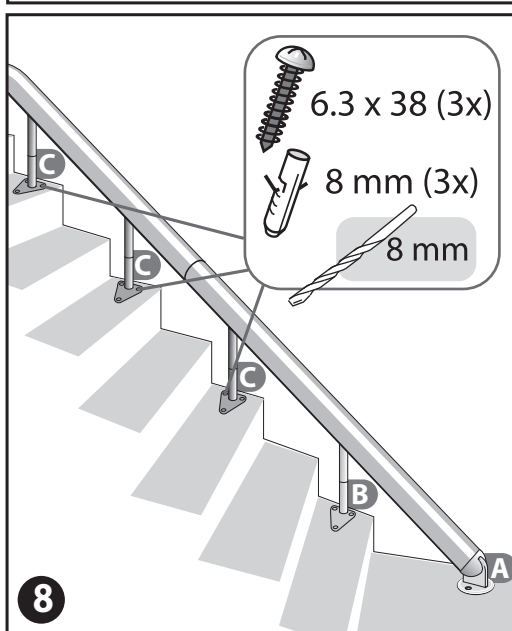
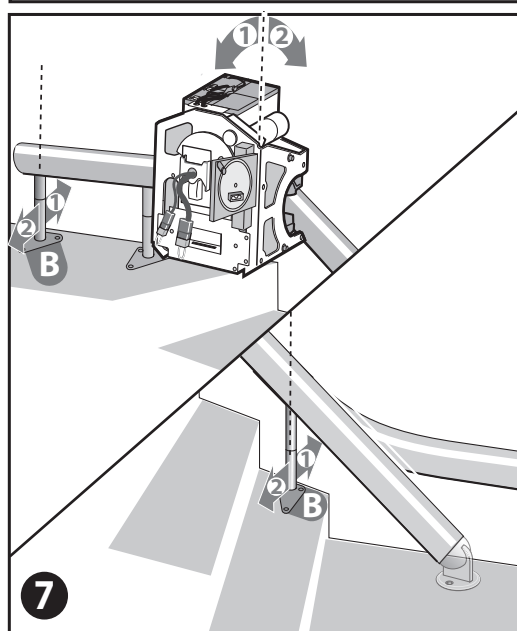
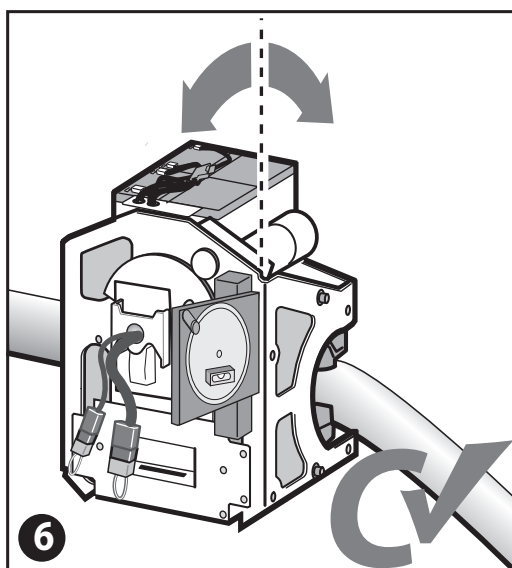
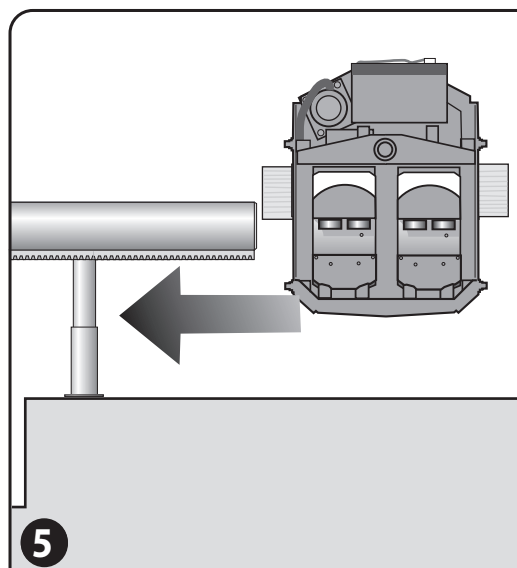
UK [1] Position the rail on the stairs. The distance to the wall is mentioned on the enclosed drawing. [2] After installing of the joint according to the manual [page 8/9], do an extra check on the alignment of domexstrip/tooth-rail. [3] Fix two supports {A} on the bottom/top floor according to the manual [page 24]. [4] In case of rail A2 you start with the lower support at the bottom floor and the first support at the top landing.

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UK [5] Ride the unit on the rail and position it in the curve of the rail. [6] Check with a spirit level if the unit is exactly vertical. [7] If the unit leans forwards{1} or backwards{2}, the rail shall be adjusted by repositioning of the upper and lower support{B}. This will tilt the rail forwards or backwards. In case of rail A1 the support on the staircase first. [8] Fix the other supports to the stairs{C}. [9] In case of rail A2 also fix the other supports{C}.

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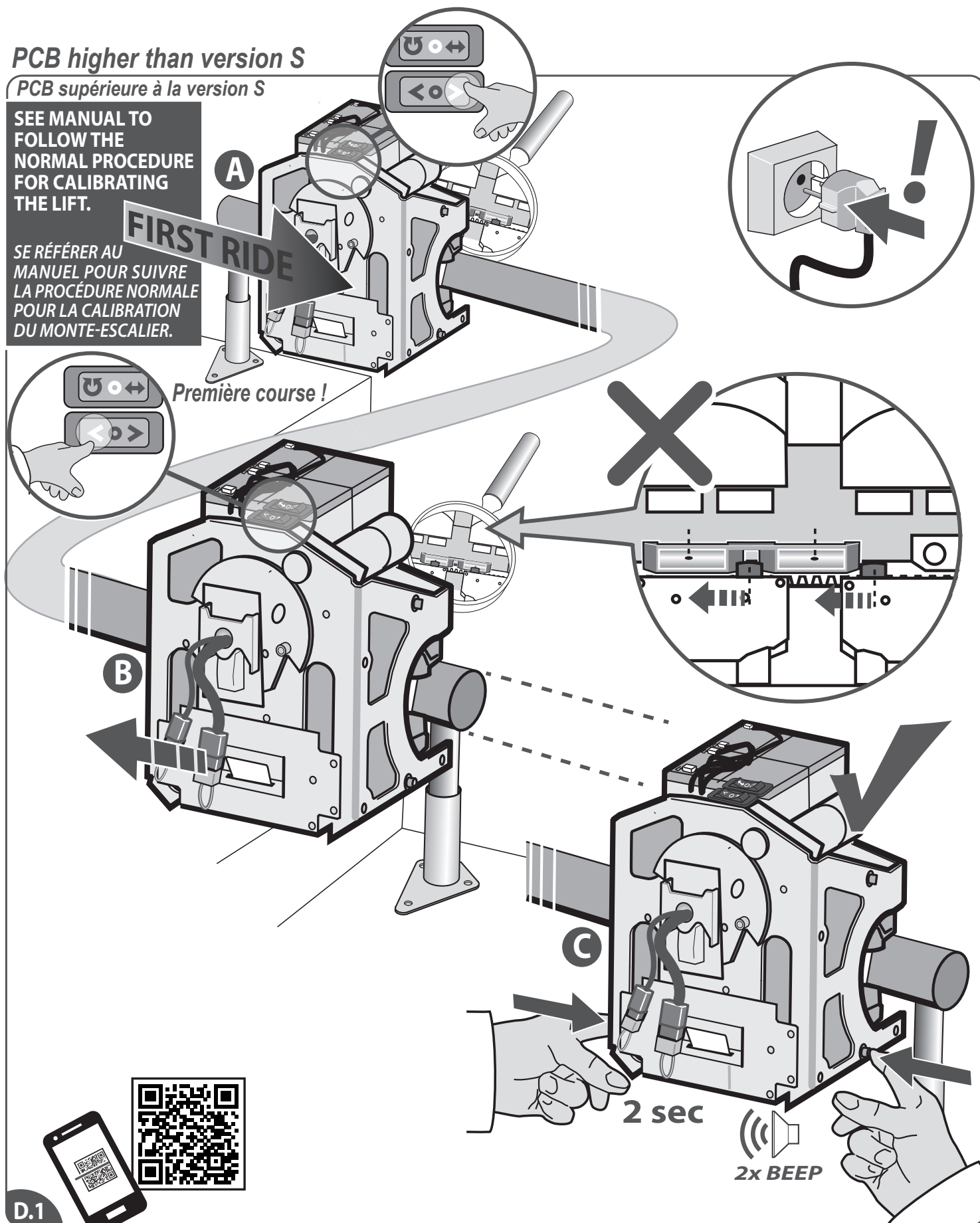
PCB higher than version S

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SEE MANUAL TO FOLLOW THE NORMAL PROCEDURE FOR CALIBRATING THE LIFT.

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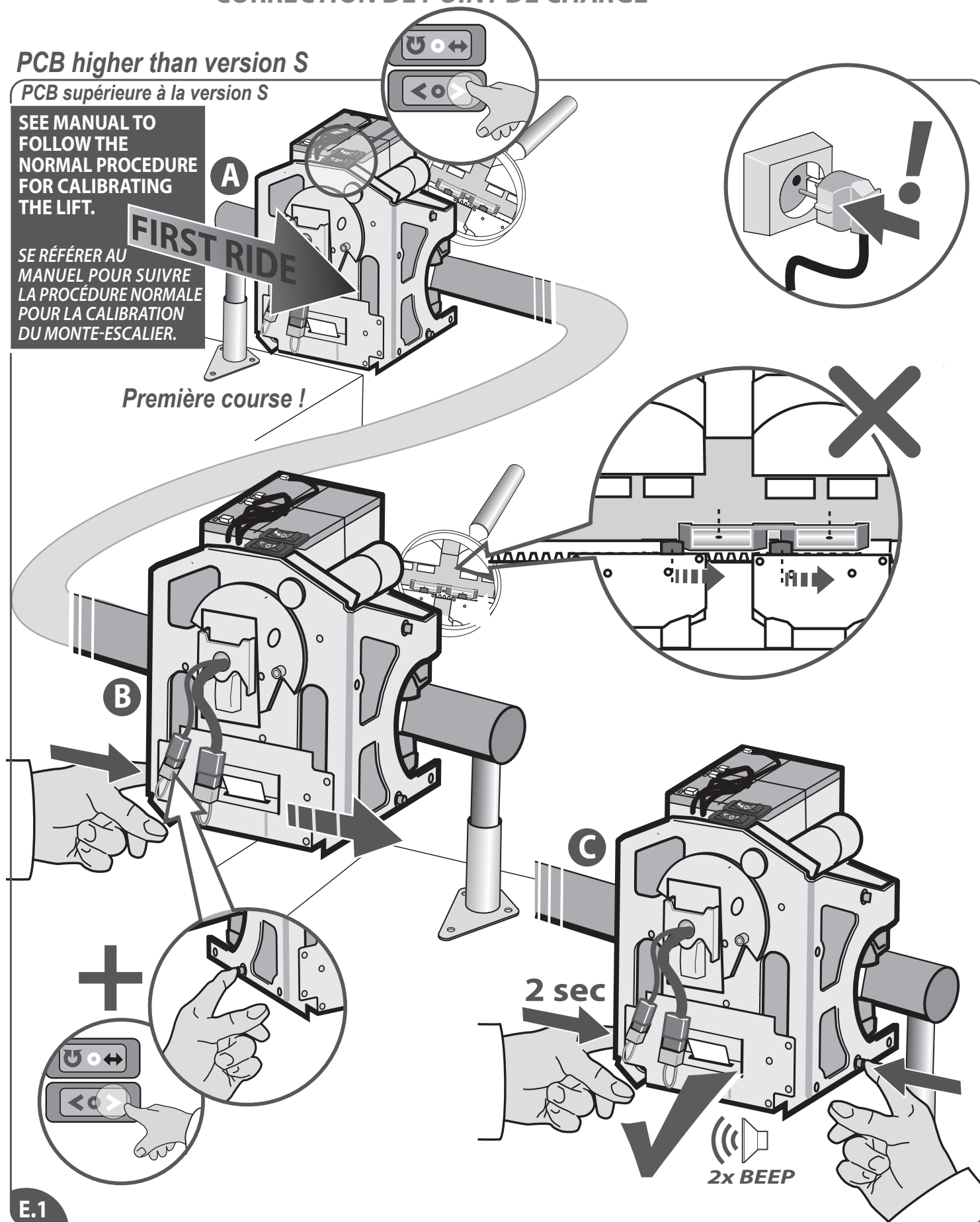
CORRECTION DE POINT
DE RAIL ESCAMOTABLE

PCB higher than version S

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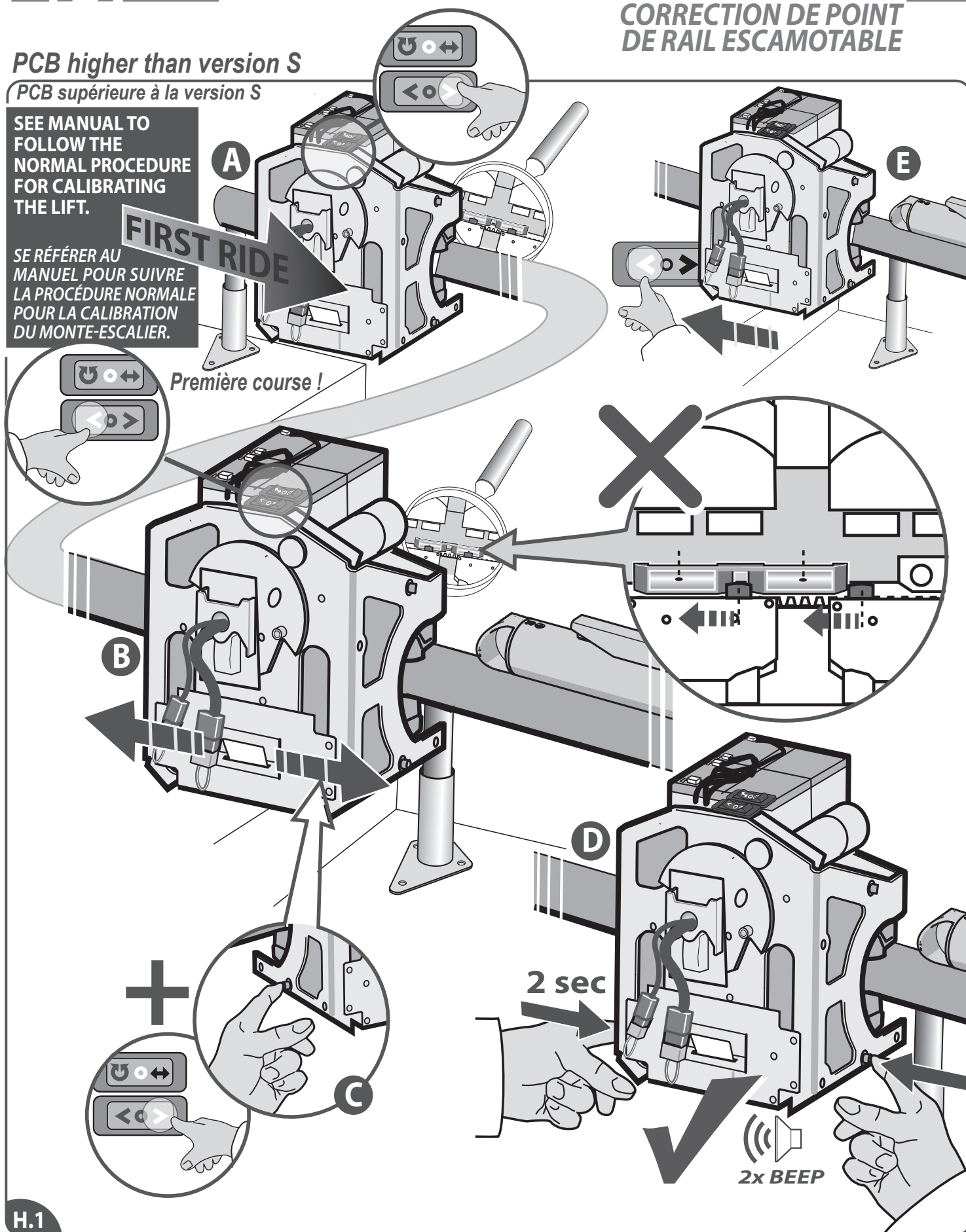
SEE MANUAL TO
FOLLOW THE
NORMAL PROCEDURE
FOR CALIBRATING
THE LIFT.

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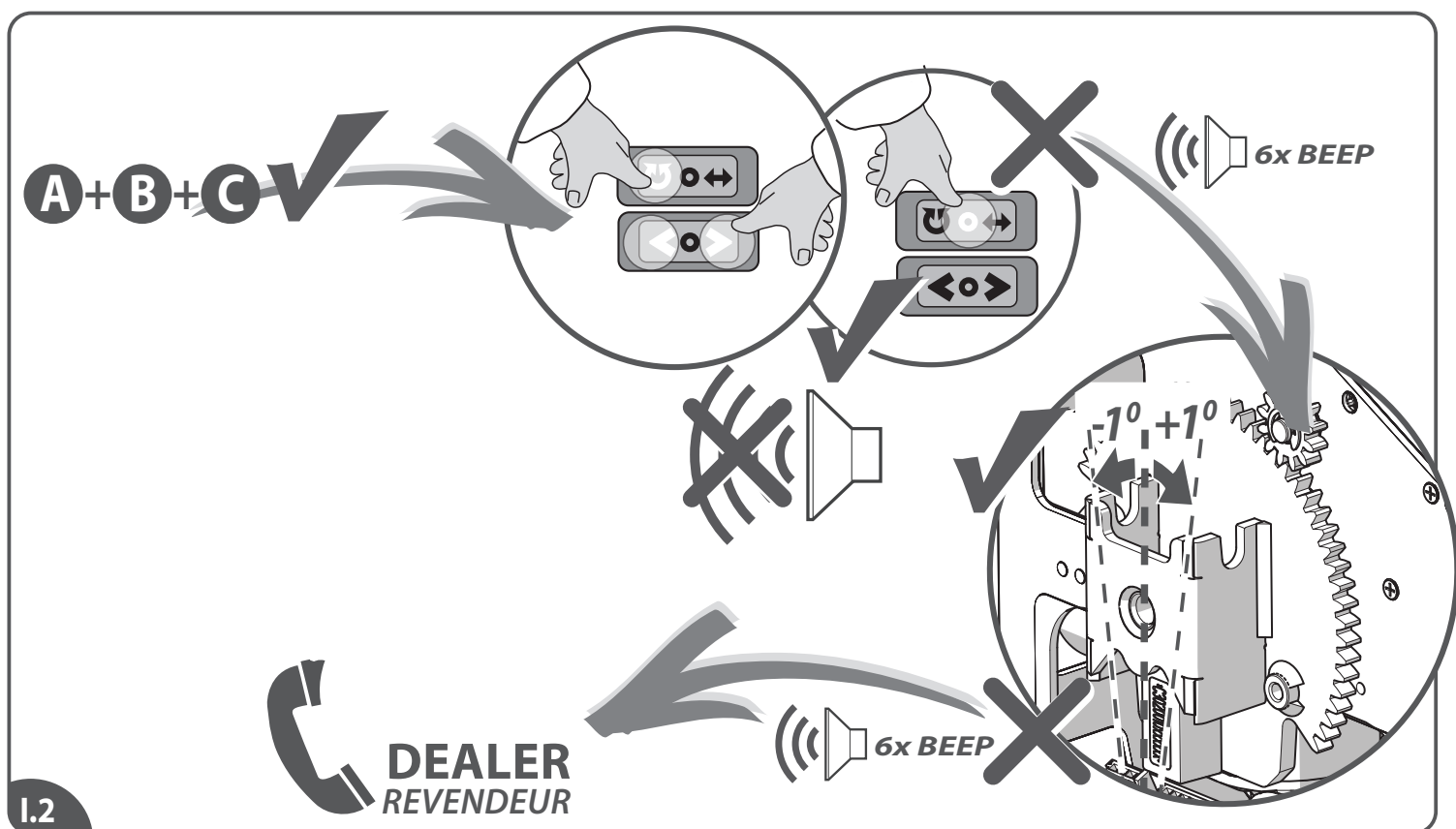
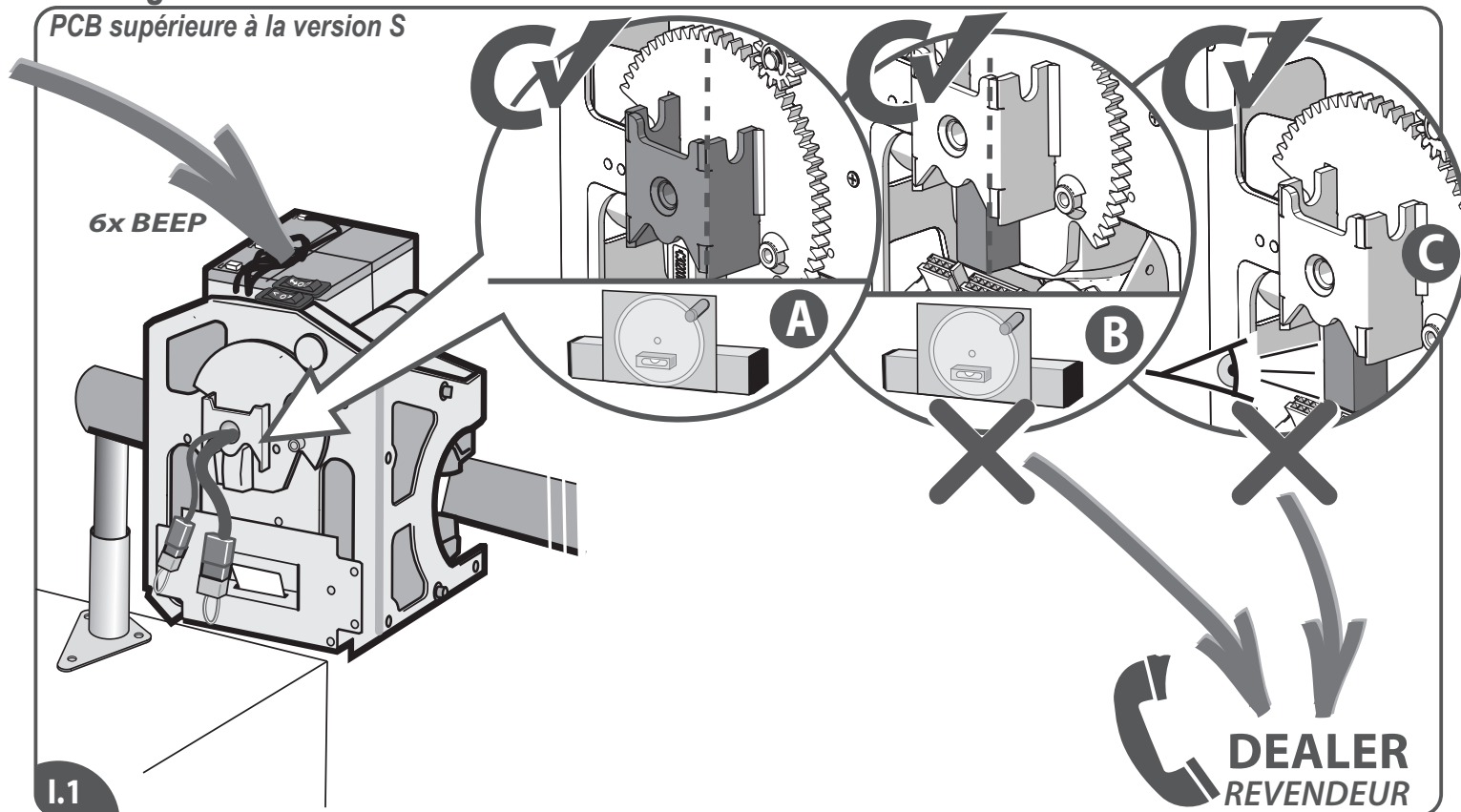
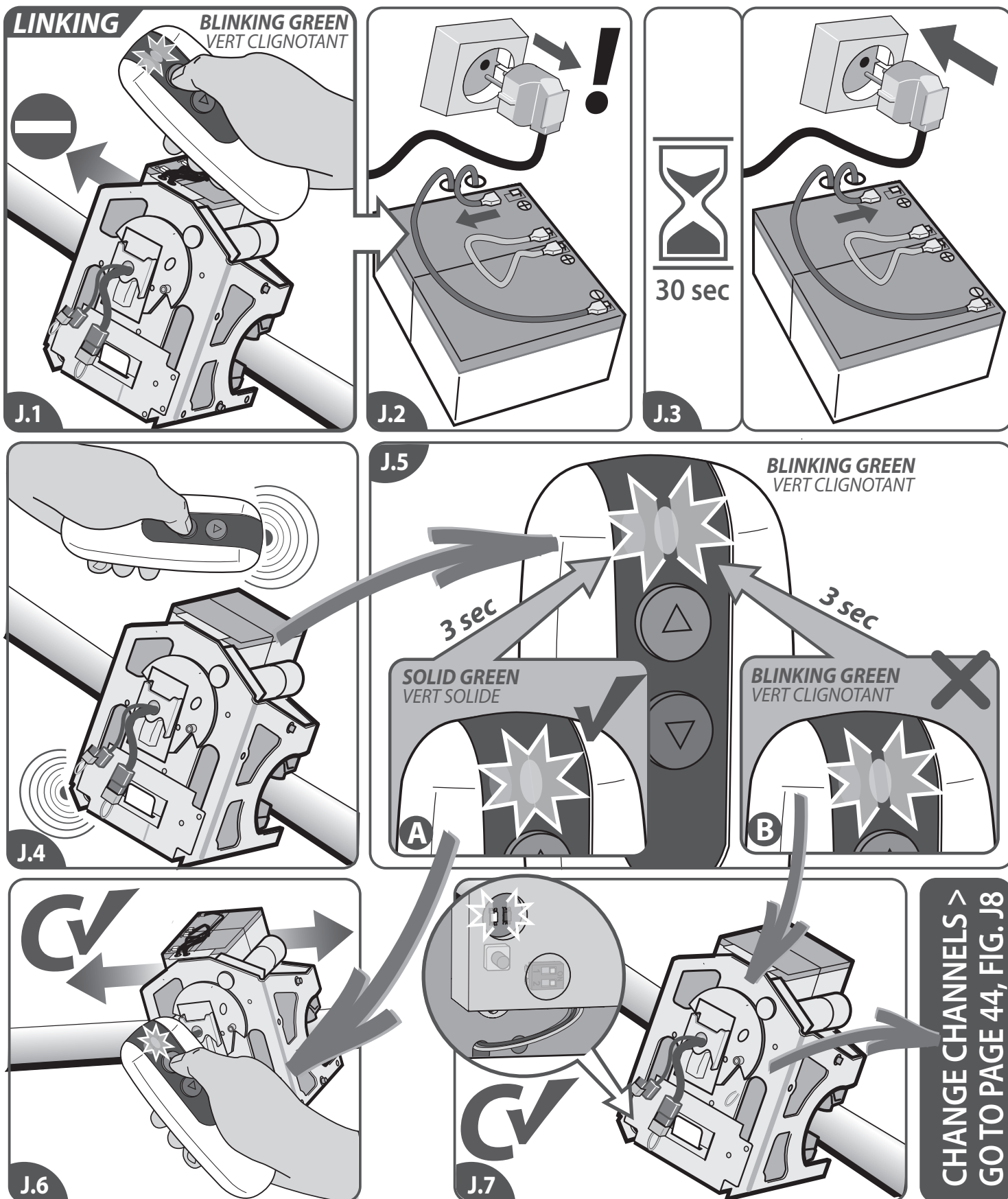
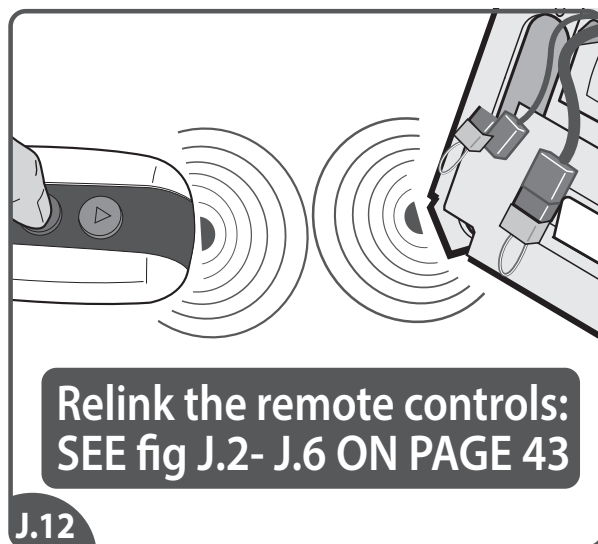
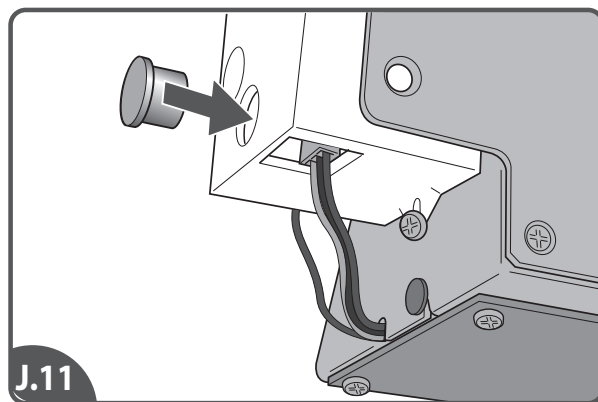
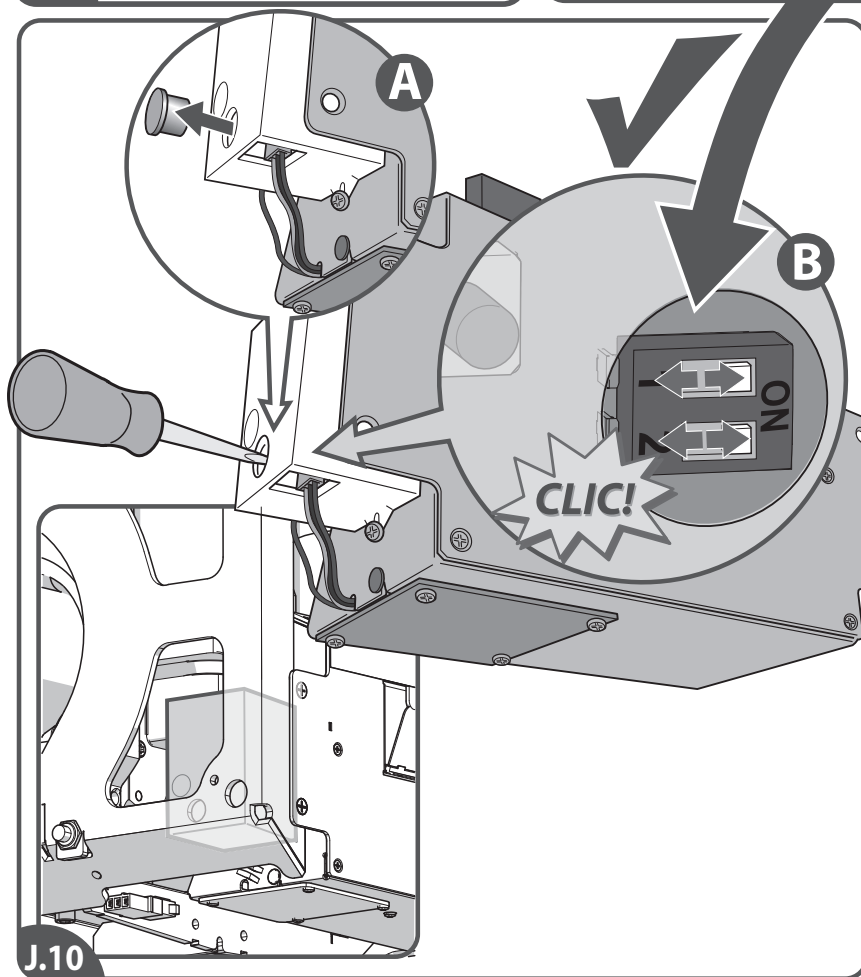
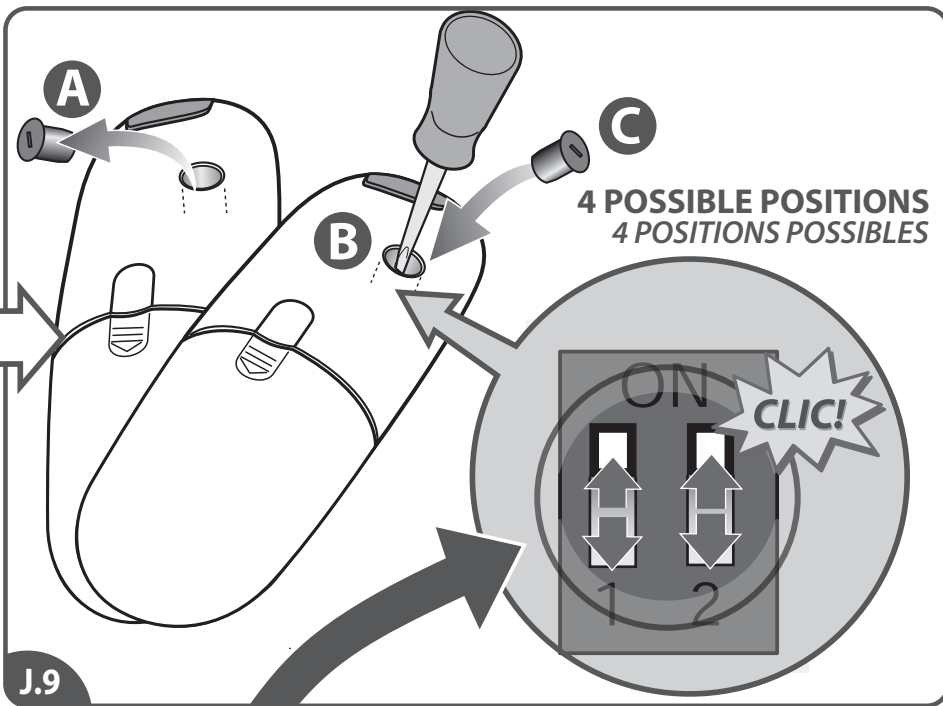
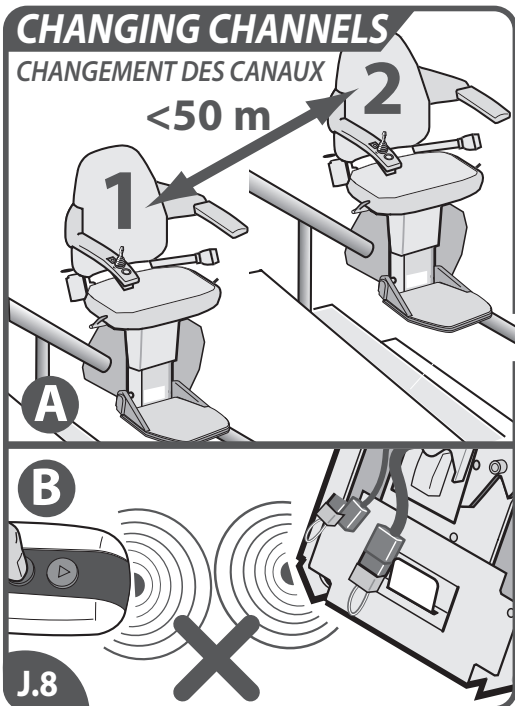
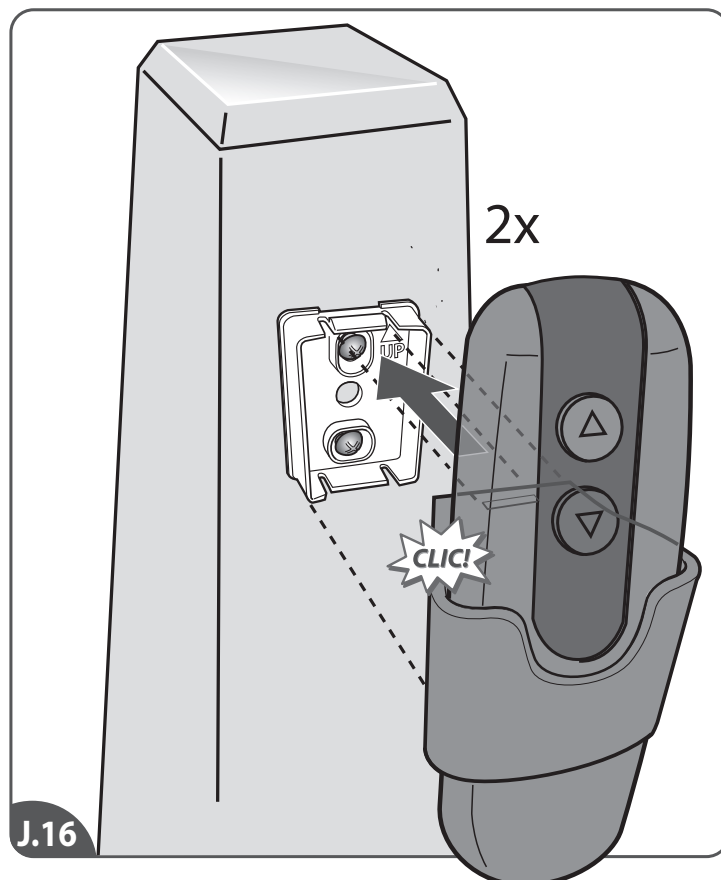
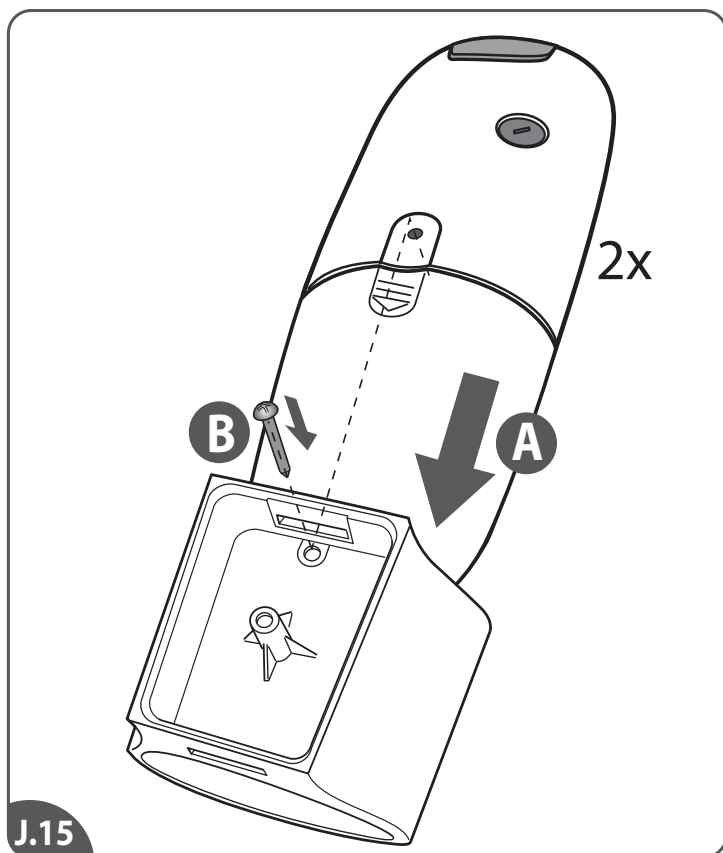
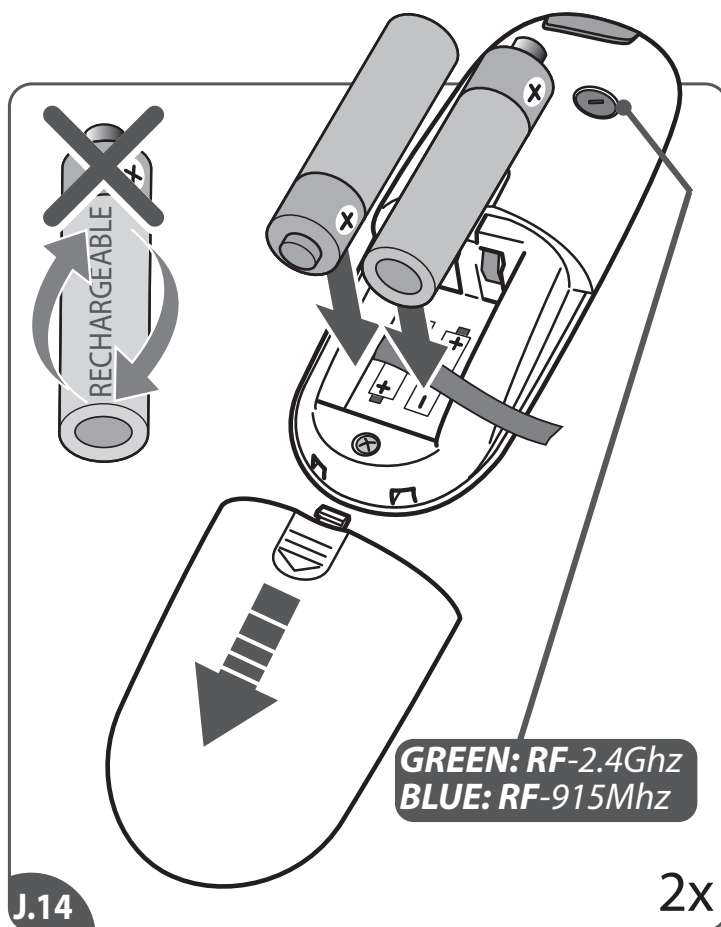
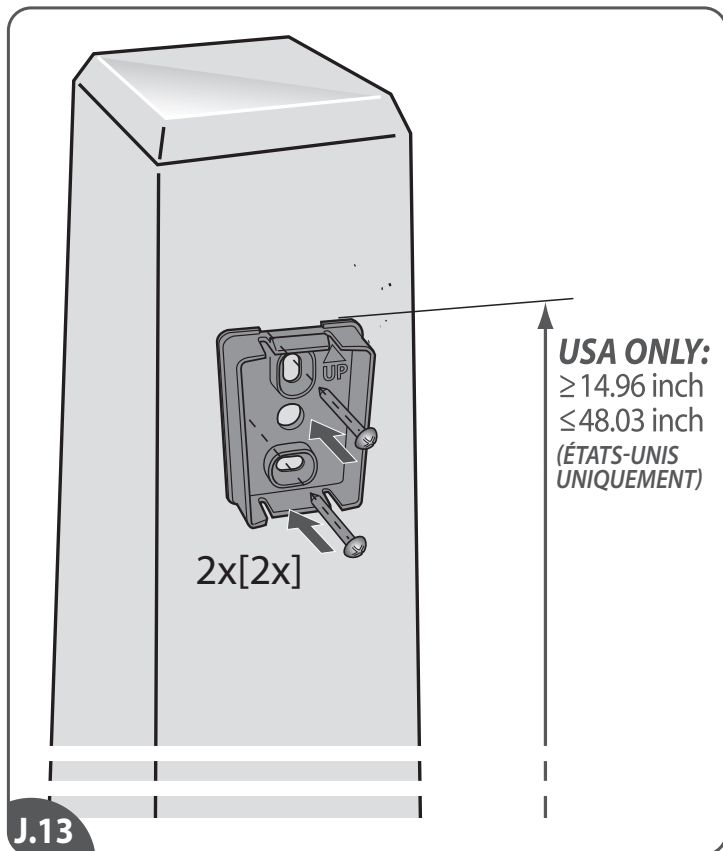
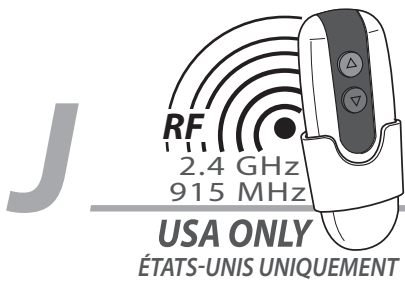
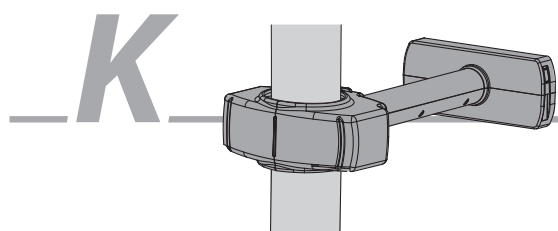
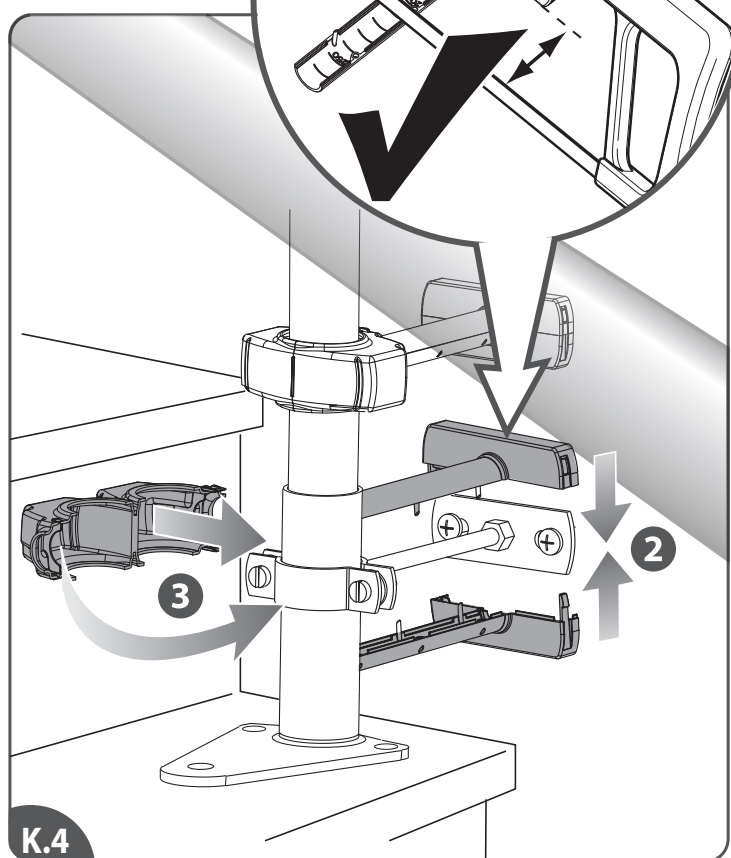
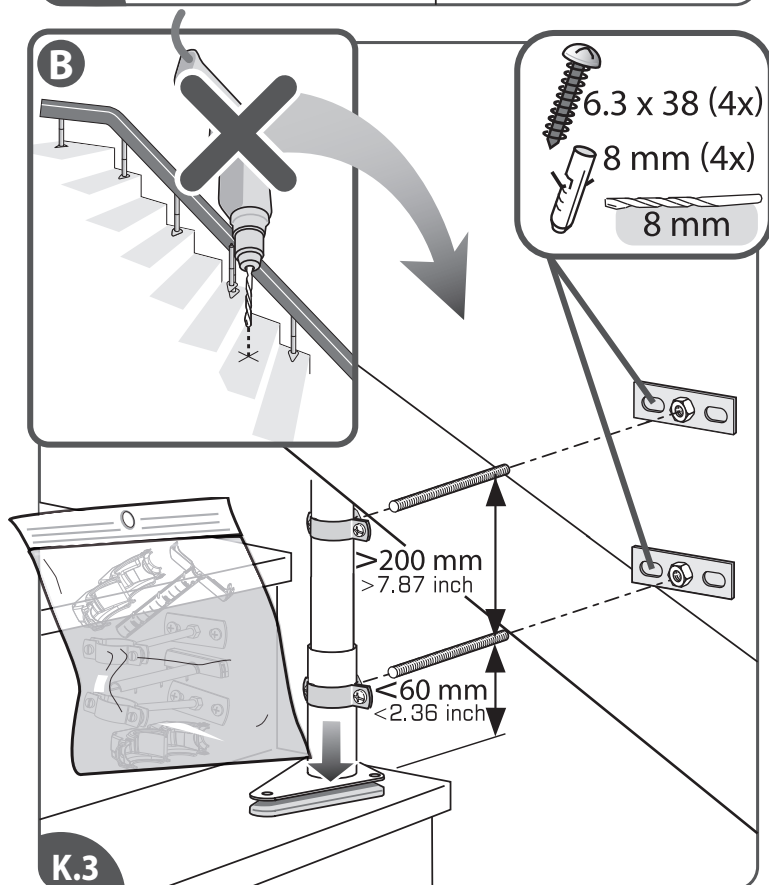
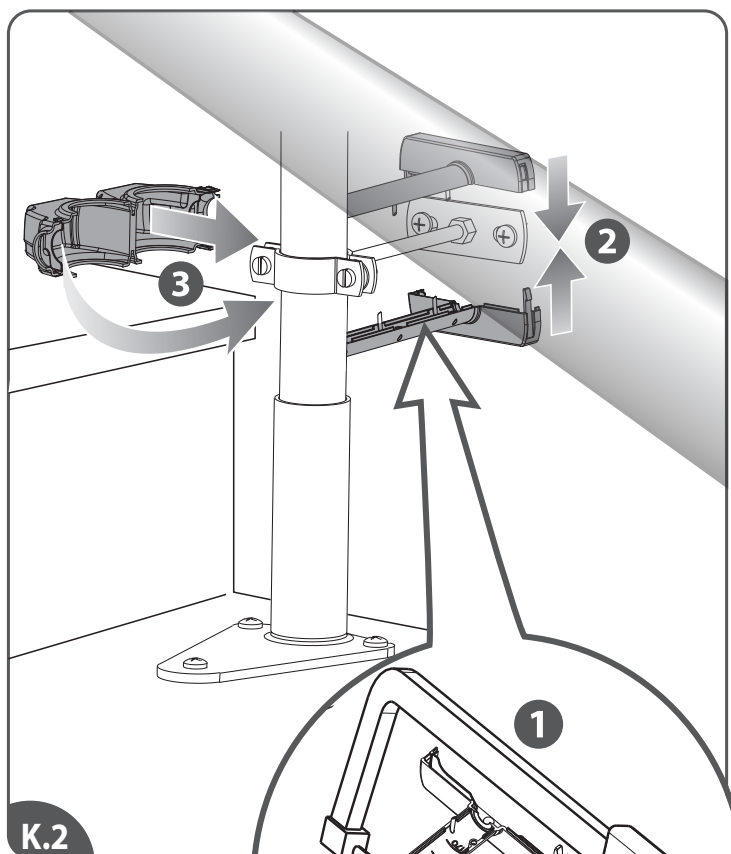
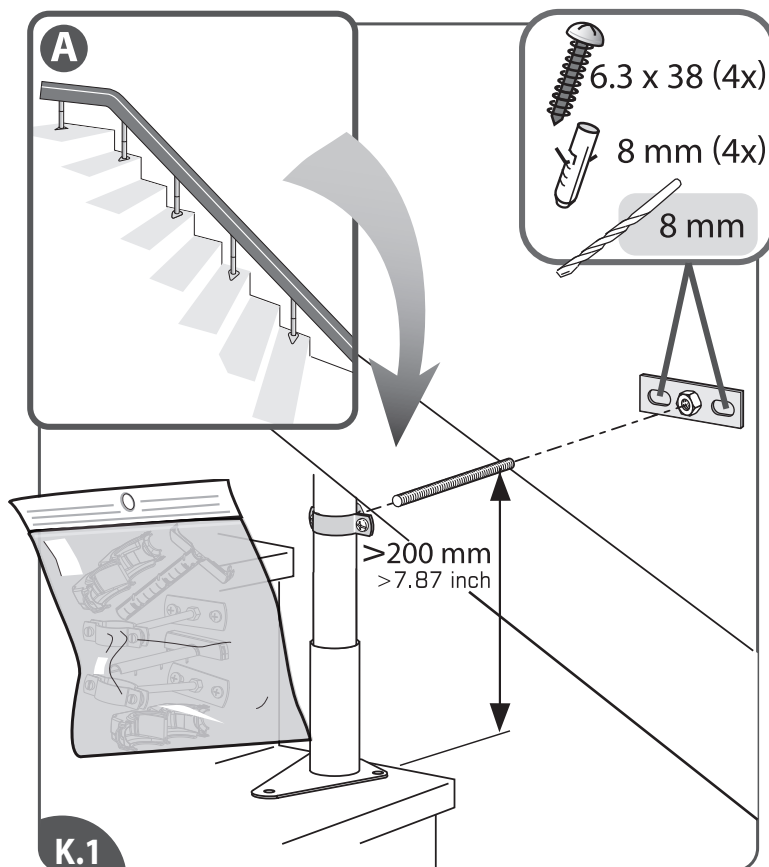
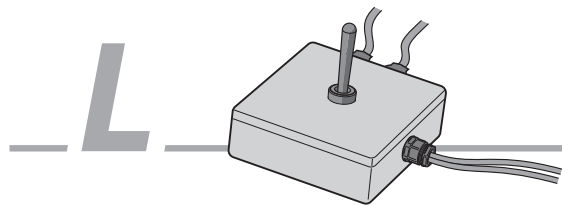
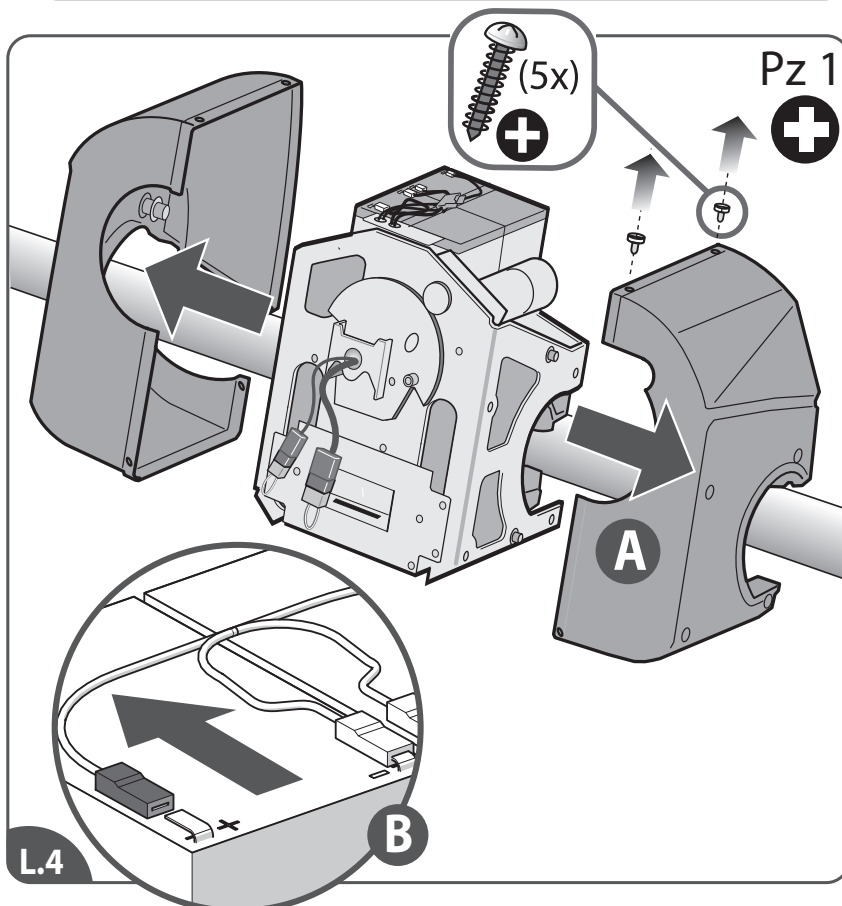
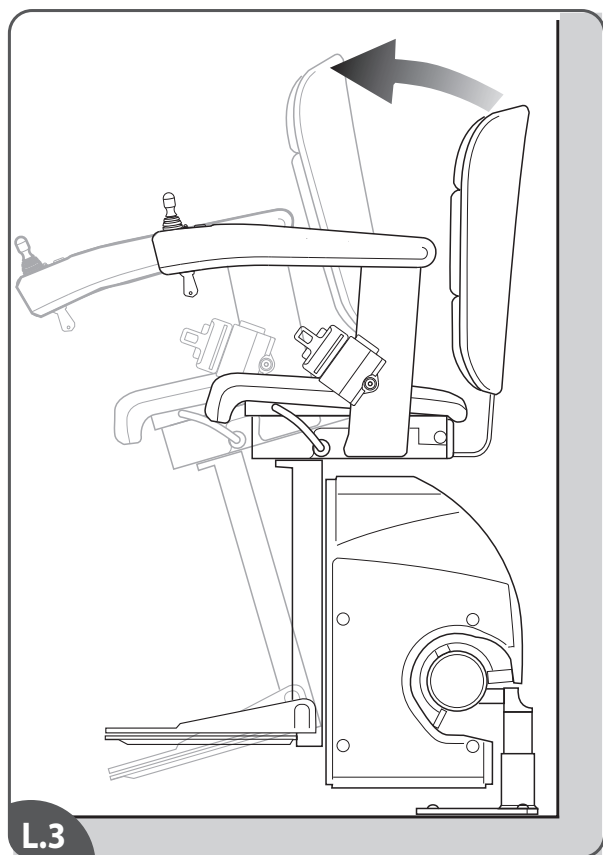
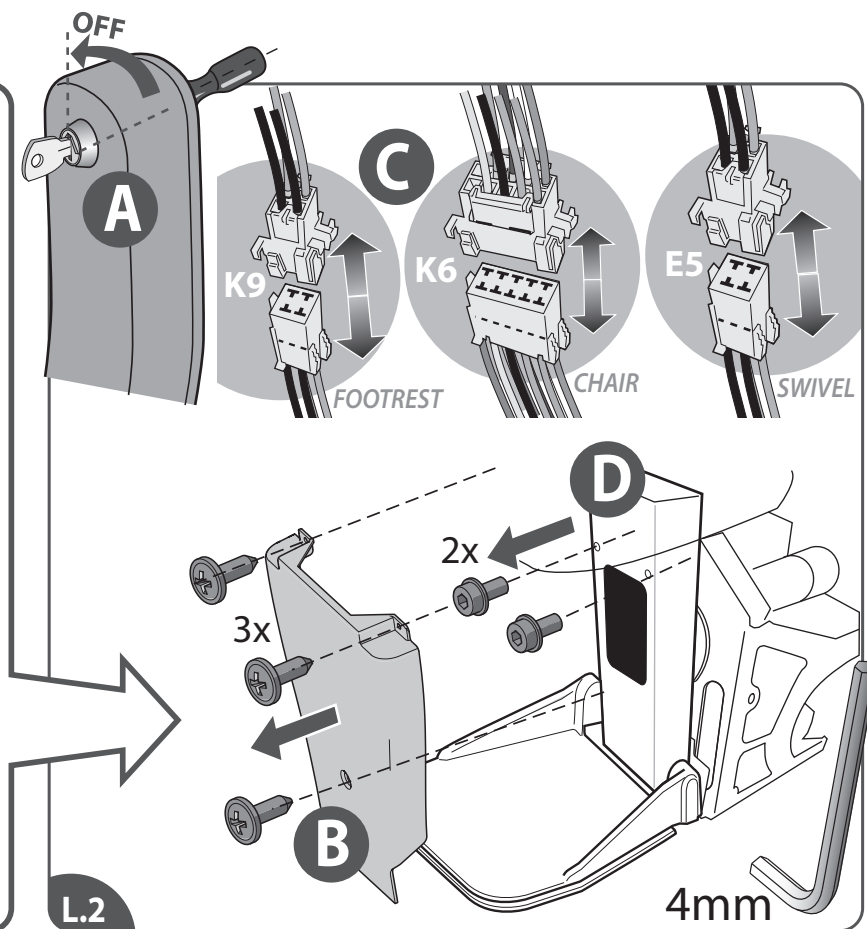
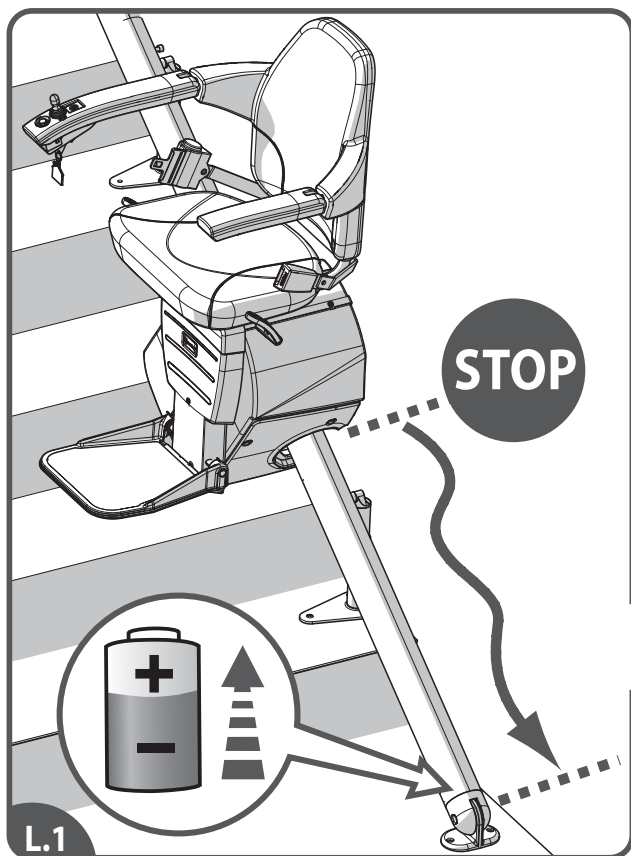
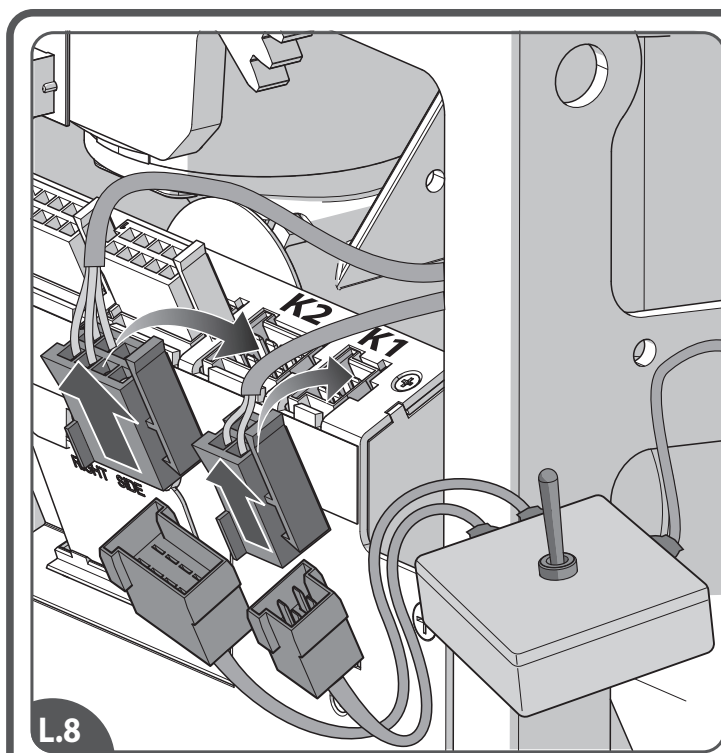
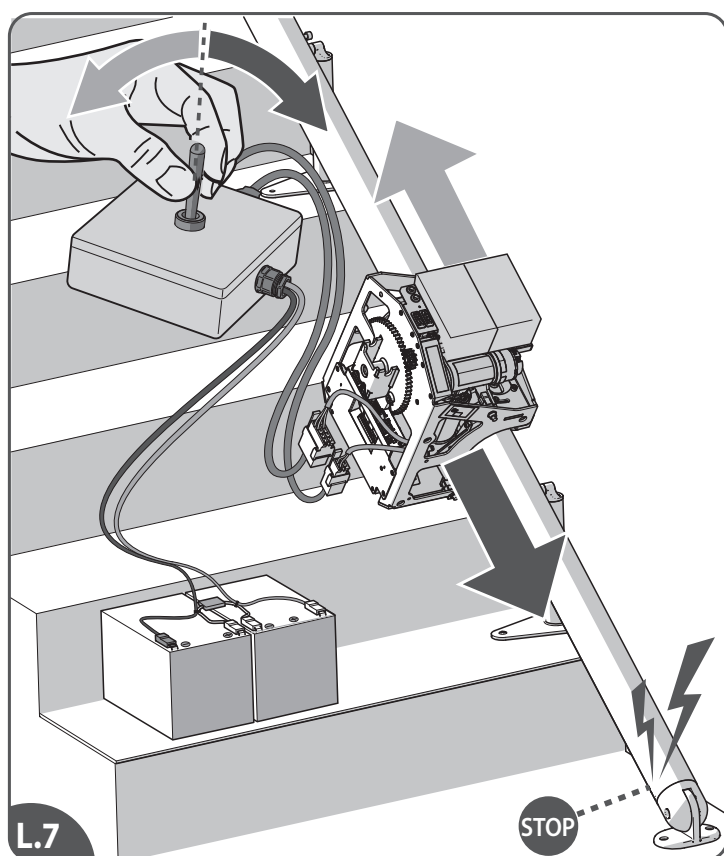
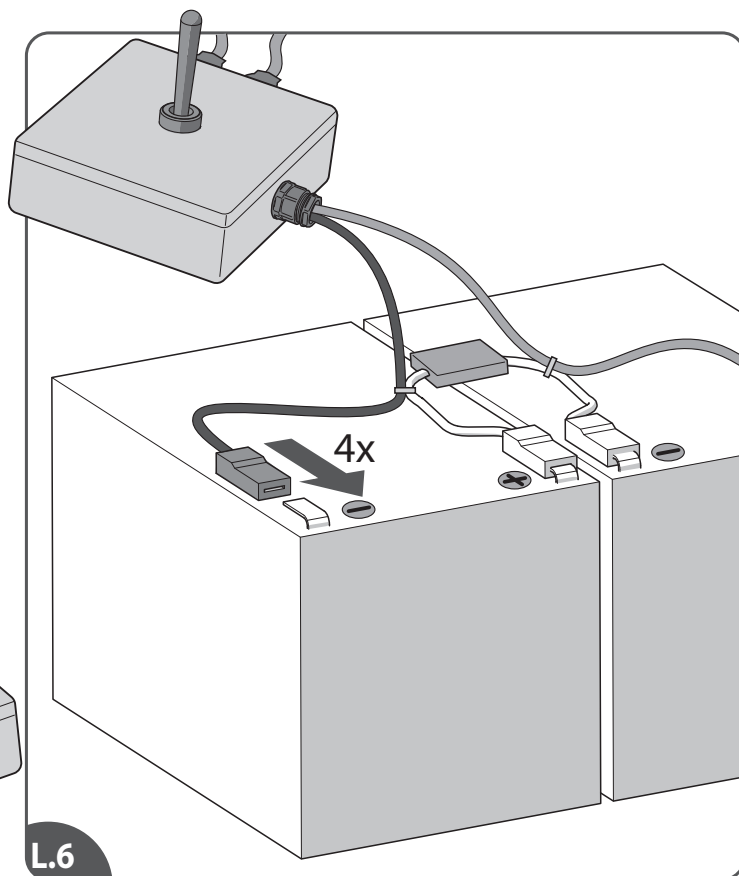
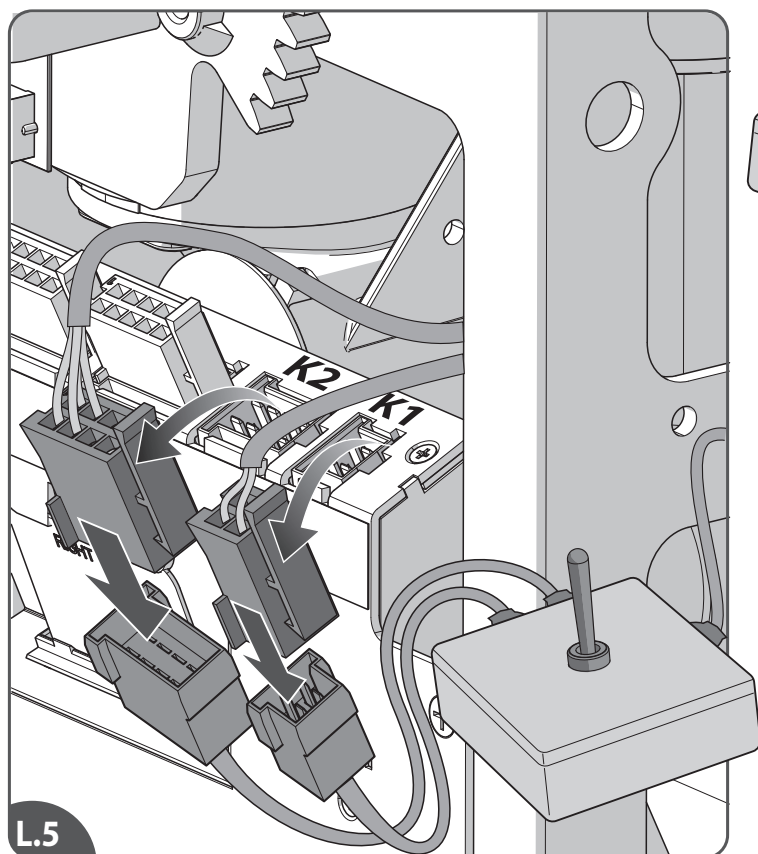
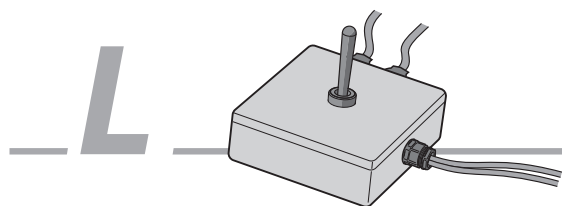
Prism Medical Inc.

10888 Metro Court
St. Louis, MO 63043
United States of America
Toll Free: 1.866.891.6502
Local: 314.692.9135
Fax: 314.692.7858
info@prismmedicalinc.com

INSTALLATION MANUAL

MANUAL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONANLEITUNG
MANUALE PER L'INSTALLAZIONE
MANUAL DE INSTALACION
MANUAL DE INSTALACAO
INSTALLASJONSMANUALEN
INSTALLATIONS MANUAL
INSTRUKCJA

11P00045_G1

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Handicare GmbH